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
Prüfgerät PG 041

(Type 007)

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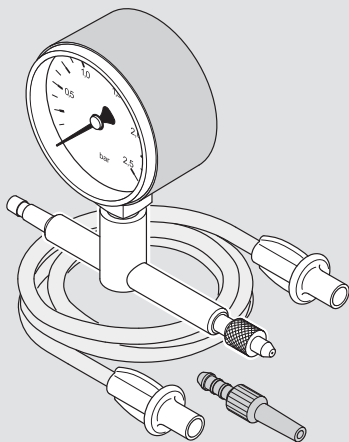 The symbol consists of the letters 'REF' in a bold, sans-serif font, enclosed within a square border.	Artikelnummer
 The symbol is a stylized black silhouette of a factory with three chimneys and a tall chimney on the right side.	Hersteller

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
(Type 007)

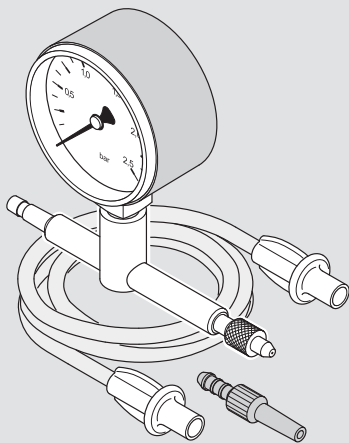Analogue pressure gauge

Read all the instructions for use carefully before using the product. Follow all instructions and safety instructions. Keep the instructions for use in a safe place.

Validity of instructions for use

PG 041 test device

Contact

Email: info@pari.de

Tel.: +49 (0)8151-279 220 (international)

+49 (0)8151-279 279 (German)

Copyright

©2021 PARI GmbH Specialists in effective inhalation
Moosstraße 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved.

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No part of this documentation may be reproduced or processed, duplicated, translated or disseminated by electronic means without the prior, written consent of INQUA GmbH.

All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

Trade marks

Registered trade marks of PARI GmbH Specialists for effective inhalation in Germany and/or other countries:

The following brand names are registered trademarks of INQUA GmbH in Germany and/or other countries.

PARI®, PARI BOY®

TABLE OF CONTENTS

IMPORTANT INFORMATION	4
Intended use.....	4
Safety instructions.....	4
PRODUCT DESCRIPTION	5
Components.....	5
Description of function.....	6
Calibration.....	6
Operating life.....	6
APPLICATION	7
Connecting the pressure gauge.....	7
Checking the operating pressure.....	7
REPROCESSING	9
Preparation.....	9
Cleaning.....	9
Inspecting.....	9
Storage.....	9
TECHNICAL DATA	10
Pressure gauge.....	10
Test nozzle.....	10
Ambient conditions.....	10
MISCELLANEOUS	11
Disposal.....	11
Links.....	11
Labelling.....	11

1 IMPORTANT INFORMATION

1.1 Intended use

The PG 041 is an accessory for PARI compressors. It is used to measure the operating pressure of a compressor, thus checking that the compressor is in working order.

It must be used only for PARI compressors.

1.2 Safety instructions

The present instructions for use contain important information, safety instructions and precautionary measures. The user must follow these in order to guarantee safe operation of this PARI product.

Use this PARI product only as described in these instructions for use.

The instructions for use for the PARI product that is to be tested must also be followed.

Hazard due to small parts which can be swallowed

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Keep all components of the product out of reach of babies and infants at all times.

Identifying and classifying warning instructions

Safety-critical warnings are categorised according to the following hazard levels in these instructions for use:

DANGER

DANGER indicates a hazardous situation which will lead to severe injuries or death if it is not avoided.

WARNING

WARNING indicates a hazardous situation which can lead to severe injuries or death if it is not avoided.

⚠ CAUTION

CAUTION indicates a hazardous situation which can lead to mild or moderate injuries if it is not avoided.

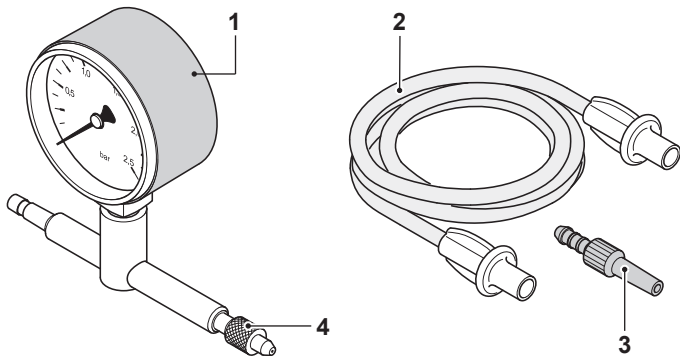
NOTE

NOTE indicates a situation which can lead to material damage if it is not avoided.

2 PRODUCT DESCRIPTION

2.1 Components

The following components are included in the package:

- (1) Pressure gauge, 0 to 2.5 bar
- (2) Connection tubing
- (3) Tubing adapter
- (4) Test nozzle (\varnothing 0.48 mm)

2.2 Description of function

The PG 041 is an analogue pressure gauge. It is connected to the air connection of a PARI compressor using connection tubing. The compressed air generated by the compressor is channelled through the pressure gauge via the tube. The test nozzle, inserted into the air outlet of the PG 041, generates atmospheric pressure. The measured value is displayed on the scale of the pressure gauge. The diameter of the test nozzle is 0.48 mm, the same as the PARI nebuliser nozzle. The test nozzle simulates the expiratory resistance of a PARI nebuliser.

2.3 Calibration


The PG 041 is used for measurements and as such should be calibrated for the first time after 3 years and then once every 2 years. For this, please contact PARI GMBH.

2.4 Operating life

Product component	Operating life
Connection tubing	max. 1 year
Tubing adapter	Max. 1 year

3 APPLICATION

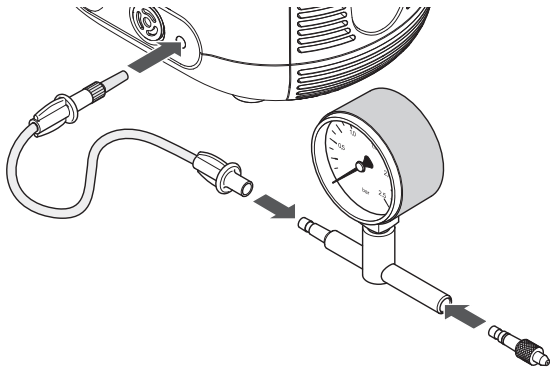3.1 Connecting the pressure gauge

- Fit the test nozzle onto the appropriate connector on the pressure gauge.
- Fit the connection tubing onto the appropriate connector on the pressure gauge.
- Connect the other end of the connection tubing to the air connection on the compressor.
Depending on the compressor, the tubing adapter may need to be used.

3.2 Checking the operating pressure

Check the operating pressure of PARI compressors once a year. For rented devices, check the operating pressure every time they are rented. This is the only way to ensure it is in good working order.

- Switch on the compressor and leave it running for at least 10 seconds.
- Read the operating pressure measured on the pressure gauge.
- If the operating pressure is less than the values stated in the table below, contact PARI Service.

NOTE**Incorrect measurement results caused by a defective pressure gauge**

If the pressure gauge is damaged, the measurement results displayed may be incorrect.

- If the PG 041 has been dropped or suffered a similar impact, send it to PARI GmbH for testing, which is subject to a fee.

Type	Compressor name	Minimum pressure
028	PARI SINUS	1.2 bar
047	PARI BOY mobile S PARI BASIC	0.7 bar
052	PARI COMPACT PARI COMPACT Junior	0.7 bar
053	PARI BOY S	0.8 bar
084	PARI Master	1.4 bar
085	PARI BOY SX PARI XLent PARI BOY N	1.2 bar
128	PARI SINUS2	1.2 bar
130	PARI BOY Classic PARI BOY Pro PARI BOY Junior	1.2 bar
152	PARI COMPACT2 PARI COMPACT2 Junior	0.8 bar
430	OEM compressor	0.8 bar

4 REPROCESSING

4.1 Preparation

- Disconnect the connection tubing (together with the tubing adapter if used) from the air connection of the compressor.
- Remove the tubing adapter from the connection tubing.
- Disconnect the connection tubing from the pressure gauge.
- Remove the test nozzle from the pressure gauge.

4.2 Cleaning

Wipe the pressure gauge, the connection tubing, the test nozzle and the tubing adapter with a damp cloth.

Replace the connection tubing if these have visible signs of dirt.

Take care to ensure that water does not get into the connections on the pressure gauge or into the test nozzle.

4.3 Inspecting

Inspect all components every time they have been cleaned.

Replace any broken, misshapen or seriously discoloured parts.

4.4 Storage

Store this product in a dry, dust-free environment.

5 TECHNICAL DATA

5.1 Pressure gauge

Display area	0 bar to 2.5 bar
Accuracy class	2.5
Pressure gauge standard	International (Europe)
Measuring system	Reed pen

5.2 Test nozzle

The test nozzle has a tolerance of +3 %/-1 % at an atmospheric pressure of 1,000 hPa.

5.3 Ambient conditions

During operation

See the instructions for use of the compressor used for the ambient conditions for operation.

During transportation and storage

Minimum ambient temperature (without monitoring of relative humidity)	-25 °C
Maximum ambient temperature (with relative humidity of up to 93%, non-condensing)	+70 °C
Humidity	max. 93%
Atmospheric pressure	500 hPa – 1,060 hPa

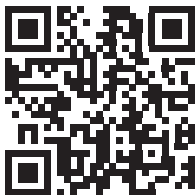
6 MISCELLANEOUS

6.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

All product components must be disposed of in accordance with the Ordinance on the European Waste Catalogue (EWC). Where applicable, country-specific disposal regulations must also be observed.

6.2 Links

Terms and conditions of warranty:
www.pari.com/garantiebedingungen

6.3 Labelling

The following symbols can be found on the product and/or the packaging:

 The symbol consists of the letters 'REF' in a bold, sans-serif font, enclosed within a square border.	Item number
 The symbol is a stylized black silhouette of a factory with three gabled roofs and a tall chimney on the right side.	Legal manufacturer

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©2021 PARI GmbH Spezialisten für effektive Inhalation, 007D2003-A 2021-03-17