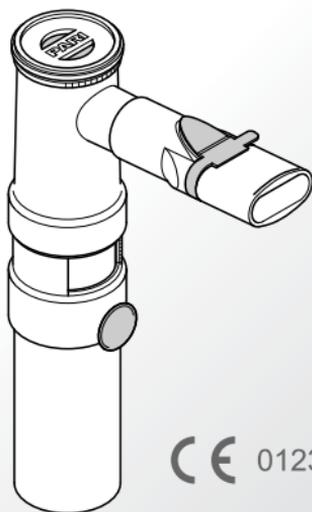
LL Vernebler

CE 0123

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en Instructions for use 25

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Instructions for use

Information as of: 2012-12. We reserve the right to make technical changes.

IMPORTANT INFORMATION	26
General	26
Information about the instructions for use	26
Structure of safety instructions	26
Treatment of children and anyone who requires assistance	27
Hygiene	27
PRODUCT DESCRIPTION	28
Components	28
Material information	29
Product combinations	29
Intended use	29
Operating life	30
INHALATION	31
Assembling the nebuliser	31
Filling with medication	33
Performing the inhalation	34
CLEANING AND DISINFECTION	36
Preparation	36
Care of the connection tubing	38
Cleaning	38
Disinfection	39
Cleaning the nozzle	40
Visual inspection	41
Drying and storage	41
MISCELLANEOUS	42
Disposal	42
Contact	42
Technical data	42
REPLACEMENT SETS, SPARE PARTS AND ACCESSORIES	43

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

If symptoms persist or if your condition worsens, discontinue the inhalation treatment and contact your doctor immediately.

1.2 Information about the instructions for use

These instructions for use are intended for the user at home. For medical specialists, a copy of instructions for hygienic re-use in a professional environment is included with the product.

All instructions for use can be retrieved on the internet at www.pari.de/en/products (on the respective product page), or they are available in hardcopy form on request from PARI GmbH. **Check for updated versions of all instructions for use at regular intervals.**

1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.4 Treatment of children and anyone who requires assistance

Children and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. This is the only way to ensure safe and effective treatment. Individuals in this group often underestimate the hazards involved (e.g. strangulation with the power cord or the connection tubing), thus resulting in a risk of injury.

The product contains small parts. Small parts can block the respiratory tract and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

1.5 Hygiene

Only use nebulisers and accessories that have been cleaned, disinfected and dried as described in chapter 4 for inhalation therapy. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection

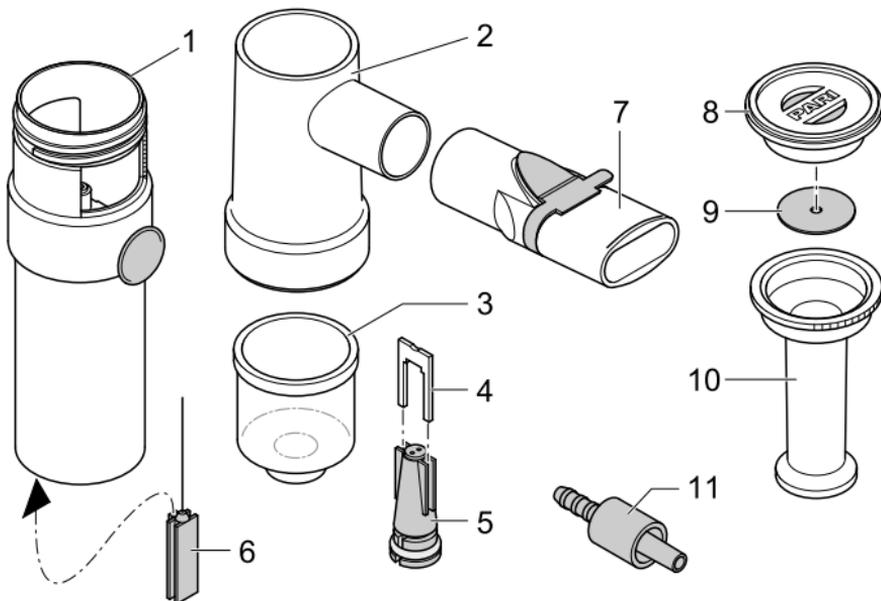
Therefore, please observe the following hygiene instructions:

- Each nebuliser must only be used at home by a single patient for hygiene reasons.
- Before every use and cleaning wash your hands thoroughly.
- Clean, disinfect and dry your nebuliser and accessories. Make absolutely sure you also carry out cleaning, disinfection and drying before using the device for the first time.
- For cleaning and disinfection always use fresh tap water of drinking water quality.
- Make sure all components are dried properly whenever you have performed cleaning or disinfection.
- Do not keep the nebuliser or accessories in a damp environment or together with damp objects.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.

- (1) Nebuliser lower section with interrupter button
- (2) Nebuliser upper section
- (3) Medication cup
- (4) Airflow control
- (5) Nozzle
- (6) Nozzle cleaner
- (7) Mouthpiece
- (8) Inspiratory valve
- (9) Valve shim
- (10) Chamber
- (11) LL adapter

2.2 Material information

Polypropylene	Inspiratory valve, nebuliser upper section, nebuliser lower section, chamber, medication cup, mouthpiece
Silicone	Valve shim
Polyamide	LL adapter
Fluoroelastomer	Nozzle gasket
Stainless steel	Airflow control, compression spring (for the interrupter button)

2.3 Product combinations

The PARI LL nebuliser can be operated with all PARI compressors that are available on the market (for example the PARI BOY S, PARI BOY SX, PARI BOY N, PARI MASTER etc.).

The manufacturer will be able to tell you whether the PARI LL nebuliser can be used with new PARI compressors or PARI CENTRALs that are introduced in the future. This information will also appear in the respective instructions for use.

2.4 Intended use

The PARI LL nebuliser is a medication nebuliser that is designed to be used in conjunction with a PARI compressor or the PARI CENTRAL and the central compressed air supply for inhalation therapy of the lower respiratory tract.

The interrupter button can be used for intermittent nebulising, which ensures that optimum use is made of the medication.

Each application generally takes between 5 and 10 minutes, but in no case more than 20 minutes (depending on the quantity of medication).

The nebulisers are only designed for patients who are able to breathe by themselves and are conscious. This is the only way to ensure effective treatment and avoid a choking hazard.

Only medication that has been approved for inhalation treatment must be used. This is generally determined by the doctor. Take note of any restrictions in the instructions for use of the medication in question. The frequency of application depends on the clinical picture and the medication used.

Please read and follow the instructions for use of the compressor and of the accessories you are using.

2.5 Operating life

Replace the nebuliser after two years at the latest as changes can occur in the material of the nebuliser over time. This might impair the effectiveness of the treatment.

3 INHALATION

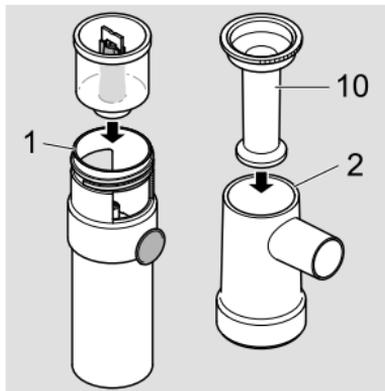
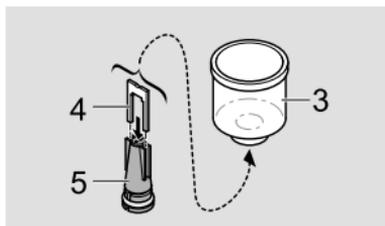
Inhalation with a mouthpiece is the most effective way of treating the lower respiratory tract as this allows the greatest possible quantity of medication to reach the lung. If it is not possible to use a mouthpiece, or if the patient prefers to inhale through a mask, PARI can provide the right accessories.

3.1 Assembling the nebuliser


CAUTION

Check all nebuliser components and the accessories before each use. Replace any broken, misshapen or seriously discoloured parts. Please also follow the instructions for assembly given below. Damaged components and/or an incorrectly assembled nebuliser may impair **functioning of the nebuliser** and thus **treatment as well**.

- Push the airflow control (4) as far as it will go onto the nozzle (5) and then push both parts into the medication cup (3) from below.
- Insert the medication cup in the nebuliser lower section (1).
- Slide the chamber (10) into the nebuliser upper section (2).

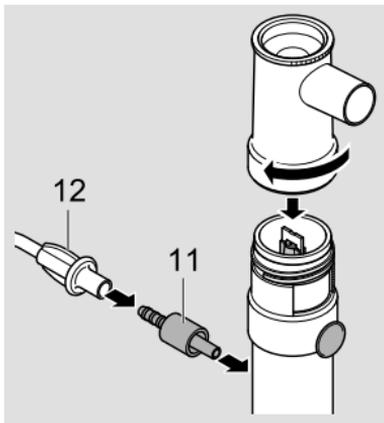
- Screw the nebuliser upper section onto the nebuliser lower section.
- Attach the LL adapter (11) to the nebuliser lower section.
- Fit the connection tubing (12) onto the LL adapter.


Using with a mouthpiece

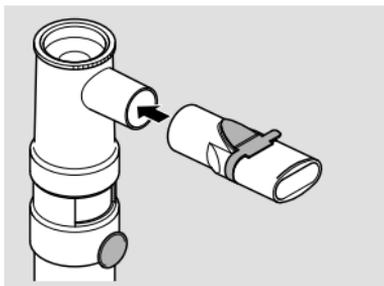- Attach the mouthpiece to the nebuliser upper section.

Using with a child or adult mask

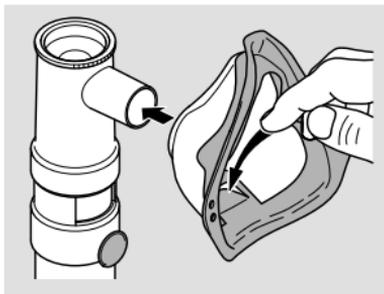
- Attach the child or adult mask to the nebuliser upper section.


i When using a mask, make sure that the expiratory valve plate is pressed out so you can exhale freely during inhalation.

3.2 Filling with medication

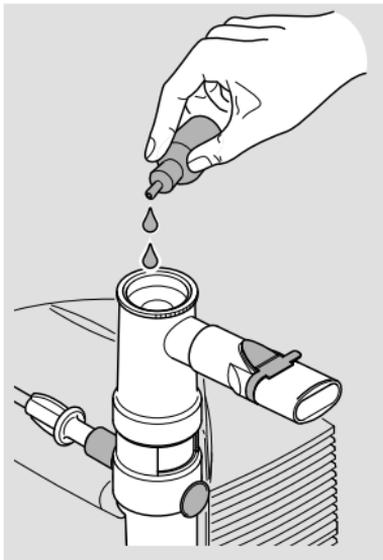- Pour the quantity of medication prescribed by your doctor into the nebuliser chamber. Ensure that the medication is filled no higher than the upper scale marking (max. level 8 ml). Overfilling can greatly impair the nebulisation of medication.

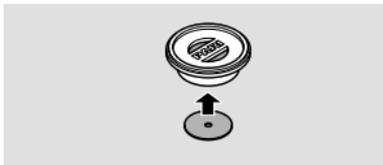- Put the inspiratory valve with the valve shim on the nebuliser.

If you are to inhale several medications one after the other in one application, rinse out the nebuliser with tap water between inhalations. Remove excess water by shaking the nebuliser and then fill with medication as described.

3.3 Performing the inhalation

Before the patient starts inhaling, ensure that all parts are firmly connected to each other.

Perform inhalation until the medication is used up (indicated by a change in the sound of the nebuliser).

- Several drops of medication will remain in the nebuliser after inhalation.

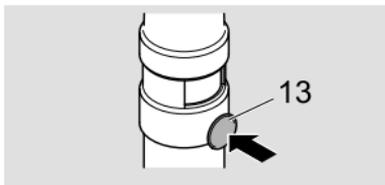
When using a PARI compressor:

- Connect the connection tubing to the compressor and switch the compressor on.

When using with a PARI CENTRAL:

Prerequisite: A PARI CENTRAL must be connected to the wall outlet fitting in the hospital.

- Insert the connection tubing into the PARI CENTRAL using the adapter provided for this purpose. Twist slightly to make the connection easier.
- Sit in an upright position and relax.
- Take the mouthpiece between your teeth and close your lips around it or press the mask lightly over the mouth and nose.
- Press the interrupter button (13).
- Breathe in through the mouthpiece or mask as slowly and deeply as possible.
- Release the interrupter button and exhale in a relaxed manner through the mouthpiece or mask.

- For continuous nebulising, press the interrupter switch and turn it gently clockwise to lock it in place.

- Continue breathing in and out as described until a change in the sound of the nebuliser signals that the medicine has been used up.
- Check whether a clearly visible aerosol is still flowing from the nebuliser (press interrupter button).
- Stop treatment as soon as the aerosol emerges irregularly.
- As soon as your inhalation session is over, switch the compressor off and disconnect the connection tubing together with the adapter from the PARI CENTRAL.

4 CLEANING AND DISINFECTION

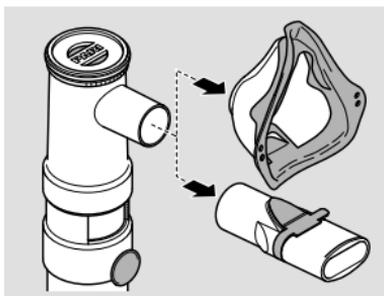
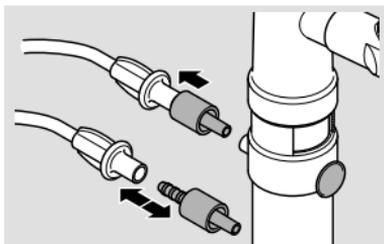
This chapter describes how to clean and disinfect the nebuliser when it is used in the home. A separate set of instructions for hygienic re-use in the professional environment is included with the nebuliser.

- i** The nebuliser and the accessories used must be cleaned after each application and disinfected at least once a day. The connection tubing cannot be cleaned or disinfected.


4.1 Preparation

Dismantle the nebuliser into its individual parts:


- Pull the connection tubing together with the LL adapter off the nebuliser.
- Disconnect the LL adapter from the connection tubing.
- Pull the mouthpiece or mask off the nebuliser.

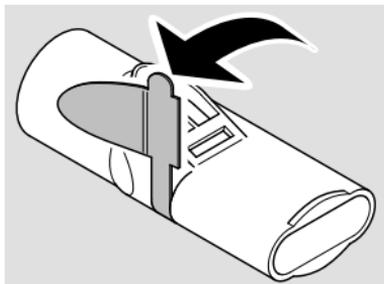
- Carefully pull the expiratory valve out of the slot in the mouthpiece.
The valve must still be attached to the mouthpiece.

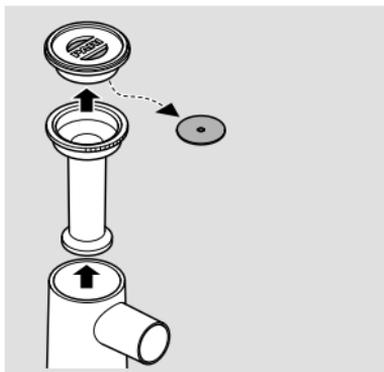- Detach the inspiratory valve and the chamber from the nebuliser.
- Remove the valve shim from the inspiratory valve.

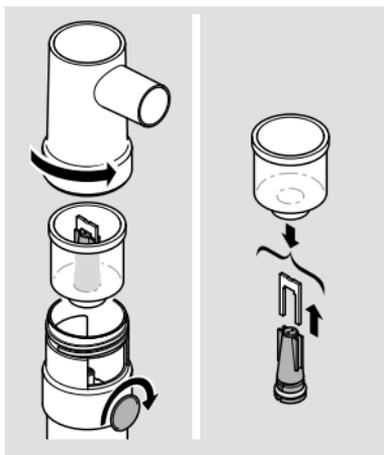- Unscrew the nebuliser upper section from the nebuliser lower section.
- Push the medication cup up as far as the windows and then pull it out of the nebuliser lower section.
- Press the nozzle together with the airflow control down and out of the medication cup.
- Pull the airflow control up and off from the nozzle.
- Turn the interrupter button clockwise as far as it will go.


4.2 Care of the connection tubing

- Connect the connection tubing to the compressor.
- Switch the compressor on.
- Leave the compressor running until any condensation in the tubing has been removed by the air flowing into the tube.

4.3 Cleaning

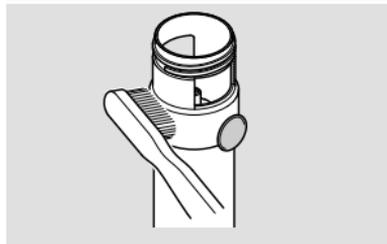- Thoroughly clean all components (except for the connection tubing) in warm tap water and dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose dirt (the brush must be reserved exclusively for this purpose).
- Rinse all parts thoroughly under warm running water.
- You can remove excess water more quickly by shaking all parts.

4.4 Disinfection

After cleaning, disinfect the dismantled nebuliser and accessories used (except for the connection tubing) (effective disinfection is only possible if the nebuliser has been cleaned).

● Information when using masks with rubber bands:

I The rubber band can only be cleaned as it might be damaged by the high temperatures used during disinfection.

CAUTION

A damp environment may encourage the growth of bacteria. Therefore, remove all parts from the pot or disinfector as soon as disinfection has finished. Dry the parts. The **risk of infection** is reduced when the parts are dried completely.

In boiling water

- Place all the individual parts in boiling water for at least 5 minutes.

Use a clean pot and fresh drinking water.

● If your water is very hard, scale may be deposited on the plastic parts as a milky coating. This can be prevented by using softened water.

NOTICE

Plastic will melt if it comes into contact with the hot base of the pot. Therefore, make sure there is plenty of water in the pot. This way you will avoid **damaging the components**.

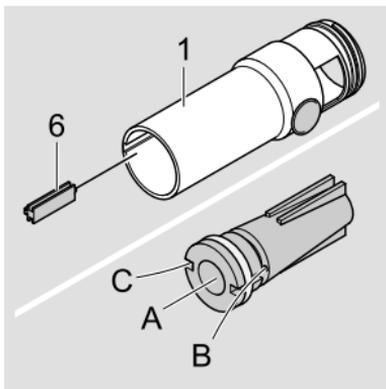
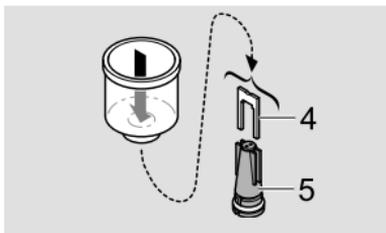
Using a standard thermal disinfectant for baby bottles (not a microwave oven)

Disinfection must last at least 6 minutes. Regarding disinfection and the quantity of water required for this, it is essential to follow the instructions for use of the disinfectant you are using. Always check carefully that it is clean and fully functional.

4.5 Cleaning the nozzle

Should the nozzle be blocked, you can clear it yourself as follows:

- Unscrew the nebuliser upper section from the nebuliser lower section.
- Push the medication cup up as far as the windows and then pull it out of the nebuliser lower section.
- Press the nozzle (5) together with the airflow control (4) down and out of the medication cup.
- Pull the airflow control up and off from the nozzle.
- Take the nozzle cleaner (6) out of the nebuliser lower section (1).
- Carefully poke through the three nozzle bore holes "A" to "C" (only from below).
- Reassemble the nebuliser.

NOTICE

There is a **risk of damaging the nozzle bore holes irreparably**. Therefore, always use only an original PARI nozzle cleaner and proceed extremely carefully. If the nozzle becomes blocked repeatedly, replace the nozzle.

4.6 Visual inspection

Inspect all product components after each cleaning and disinfection. Replace any broken, misshapen or seriously discoloured parts.

4.7 Drying and storage

- You can remove excess water more quickly by shaking all parts. Remember to hold the cap of the nebuliser upper section, otherwise the hinge may be damaged by shaking.
- Place all parts on a dry, clean and absorbent surface and let them dry completely.
- For drying, between applications, and in particular during long breaks, wrap the nebuliser in a clean, lint-free cloth (e.g. a tea-towel) and keep in a dry, dust-free environment.
- During storage, make sure the nebuliser and accessories are protected from continuous exposure to direct sunlight.

5 MISCELLANEOUS

5.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

5.2 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)
+49 (0)8151-279 220 (international)

5.3 Technical data

Operating gases	Air, oxygen
Minimum flow	3.5 l/min.
Corresponding minimum operating pressure	0.8 bar / 80 kPa
Maximum flow	6.0 l/min.
Corresponding maximum operating pressure	2.0 bar / 200 kPa
Minimum fill volume	2 ml
Maximum fill volume	8 ml

Aerosol data according to DIN EN 3544-1 is available on request from PARI GmbH or on the internet at www.pari.de/en/products (under "Technical data" on the respective product page).

6 REPLACEMENT SETS, SPARE PARTS AND ACCESSORIES

Description	Item no.
Replacement sets:	
PARI LL nebuliser Year Pack	012G6310
Spare parts and accessories:	
PARI LL nebulisers	012G6202
Nebuliser upper section	012B1800
Nebuliser lower section	012B1200
Medication cup	012E1601
Nozzle	019B2110
Inspiratory valve	022B0510
Universal mouthpiece	022E3050
LL adapter	012E1024
Connection tubing for PARI compressors	041B4591
Adult mask soft	041G0740
Child mask soft "Spiggy"	041G0741
PARI SMARTMASK (including knee)	041G0730
PARI Filter/Valve Set	041G0500
PARI PEP System I	018G6100
PARI PEP System II	018G6200
PARI PEP S System	018G4000
PARI TRACHEO Set	025G0007

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