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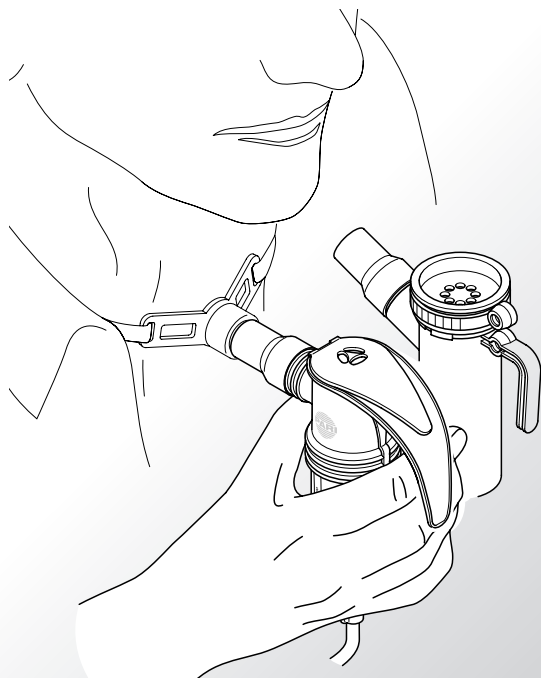
Information as of: February 2012

We reserve the right to make technical changes.

Important information

The tracheal adapter must not be used by persons who have not received competent instruction. Please read these instructions for use and the instructions for use of the respective nebuliser carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

Intended use

The tracheal adapter is intended for use with PARI nebulisers (except the PARI TIA and PARI LC SPRINT SINUS). The adapter is for use by patients who have undergone tracheostomy and are unable to inhale through a mouthpiece. It serves to connect the nebuliser to the tracheal cannula. This enables these patients to use the nebulisers in inhalation treatment of the lower respiratory tract.

Information

The tracheal adapter is suitable for application with blocked and unblocked tracheal cannulae. However, it must not be used on an open tracheostoma.

Nebulisers with a tracheal adapter should not be used except by patients who are conscious and able to breathe unaided.

The patient should be sitting up during treatment.

Clean the tracheal adapter together with the nebuliser after each use as described in the instructions for use of the nebuliser.

Materials used: Polypropylene

Application

- Prepare the nebuliser for inhalation (as described in the instructions for the nebuliser you are using).
- Hold the nebuliser upright.

Figure 1:

- Place the tracheal adapter on the nebuliser.

Figure 2:

- Attach the tracheal adapter to the tracheal cannula.

Figure 3:

- Hold the nebuliser as shown in the figure.
- Perform the inhalation as described in the instructions for the nebuliser you are using.

Figure 4:

- Disconnect the tracheal adapter and the nebuliser from the tracheal cannula.

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PARI GmbH