
PARI CENTRAL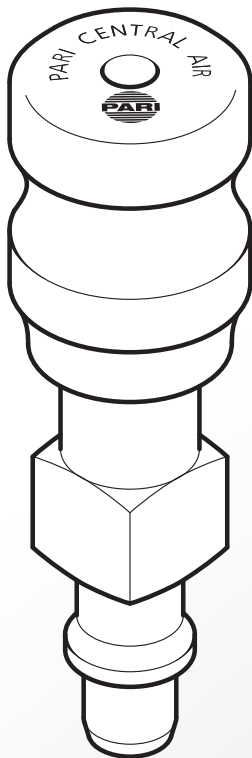

CE 0123

de	Gebrauchsanweisung.....	3
en	Instructions for use	17
fi	Instructions for use	31
fr	Notice d'utilisation	45
sv	Bruksanvisning	59

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Stand der Information: August 2010

en Instructions for use

Read these instructions carefully and completely before using for the first time. Keep these instructions in a safe place.

Risks caused by failure to comply with the instructions:

Failure to comply with the instructions may cause personal injury and/or damage the product. Specifically, failure to follow the instructions may have the following dangerous consequences:


- Failure of important functions of the product – resulting in inefficient treatment
- Danger of personal injury from mechanical or other effects – in all variations and with all associated consequences

Table of contents

1	Components	19
2	Important information.....	20
3	Intended use.....	21
4	Preparation for use	22
5	Cleaning.....	23
6	Disposal	23
7	Technical data	24
8	Transportation and storage	26
9	Troubleshooting.....	26
10	Terms and conditions of warranty	28
11	Explanation of symbols.....	29
12	Spare parts and accessories	29

1 Components

Please check that your PARI CENTRAL contains all the following components. If not, please notify the dealer from whom you obtained this PARI CENTRAL immediately.

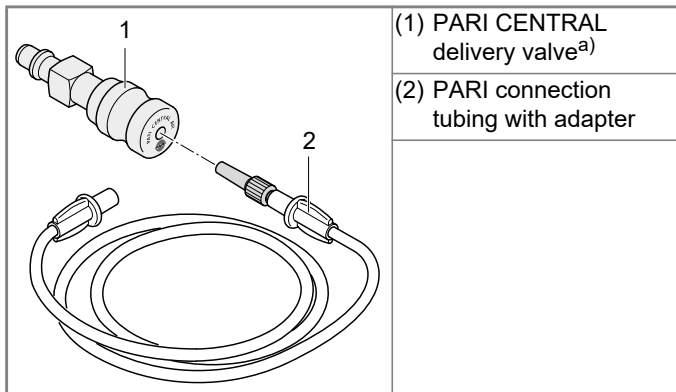a) Figure: PARI CENTRAL AIR, DIN 13260-2. The figure is provided as an example only, and may differ from the version purchased.

2 Important information

The PARI CENTRAL cannot be operated unless the design of the connectors for the central gas supply conforms to the applicable standard (e.g., DIN 13260-2, SS 87 524 30 or NF S90-116).

The PARI CENTRAL must only be used in conjunction with the PARI connection tubing and other compatible PARI accessories.


- The PARI CENTRAL O2 must not come into contact with lubricants, greases or oils of any kind.
- Care must be taken to ensure that the PARI CENTRAL is disconnected from the central gas supply and placed out of reach of the patient after each use.
- The PARI CENTRAL must not be installed, removed or used except by medical technical personnel or operators trained for this purpose.
- Whenever the PARI CENTRAL is used, a thorough knowledge of the instructions for use and compliance with the same is assumed. In all cases, the operator is responsible for the safe operation of the PARI CENTRAL if it is not used appropriately for its intended purpose. Moreover, PARI GmbH accepts no liability for any operations carried out on the PARI CENTRAL.
- Make sure that the tubing does not kink as this might deprive patients dependent on O2 of an adequate supply.

3 Intended use

The PARI CENTRAL is a delivery valve for connection to the pipe system for medical gases in a hospital or medical practice. It is used to reduce the gas pressure and flow rate.

All PARI nebulisers (except the PARI LC PLUS UNIVERSAL) can be connected to the PARI CENTRAL using the PARI CENTRAL connection tubing. During operation, the PARI CENTRAL ensures that the PARI nebuliser receives the operating pressure for which it is designed.

4 Preparation for use

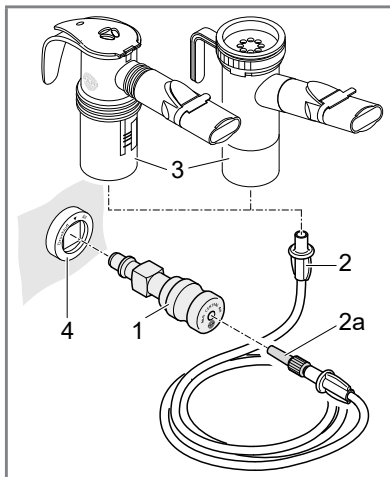To prepare the PARI CENTRAL for use, proceed as follows:

- Connect the PARI CENTRAL (1) to the central gas supply outlet (4) provided for this purpose.
- Connect the connection tubing (2) to the PARI nebuliser (3) and ensure that tubing and nebuliser are connected properly.

- Start inhalation by inserting the adapter (2a) on the PARI connection tubing (2) in the PARI CENTRAL.

Twist the adapter slightly as you insert it; this will hold the connection tubing in place more firmly. At the same time, the valve in the PARI CENTRAL will open.

After treatment, twist the connection tubing slightly as you pull it out of the PARI CENTRAL. This will cut off the supply of air or oxygen to the nebuliser.

Please read the instructions for use of the nebuliser carefully and thoroughly and use the nebuliser accordingly.

5 Cleaning

Wipe the outer surface of your PARI CENTRAL with a damp cloth.

The PARI CENTRAL must not be immersed in water or sterilised under any circumstances.

Care of the connection tubing

Please read and follow the hygiene instructions in the instructions for use of the PARI nebuliser.

6 Disposal

The PARI CENTRAL must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers).

Materials recycling helps to reduce the consumption of raw materials and protect the environment.

7 Technical data

PARI CENTRAL AIR delivery valve (DIN 13260-2)

Item no. 058G1100

Inlet pressure:	500 kPa (5 bar \pm 10%)
Outlet/ operating pressure:	160 kPa (0.8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 86 mm Diameter: 26 mm Weight: 67 g

PARI CENTRAL O2 delivery valve (DIN 13260-2)

Item no. 058G1200

Inlet pressure:	500 kPa (3.6 bar to 5.5 bar)
Outlet/ operating pressure:	160 kPa (0.8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 86 mm Diameter: 26 mm Weight: 67 g

PARI CENTRAL AIR delivery valve (SS 87 524 30)

Item no. 058G1110

Inlet pressure:	500 kPa (3.6 bar to 5.5 bar)
Outlet/ operating pressure:	160 kPa (0.8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 99 mm Diameter: 26 mm Weight: 80 g

PARI CENTRAL O2 delivery valve (SS 87 524 30)

Item no. 058G1210

Inlet pressure:	500 kPa (3.6 bar to 5.5 bar)
Outlet/ operating pressure:	160 kPa (0.8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 99 mm Diameter: 26 mm Weight: 78 g

PARI CENTRAL AIR delivery valve (NF S90-116)

Item no. 058G1130

Inlet pressure:	500 kPa (3.6 bar to 5.5 bar)
Outlet/operating pressure:	160 kPa (0,8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 100 mm Diameter: 36 mm/26 mm Weight: 158 g

PARI CENTRAL O2 delivery valve (NF S90-116)


Item no. 058G1230

Inlet pressure:	500 kPa (3.6 bar to 5.5 bar)
Outlet/operating pressure:	160kPa (0.8 bar to 2.0 bar), measured with the PARI reference nozzle
Dimensions:	Length: 100mm Diameter: 36mm/26mm Weight: 158g

8 Transportation and storage

- Do not store in damp rooms or transport together with damp objects.
- Protect from continuous direct sunlight during storage and transportation.

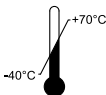Ambient temperature $-40\text{ }^{\circ}\text{C}$ to $+70\text{ }^{\circ}\text{C}$

RH

10% - 95%

Relative humidity 10% to 95% (non-condensing)

P

500hPa-1060hPa

Atmospheric pressure 500 hPa to 1060 hPa

9 Troubleshooting

The PARI CENTRAL does not contain any parts that can be repaired by the user. If a fault occurs, please follow the instructions below.

If it develops a fault, the PARI CENTRAL must be disconnected from the central gas supply immediately.

Problem	What to do
<p>Air or oxygen escapes from the PARI CENTRAL after it is inserted in the central gas outlet but before the tubing is inserted (sound is clearly audible and/or gas flow can be felt).</p>	<p>Do not use the PARI CENTRAL under any circumstances!</p> <p>Send the PARI CENTRAL to the Technical Service Dept. of PARI GmbH!</p>
<p>No air or oxygen flows through the nebuliser, even though the tubing is properly connected.</p>	<ul style="list-style-type: none"> • Disconnect the tubing from the nebuliser. • Check whether air or oxygen flows out of the end of the tubing (hissing noise and/or flow can be felt). <p><i>If air or oxygen is escaping:</i></p> <p>The nebuliser is defective. Read the instructions for use of the PARI nebuliser to learn possible causes and remedies.</p> <p><i>If no air or oxygen is escaping:</i></p> <p>The PARI CENTRAL is defective. Send the PARI CENTRAL to the Technical Service Dept. of PARI GmbH.</p>

10 Terms and conditions of warranty

During the warranty period (one year), we or our customer service will repair any defects resulting from faults in material or workmanship free of charge. The owner shall not be entitled to cancel the sale or demand a partial or full refund of the purchase price. This warranty does not cover damage caused by improper use of the appliance. The warranty shall become void if repairs are carried out on the appliance by unauthorised persons. No compensation will be paid for consequential or direct damage.

In the event of a claim under this warranty, please bring the PARI CENTRAL to your dealer or send it to us, packed in the original carton and postage paid, together with the warranty certificate, stamped by the dealer.

Wearing parts are excluded from the warranty.

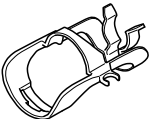
The warranty period shall commence on the date of purchase.

11 Explanation of symbols

- Caution, observe the instructions for use
- Warning

12 Spare parts and accessories

	Description	Part no.
	Connection tubing for compressed air system (2.1 m)	041G4550
	PARI CENTRAL Clip 5-pack	058G2001

We reserve the right to make technical changes.
Domestic and foreign patents.

For all product information please contact our Service Center:
Tel.: +49-(0)8151-279 220

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