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# PARI ProtECT®

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Please read attentively.

## Mouth and Throat Spray

Read this information carefully before you start using PARI ProtECT Mouth & Throat Spray. DO NOT THROW THIS LEAFLET AWAY! It contains important information. If you have any questions about PARI ProtECT Mouth & Throat Spray please ask your healthcare provider, pharmacist, or the distributor.

### What is PARI ProtECT Mouth & Throat Spray and what is it used for

PARI ProtECT Mouth & Throat Spray is a medical device for use if suffering from sore throat and hoarseness caused by dry and irritated mucous membranes in the mouth and throat. The Mouth & Throat Spray contains Ectoin®, a natural cell-protective molecule. Its membrane-stabilizing properties form a protective "Hydro Complex" shield in the mouth and throat.

PARI ProtECT Mouth & Throat Spray reduces typical symptoms of common cold such as sore throat, cough, hoarseness and pain on swallowing. The Mouth & Throat Spray is a hypertonic solution that hydrates dry and irritated epithelia and supports mucolysis and the regeneration process of dry and irritated membranes caused by cold air or air conditioning.

PARI ProtECT Mouth & Throat Spray is free of preservatives. The effect is based on 100% natural ingredients.

### Warnings and precautions

- If recommended by your doctor/health care professional, PARI ProtECT Mouth & Throat Spray may be used after surgery or trauma.
- Children younger than 2 years should not use PARI ProtECT Mouth & Throat Spray.
- PARI ProtECT Mouth & Throat Spray should not be used in the case of hypersensitivity to Ectoin® or any of the other ingredients of the mouth and throat spray.
- If allergic reactions occur, the use of PARI ProtECT Mouth & Throat Spray must be discontinued immediately.
- To ensure safe hygiene, each spray bottle should be used by one person only.
- Do not use for more than 6 weeks after opening.

### What you need to know before using PARI ProtECT Mouth & Throat Spray

There are no data available regarding interactions between PARI ProtECT Mouth & Throat Spray and other mouth and throat sprays or lozenges.

### What should be noted during pregnancy and while breast-feeding

Currently, there are no data available regarding the use of PARI ProtECT Mouth & Throat Spray during pregnancy and while breast-feeding. Consult your healthcare provider or pharmacist for advice, before using this Medical Device, if you are:

- pregnant or breast-feeding
- not sure whether you are pregnant or not

### How to use PARI ProtECT Mouth & Throat Spray

Always use PARI ProtECT Mouth & Throat Spray according to these instructions for use. Consult your healthcare provider or pharmacist before use, if you are not sure.

If not otherwise prescribed by your healthcare provider, please apply 1 - 2 sprays into the mouth and throat several times a day.

Before using PARI ProtECT Mouth & Throat Spray for the first time, shake the bottle and pump 2 -3 times to release the air from the pump. If the spray function is impaired (which might happen if the bottle is not positioned upright), press the pump mechanism several times in upright position.

1. Shake the bottle.
2. Hold the bottle upright in your hand and place your index finger on top of the spray head.
3. Open your mouth.
4. Hold the spray in front of your mouth and press the spray head down quickly and firmly to spray.
5. Please leave the spray head outside your mouth.
6. If you touched the inner of your mouth or your tongue with the device, wipe the exposed tip of the bottle with a clean lint-free cloth.

### Possible side effects of PARI ProtECT Mouth & Throat Spray

PARI ProtECT Mouth & Throat Spray is well tolerated. Up to now, no regularly occurring or persisting side effects are known. Please inform your healthcare provider, pharmacist or the distributor of the product if you notice side effects that are associated with the use of PARI ProtECT Mouth & Throat Spray. If conditions persist or worsen, contact your healthcare provider or pharmacist.

### How to store PARI ProtECT Mouth & Throat Spray

The expiry date of PARI ProtECT Mouth & Throat Spray is printed on the box and on the bottle. PARI ProtECT Mouth & Throat Spray must not be used after the expiry date has passed. PARI ProtECT Mouth & Throat Spray should be stored between 2 °C – 25 °C.

PARI ProtECT Mouth & Throat Spray should be kept out of reach of children.

### Content of PARI ProtECT Mouth & Throat Spray and other information

20 ml solution

One spray contains 0.15 ml solution

### Ingredients

Ectoin®, Sea Salt, Phosphate Buffer, Xylitol, Sodium Cyclamate, Caprylic acid, Mint oil, Salvia oil, Water

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Date of information: 2018-04

### Explanation of the symbols used on the product:

|   |  |  |  |
|---|--|--|--|
|  CE mark    |  Manufacturer         |  Please consider the instructions for use |  Caution, check the instructions for use (for important warnings/precautions) |
|  LOT number |  Use by (Expiry date) |  25°C<br>Temperature limitation           |  Order no.  |

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bitop AG
PARI GmbH