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PARI ProtECT® Nasenspray

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Please read attentively.

PARI ProtECT® Nasal Spray

Read this information carefully before you start using PARI ProtECT Nasal Spray. DO NOT THROW THIS LEAFLET AWAY! It contains important information. If you have any questions about PARI ProtECT Nasal Spray please ask your healthcare provider, pharmacist, or the distributor.

What is PARI ProtECT Nasal Spray and what is it used for

PARI ProtECT Nasal Spray is a medical device for use if suffering from a blocked nose (rhinitis and rhinosinusitis).

The Nasal Spray contains Ectoin®, a natural cell-protective molecule. Its membrane-stabilizing properties form a protective "Hydro Complex" shield on the mucous membranes.

It alleviates typical symptoms such as sneezing, itching and crust formation. PARI ProtECT Nasal Spray decongests the nose and sinuses on a natural way. It supports the regeneration process of irritated nasal epithelium. PARI ProtECT Nasal Spray is free of preservatives.

Warnings and precautions

- PARI ProtECT Nasal Spray should not be used after nasal operations or injuries of the nose.
- PARI ProtECT Nasal Spray should not be used in the case of hypersensitivity to Ectoin® or any of the other ingredients of the nasal spray.
- If allergic reactions occur, the use of PARI ProtECT Nasal Spray must be discontinued immediately.
- To ensure safe hygiene, each nasal spray bottle should only be used by one person.
- Do not use for more than 6 weeks after opening.

What you need to know before using PARI ProtECT Nasal Spray

There are no known interactions between PARI ProtECT Nasal Spray and other nasal sprays.

What should be noted during pregnancy and while breast-feeding

Currently, there are no data available regarding the use of PARI ProtECT Nasal Spray during pregnancy and while breast-feeding.

Consult your healthcare provider or pharmacist for advice, before using this Medical Device, if you are:

- pregnant or breast-feeding
- not sure whether you are pregnant or not

How to use PARI ProtECT Nasal Spray

Always use PARI ProtECT Nasal Spray according to these instructions for use. Consult your healthcare provider or pharmacist before use if you are not sure.

If not otherwise prescribed by your healthcare provider, please apply 1 - 2 sprays in each nostril several times a day. Children 10 years of age and younger should be assisted by an adult during the application of PARI ProtECT Nasal Spray.

Before using PARI ProtECT Nasal Spray for the first time, take off the cap and pump 2 – 3 times to release the air from the pump. Do not cut the tip. If the spray function is impaired (which might happen if the bottle is not positioned upright), press the pump mechanism several times in upright position.

1. Gently blow your nose to clean your nostrils.
2. Remove the protecting cap from the bottle.
3. Hold the bottle with your index and middle finger on either side of the finger rests and your thumb underneath the bottle. Bend your head slightly forward and close one nostril by gently pressing against the side of your nose with your finger.
4. Insert the tip of the spray into your nostril and start to breathe-in gently.
5. Press the pump down quickly and firmly to spray into your nostril. Continue breathing gently through the nostril.
6. Insert the tip of the spray into your other nostril.
7. Repeat steps described above for the other nostril.
8. Wipe the exposed tip of the bottle with a clean lint-free cloth.
9. Replace cap on the bottle.

Possible side effects of PARI ProtECT Nasal Spray

PARI ProtECT Nasal Spray is well tolerated. Up to now, no regularly occurring or persisting side effects are known.

Please inform your healthcare provider, pharmacist, or the distributor of the product if you notice side effects that are associated with the use of PARI ProtECT Nasal Spray.

If conditions persist or worsen, contact your healthcare provider or pharmacist.

How to store PARI ProtECT Nasal Spray

The expiry date of PARI ProtECT Nasal Spray is printed on the box and on the bottle. PARI ProtECT Nasal Spray must not be used after the expiry date has passed.

PARI ProtECT Nasal Spray should be stored between 2 °C – 25 °C. PARI ProtECT Nasal Spray should be kept out of reach of children.

Content of PARI ProtECT Nasal Spray and other information

20 ml solution
One spray contains 0.14 ml solution

Ingredients

Ectoin®, Sea Salt, Water, and Phosphate Buffer

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Date of information: 2018-04

Explanation of the symbols used on the product:

 CE mark	 Manufacturer	 Please consider the instructions for use	 Caution, check the instructions for use (for important warnings/precautions)
 LOT number	 Use by (Expiry date)	 ^{25°C} Temperature limitation	 Order no.

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bitop AG
PARI GmbH