
## PARI MONTESOL

Nasendusche

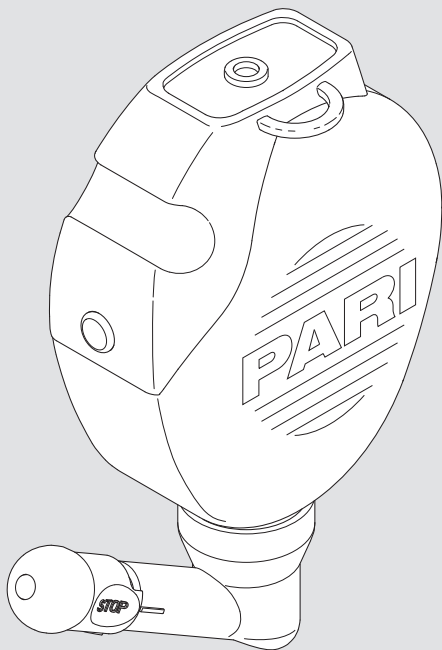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# en Instructions for use

Information as of: January 2013. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

<b>IMPORTANT INFORMATION</b> .....	<b>18</b>
General.....	18
Information about the instructions for use .....	18
Structure of safety instructions .....	18
Contraindications and side effects .....	19
Instructions for application.....	19
Hygiene .....	19
<b>PRODUCT DESCRIPTION</b> .....	<b>20</b>
Components .....	20
Intended purpose .....	21
Material information .....	21
Operating life .....	21
<b>APPLICATION</b> .....	<b>22</b>
Prepare nasal douche .....	22
Fill the nasal douche .....	22
Perform the nasal rinse .....	24
Tips.....	25
<b>CLEANING AND DISINFECTION</b> .....	<b>26</b>
Preparation .....	26
Cleaning .....	26
Disinfection .....	27
Visual inspection .....	28
Drying and storage .....	28
<b>MISCELLANEOUS</b> .....	<b>29</b>
Disposal.....	29
Contact .....	29
Explanation of symbols .....	29

# 1 IMPORTANT INFORMATION

## 1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

**If symptoms persist or if your condition worsens, discontinue the application and contact your doctor immediately.**

## 1.2 Information about the instructions for use

If these instructions for use are lost, you can request another copy from PARI GmbH [see: Contact, page 29]. The instructions for use for some products can be retrieved on the internet in both German and English. Simply visit: [www.pari.de/de-en/products](http://www.pari.de/de-en/products) (on the respective product page).

## 1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

## 1.4 Contraindications and side effects

The PARI MONTESOL nasal douche must not be used:

- if the mucous membranes of the nose are severely irritated.
- if you have open wounds in your nose. In this case, flushing would cause heavier bleeding.
- if you are unable to close your soft palate, e.g., because of palatine paralysis.

## 1.5 Instructions for application

Only use rinse solutions that are approved for use as nasal rinses. Before using the PARI MONTESOL nasal douche, read the instructions for use of the rinse solution you intend to use.

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Therefore, make sure that you always keep this product out of the reach of babies and infants.

## 1.6 Hygiene

Observe the following hygiene instructions:


- Always use a clean, disinfected and dried nasal douche for nasal rinses. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection.
- Each nasal douche must only be used by a single patient for hygiene reasons.
- Before every use and cleaning, wash your hands thoroughly.
- Make absolutely sure you also carry out cleaning and drying before using the device for the first time.
- For cleaning and disinfection, always use fresh tap water of drinking water quality.

- Make sure all components are dried properly whenever you have performed cleaning or disinfection.
- Do not keep the nasal douche in a damp environment or together with damp objects.

## 2 PRODUCT DESCRIPTION

### 2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.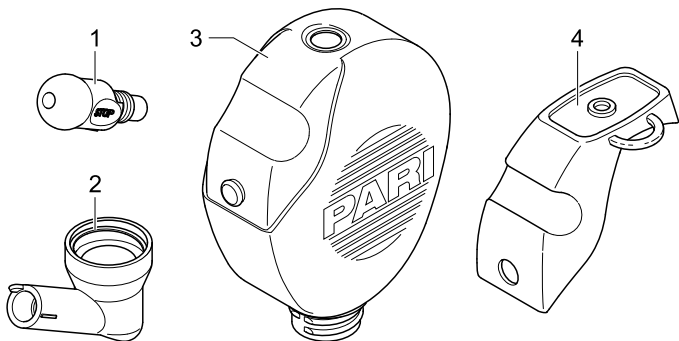

- (1) Nasal insert
- (2) Bend
- (3) Reservoir (250 ml fill volume)
- (4) Valve

## 2.2 Intended purpose

The PARI MONTESOL nasal douche is a medical aid designed for flushing the nasal sinuses. Using the nasal douche moistens the mucous membranes of the nose and cleans the nasal sinuses.

The nasal douche is suitable for use by children aged 3 and older as well as by adults.

Only solutions that have been approved for use as nasal rinses may be used. Follow the instructions for use of the respective nasal rinse.

The nasal douche is used to clean and moisten the nasal sinuses in cases of:

- frequently occurring nasal congestion
- predisposition to cold infections
- dry nose and dried mucus
- very dusty or contaminated atmospheres
- pollen allergy
- chronic inflammations of the nose and nasal sinuses
- following surgical procedures in the area of the nose and nasal cavities (only if your doctor agrees)

## 2.3 Material information

Polypropylene	Bend, reservoir
Thermoplastic elastomer	Nasal insert, valve


## 2.4 Operating life

Replace the nasal douche after eighteen months at the latest as changes can occur in the material of the product over time.


## 3 APPLICATION

### 3.1 Prepare nasal douche

- Attach the valve to the reservoir.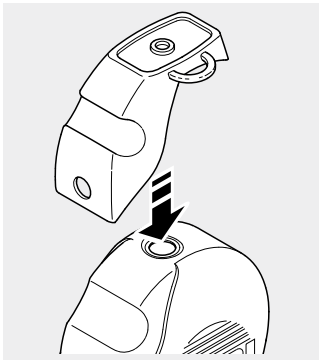


- Place the nasal insert on the bend. Make sure that the small pin (a) on the bend engages in the hole (b) in the nasal insert.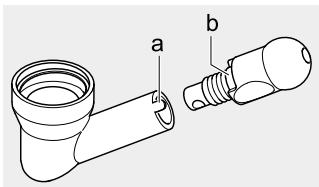

### 3.2 Fill the nasal douche

In the following, filling the nasal douche for use in conjunction with PARI MONTESOL nasal rinse is described. If you are using a different nasal rinse, fill the nasal douche as instructed in the respective instructions for use.

- Place the nasal douche on the valve.
- Fill the nasal douche with the nasal rinse.  
Instructions regarding proper dosage are included in the instructions for use of the nasal rinse you are using.
- Fill the nasal douche with water at body temperature up to the bottom edge of the screw thread.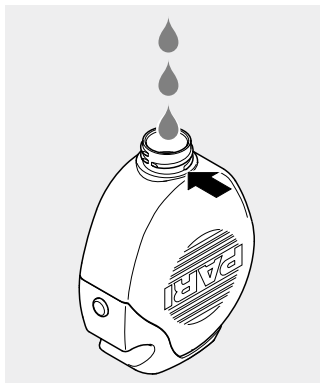

**⚠ CAUTION**


In order to avoid **danger to health from bacteria-infected water**, the water used must be of drinking water quality and must always be fresh.

In the case of particular diseases, such as cystic fibrosis or immediately following surgical procedures, it may be necessary to use boiled or sterile water.

- Screw the bend onto the reservoir tightly.


- Twist the nasal insert to the STOP position, to ensure that no liquid can escape yet (the line on the bend points to "STOP").
- Shake the nasal douche back and forth so that the nasal rinse is mixed with the drinking water.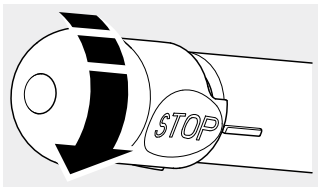


### 3.3 Perform the nasal rinse

- To rinse the left nostril, hold the nasal douche in your left hand, hold it in your right hand for the right nostril. Rest your index finger on the valve.
- With your free hand, twist the nasal insert to the rinse position (the line on the bend will point to the triangle).

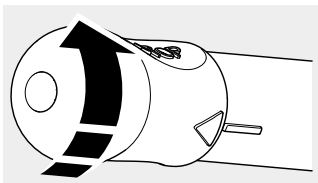- Bend your head slightly over the basin and open your mouth wide so that your soft palate closes.
- Continue breathing deeply and calmly through your nose.
- Place the nasal insert against your nostril so as to block it completely.
- Tilt your head slightly to one side (in the direction of the nostril that is not currently being treated).

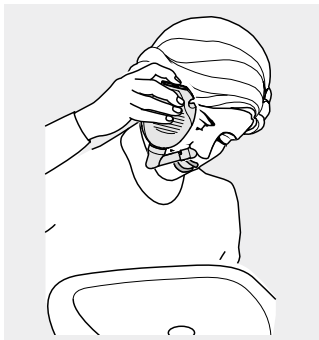- Press your index finger gently against the nasal douche valve.  
The nasal rinse will begin. The rinse solution flows in one nostril, through the nasal sinuses and back out of the other nostril.
  - If necessary, you can interrupt the treatment simply by taking your index finger off of the valve.
- i** *At the end of rinsing, there will still be a small quantity of the solution left in the bottom of the nasal douche.*

### 3.4 Tips

#### **What should I do if I get rinsing fluid in my eyes and it causes irritation?**

Rinse your eyes with abundant fresh water.

#### **What should I do if the rinse solution runs into my throat?**

It is almost impossible to prevent small quantities of nasal rinse solution from running down the throat. If you tilt your head

slightly forwards and to one side while rinsing, this small quantity will flow quite naturally out of your mouth.

If larger quantities pass into the throat, either your mouth is not open wide enough (so your soft palate is not closed), or the nasal passage on the other side is so congested that the rinse solution cannot flow back. In the first case, remember to open your mouth as wide as possible, and if necessary shift the position of your head slightly. In the second case, stop rinsing and begin by rinsing the other nostril.

## **4 CLEANING AND DISINFECTION**

The PARI MONTESOL nasal douche must be cleaned after each use and disinfected at least once a day.

### **4.1 Preparation**

Dismantle the nasal douche into its individual parts:

- Unscrew the bend from the reservoir.
- Pull the nasal insert off from the bend.
- Remove the valve from the reservoir.

### **4.2 Cleaning**

- Thoroughly clean all components in warm tap water and dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose dirt (the brush must be reserved exclusively for this purpose).
- Rinse all parts thoroughly under warm running water.
- You can remove excess water more quickly by shaking all parts.

## 4.3 Disinfection

Disinfect the components immediately after cleaning (only parts that have been cleaned can be disinfected effectively).

### CAUTION

A damp environment may encourage the growth of bacteria. Therefore, remove all parts from the pot or disinfector as soon as disinfection has finished. Dry the parts. The **risk of infection** is reduced when the parts are dried completely.

## In boiling water

- Place all the individual parts in boiling water for at least 5 minutes.

Use a clean pot and fresh drinking water.

**I** *If your water is very hard, scale may be deposited on the plastic parts as a milky coating. This can be prevented by using softened water.*

### NOTICE

Plastic will melt if it comes into contact with the hot base of the pot. Therefore, make sure there is plenty of water in the pot. This way you will avoid **damaging the components**.

## Using a standard thermal disinfector for baby bottles (not a microwave oven)

For effective disinfection, use a disinfector with a runtime of at least 6 minutes. Regarding disinfection, the duration of the disinfection procedure and the quantity of water required for this, follow the instructions for use of the disinfector you are using.

## CAUTION


Inadequate disinfection encourages the growth of bacteria and thus increases the **risk of infection**. Thorough disinfection has not been completed until the disinfectant automatically switches itself off, or the minimum disinfection time specified in the instructions for use of the disinfectant has elapsed. Therefore, do not switch the device off prematurely. Also make sure that the disinfectant is kept clean and regularly check that it is in good working order.

### 4.4 Visual inspection

Inspect all product components after each cleaning and disinfection. If any components have been damaged, the whole product must be replaced.

### 4.5 Drying and storage

- You can remove excess water more quickly by shaking all parts.
- Attach the valve to the reservoir.
- Insert the knee in the hole in the valve.
- Insert the nasal insert in the aperture in the knee.
- Hang the nasal douche up by the strap on the valve to dry.

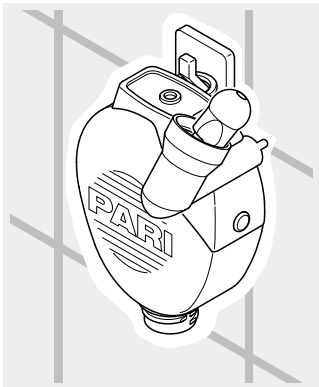## 5 MISCELLANEOUS

### 5.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

### 5.2 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)  
+49 (0)8151-279 220 (international)

### 5.3 Explanation of symbols

The following notices are displayed on the packaging for your PARI MONTESOL nasal douche:

	Manufacturer
	Please follow the instructions for use
	The product satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.
	Order no.
	Lot no.

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