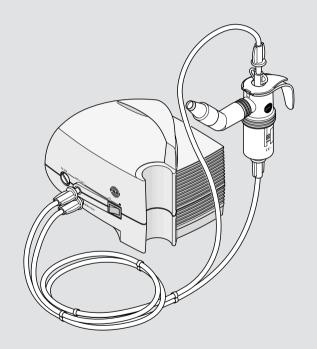
PARI SINUS

Type 028

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en Instructions for use

Information as of: April 2017. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

IMPORTANT INFORMATION	47
General	47
Information about the instructions for use	47 47
Using the compressor	48 49
Treatment of children and anyone who requires assistance	49
PRODUCT DESCRIPTION	50
Components	50
Intended purpose	52
Intended use	52
Contraindications	53
Product combinations	53
Working parts	54
Mode of action in treatment of the upper airways	55
Material information	55
Operating life	56
TREATMENT	57
Preparing the treatment	57
Performing treatment	63
Ending the treatment	67
MAINTENANCE AND CARE	67
Preparation	67
Drying the connection tubes	68
Maintaining the compressor	68
Care of the nebuliser and accessories	71

TROUBLESHOOTING	77
TECHNICAL DATA	78
General compressor data	78 79 79 80 80
MISCELLANEOUS	81
Disposal Terms and conditions of warranty	81 82
Explanation of symbols	83 84

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

If symptoms persist or if your condition worsens, discontinue the application and contact your doctor immediately.

1.2 Information about the instructions for use

These instructions for use are intended for the user at home.

If these instructions for use are lost, you can request another copy from PARI GmbH [see: Contact, page 84]. The instructions for use for some products can be retrieved on the internet in both German and English. Simply visit: www.pari.com (on the respective product page).

1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word WARNING is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word CAUTION is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word NOTICE is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.4 Using the compressor

The compressor is an electrical device that is powered by supply voltage. It has been designed so that no live parts are accessible. However, if ambient conditions are unfavourable or if the compressor or power cord is damaged, this protection may no longer be assured. Therefore, please follow the instructions below to avoid damage to the device and the associated danger of contact with live parts (e.g., electric shock):

- The compressor is not splashproof. Protect the device and the power cord from liquid and moisture. Never handle the device, the power cord or the power plug with damp hands. In addition, the device must not be operated in a damp environment
- Only use the PARI power cord to operate the compressor.
- Make sure that the local supply voltage complies with the voltage data marked on the compressor identification label.
- Never leave the compressor unattended while it is in use.
- For safety reasons, always disconnect the power plug from the socket under the following circumstances:
 - if a malfunction occurs during operation
 - before cleaning and maintaining the device
 - immediately after use
- Always pull the power plug, not the cable, to disconnect the power cord from the socket.
- Make sure that the power cord is never kinked, pinched or jammed. Do not pull the power cord over sharp edges.
- Keep the compressor and the power cord away from hot surfaces (e.g., stove top, electric fire, open fire). Direct heat may damage the compressor housing or the power cord insulation.
- Keep the power cord away from domestic animals (e.g., rodents). They may damage the power cord insulation.

 The compressor must not be operated and/or the power plug must be unplugged from the socket immediately if the compressor or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock.

1.5 Using the nebuliser

Observe the following hygiene instructions:

- Do not use product components unless they have been thoroughly cleaned and dried. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection.
- Before every use and cleaning, wash your hands thoroughly.
- Make absolutely sure you also carry out cleaning and drying before using the device for the first time.
- For cleaning and disinfection, always use fresh water with reduced calcium content.
- Make sure all components are dried properly after cleaning or disinfection.
- Do not keep the product and accessories in a damp environment or together with damp objects.

1.6 Treatment of children and anyone who requires assistance

Children and anyone who requires assistance must be supervised constantly by an adult while carrying out the treatment. This is the only way to ensure safe and effective treatment. Individuals in this group often underestimate the hazards involved (e.g. strangulation with the power cord or the connection tubing), thus resulting in a risk of injury.

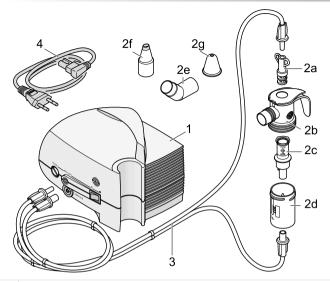
The product contains small parts. Small parts can block the airways and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

The compressor and the tubes, where they are attached to the compressor, can become very warm during normal operation. Therefore, particularly with individuals who require special assistance, ensure that the user's skin does not come into contact with the compressor. Prolonged contact may cause burns, particularly for patients with impaired temperature perception.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.

- (1) PARI SINUS compressor incl. air filter
- (2) PARI LC SPRINT SINUS nebuliser, consisting of:
 - (2a) Adapter (for vibration)
 - (2b) Nebuliser upper section
 - (2c) Nozzle attachment
 - (2d) Nebuliser lower section
 - (2e) Bend
 - (2f) Nasal joining piece
 - (2g) Nasal plug
- (3) PARI SINUS tube system, consisting of two identical tubes for compressed air and vibration
- (4) Power cord

2.2 Intended purpose

The PARI SINUS is a medical device intended for use in treating the upper airways with nebuliser therapy. It consists of a PARI SINUS compressor and a LC SPRINT SINUS nebuliser.

2.3 Intended use

The product is intended exclusively for home use.

The compressor is intended exclusively for indoor use and can be operated by the patient himself/herself.

For reasons of hygiene, the nebuliser, the nasal plug and the nasal joining piece must only be used by a single patient.

This product is only designed for patients who are able to breathe by themselves and are conscious.

The product is designed to treat children aged 6 and older and adults.

Saline solutions can be nebulised with the PARI SINUS to mobilise secretion. In this case, please follow the information in the instructions for use of the saline solution concerned.

If infection or inflammation of the paranasal sinuses is suspected, an appointment must be made with a doctor who can prescribe the medications suitable for use in nebuliser therapy with the PARI SINUS.

It is imperative that patients with cystic fibrosis discuss their treatment beforehand with a doctor.

An application usually takes 5 to 10 minutes.

2.4 Contraindications

Patients who respond to the treatment with adverse reactions must stop the therapy.

Following surgery or where inflammation of the middle ear is present, a risk/benefit assessment for vibrating aerosols must be carried out by the doctor before the start of treatment.

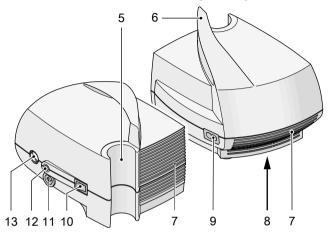
2.5 Product combinations

The PARI SINUS compressor can be used together with the following nebulisers:

- PARI LC SPRINT SINUS or PARI LC SPRINT XLent, for treatment of the upper airways with medication aerosols
- PARI LC Plus or PARI LC SPRINT, for treatment of the lower airways with medication aerosols

2.6 Working parts

The compressor includes the following working parts:

(5)	Holder for nebuliser
(6)	Carrying handle

- (7) Ventilation slits
- (8) Type identification plate (bottom of device)
- (9) Power cord socket
- (10) On/off switch
- (11) Vibration connection
- (12) Air connection
- (13) Filter holder

2.7 Mode of action in treatment of the upper airways

Nebuliser treatment of acute and chronic inflammations of the paranasal sinuses (sinusitis) using inhaled medications (saline solutions and/or drugs) is subject to particular requirements. The aim is to apply the active agent through the ostia (connections between the main nasal cavity and paranasal sinuses) directly to its site of action, the paranasal sinuses. To achieve this aim, the medical device creates a pulsating mist ("aerosol"1). The pulsation helps the active agent to pass through the ostia and into the paranasal sinuses much more effectively. The advantage of such topical (local) therapy over oral, systemic application (e.g., in tablet form) is that doses can be smaller. Thus it can be expected that there is a reduction in the occurrence of side effects. It also moistens the mucous membranes of the nose.

2.8 Material information

Polypropylene	Nebuliser, bend, nasal joining piece
Silicone	Nasal plug
Thermoplastic elastomer	Nebuliser, adapter

Aerosol: Small particles of solid, liquid or mixed composition suspended in gases or air.

2.9 Operating life

Component	Expected life cycle
Compressor	5 years, equivalent to approx. 1,000 operating hours
Air filter	200 operating hours, max. 1 year
Nebuliser and PARI SINUS tube system	max. 1 year

The nebuliser is designed for 365 applications and 52 disinfections.

If the treatment of a disease necessitates more intensive use and/or more frequent disinfections, the life cycle of the device is shortened.

The individual components must be replaced as soon as the expected life cycles indicated above have elapsed, if not sooner. A Year Pack for replacing the nebuliser, and also containing a new air filter, is available for purchase.

3 TREATMENT

3.1 Preparing the treatment

Setting up the compressor

Risk of fire due to a short circuit

A short circuit in the device can cause a fire. In order to reduce the risk of fire in such an event, follow the instructions below:

- Do not operate the compressor close to readily flammable objects such as curtains, tablecloths, or paper.
- Do not operate the compressor in areas where there is a risk of explosion or in the presence of oxidising gases (e.g., oxygen, nitrous oxide, flammable anaesthetics).

Danger of death by electrocution from a faulty device!

If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example.

- Check the compressor housing, the power cord and the power plug for damage every time before use.
- · Only use the compressor if all components are intact.

⚠ WARNING

Danger of electrocution due to damp hands

Liquids can conduct electricity, thereby posing a risk of electric shock.

 Never take hold of the power cord or the power plug with damp or wet hands.

Quality of treatment impaired by electromagnetic interference

Electrical devices can cause electromagnetic interference. Interference can impair the function of the devices and thus also the effectiveness of the treatment.

- Do not place the PARI device immediately beside or on top of other devices.
- Keep at a minimum distance of 30 cm from portable wireless communication devices (including accessories therefor, such as antenna cables or external antennas).
- Use only original spare parts and original accessories from PARI.

Risk of injury from tripping or caused by the device falling

A compressor placed in a poor position or for which the cables are routed untidily poses a risk of injury.

- Route the power cord in such a way that it is not a tripping hazard and no one can become entangled in it.
- Do not place the compressor above head height and make sure that it cannot be pulled down by the power cord or the connection tube.

NOTICE

Danger of device fault caused by falling

If the compressor falls, it may be damaged so badly inside and outside that it is no longer usable.

 Place the compressor on a firm, flat, dry surface. A soft base such as a sofa, a bed or tablecloth is not suitable.

NOTICE

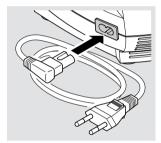
Danger of device fault caused by dust

If the device is operated in a very dusty atmosphere, more dust may collect inside the housing and cause a fault in the device .

- Do not operate the compressor on the floor, under the bed or in workshops.
- · Only operate the compressor in a low-dust environment.

The room in which the compressor is operated must satisfy certain conditions [see: Operation, page 80]. The compressor is not intended for outdoor use.

- Connect the power cord to the power cord socket of the compressor.
- Plug the power plug into a suitable socket

Assembling the nebuliser

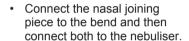
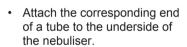
Risk of impaired treatment

Damaged components and/or an incorrectly assembled nebuliser may impair functioning of the nebuliser and thus treatment as well.

- Check all nebuliser components and the accessories before each use.
- Replace any broken, misshapen or seriously discoloured parts.
- Follow the assembly instructions in these instructions for use.

- Gently push the nozzle attachment as far as it will go onto the nozzle in the nebuliser lower section. The arrow on the nozzle attachment must point upwards.
- Place the nebuliser upper section on the nebuliser lower section and turn it clockwise to lock the two parts together.

Adding medication

NOTICE

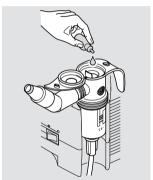
Nebuliser cap might break off

If the cap is twisted in the wrong direction, it may break off. The nebuliser will then be unusable and irreparable.

- Never move the cap except in the direction allowed by the hinge.
- Release the closure on the nebuliser upper section by pressing your thumb against the underside of the cap.
- Insert the nebuliser in the holder on the compressor intended for this purpose.
- top of the nebuliser chamber. To find out the quantity to be used, please refer to the information in the instructions for use of the medication.

 Info: Ensure that the medication is filled no higher than the upper scale marking (max. level 8 ml). Overfilling can greatly impair the nebulisation of medication.

Pour the medication into the

Close the cap of the nebuliser.
 Make sure that the cap snaps into place.

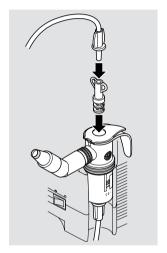
If you have to nebulise several medications one after the other in the same application, proceed as follows:

- Rinse the nebuliser out with tap water before filling it with the next medication.
- · Shake the nebuliser thoroughly to remove excess water.
- Fill the nebuliser with the next medication as described.

Connecting the tubes


The following section describes how to connect the PARI LC SPRINT SINUS nebuliser to the compressor. If you are using a different nebuliser, the adapter is not required, and only one of the two tubes is used for the compressed air supply. Then, the vibration connection on the compressor is left free.

- Insert the corresponding end of the as yet unconnected tube in the adapter.
- Insert the adapter in the nebuliser upper section.

 Insert the free ends of both tubes in the corresponding connections on the compressor.

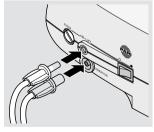Info: The tube ends can be inserted more easily if they are twisted slightly when inserting.

3.2 Performing treatment

Quality of treatment impaired by electromagnetic interference

If electrical devices are placed too close to each other, the function of both can be impaired due to electromagnetic interference

 If the PARI device has to be placed immediately beside or on top of other devices for operation, all devices must be monitored to ensure that they are working properly.

Danger of death by mixing up tubes

When several tubing systems are present, there is a possible danger of death if different connection options are confused with each other. This danger applies particularly in the case of patients in need of constant care who must receive artificial nutrition or other infusion treatments, for example.

 Check carefully to ensure that the other end of the PARI connection tube that is connected to the compressor is attached to the PARI nebuliser.

NOTICE

Danger of device fault due to overheating

Air must be able to circulate freely around the compressor to dissipate the heat generated in the housing.

- Never operate the compressor while it is in a bag.
- Never cover the compressor while it is operating.
- Make sure that the ventilation slits on the compressor are unobstructed at all times while it is operating.

General instructions for use

Before starting the treatment, ensure that all parts are firmly connected to each other.

Perform the treatment until the aerosol is used up (indicated by a change in the sound of the nebuliser).

 Several drops of medication will remain in the nebuliser after the treatment has ended.

To ensure that the treatment has the desired effect, please follow the instructions below:

- In order for the aerosol to reach all regions of the paranasal sinuses, the nose must be clear. If blocked, rinse it beforehand using for example the PARI MONTESOL nasal douche. If necessary, use a product designed to reduce swelling.
- The aerosol that is generated during nebulisation should only be delivered to the paranasal sinuses. To ensure this is the case, the soft palate² must be closed.
 - If the soft palate is not closed properly, some of the aerosol will end up in your throat. One sign of this is that the nostrils do not vibrate. In this case, it is possible that not enough saline solution or medication is reaching the paranasal sinuses.
- The aerosol is automatically delivered to the paranasal sinuses. Therefore, you should hold your breath during the treatment. In order to breathe, pause the treatment briefly. Switch the compressor off during breathing pauses.

The soft palate is the extension of the hard palate. It separates part of the oral cavity from the throat and is responsible for closing the airway off from the oesophagus.

To close the soft palate, follow the instructions below:

To close your soft palate, say the three letters "ing" as one word for as long as possible. After a short breathing pause, repeat this as often as necessary until the treatment is complete.

The nose is automatically blocked off from the mouth and throat by closing the "soft palate". The soft palate closes when swallowing or making sounds such as "ing".

Alternatively, you can breathe in and out slowly and deliberately through your mouth, as if attempting to blow a candle out (i.e. do not breathe through your nose). However, this method of closing the soft palate is not easy to perform for everybody, and some practice may be required.

You can tell the soft pallet is closed when the nostrils vibrate gently and air comes out through the hole in the nasal plug in the unobstructed nostril.

Procedure

Proceed as follows in order to carry out the treatment:

- · Sit in an upright position and relax.
- · Take the nebuliser out of its holder and hold it upright.
- Block one nostril with the aid of the nasal plug supplied with the equipment.

 It is necessary to block one nostril with the nasal plug because a certain amount of resistance is needed to ensure that the vibrating aerosol can penetrate the paranasal sinuses.

- The nasal plug has a small opening that creates a defined resistance when breathing.

 Make sure that the nostril is not completely blocked (e.g. by pressing closed with a finger) because this could create too much resistance.
- Position the nasal joining piece of the nebuliser against the free nostril so that the nostril is closed tightly.
- · Close your soft palate (as described).
- · Switch the compressor on.

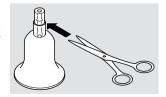
⚠ WARNING! Danger of death by electrocution from a faulty device! If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example. Switch the compressor off and disconnect the power plug from the mains socket immediately if a fault is suspected (e.g., following a fall or if there is a smell of burning plastic).

The aerosol is automatically delivered to the paranasal sinuses. Make sure that the nostrils vibrate gently for the entire period of treatment.

After about 2.5 minutes, carry out the treatment as described for a further 2.5 minutes on the other nostril. The treatment is complete after 5 minutes in all.

If you experience an unpleasant sensation of pressure in the ears - which can arise during the treatment - make the hole in the nasal plug larger (or do not press the free nostril closed so firmly):

 Push out the inside of the nasal plug and cut off the tip at the edge indicated using a pair of scissors.

- The larger the hole, the less the resistance. Take extra care not to make the hole too large, since this will mean that resistance is too low during breathing. The vibration must be felt at all times during the treatment session.
- · Push the inside of the nasal plug back in.

3.3 Ending the treatment

- · Switch the compressor off.
- Place the nebuliser back in its holder.
- · Disconnect the power plug from the mains socket.
- Complete disconnection from the mains is only certain
 when the power plug has been unplugged from the socket.

4 MAINTENANCE AND CARE

4.1 Preparation

- · Pull the two tubes and the adapter off the nebuliser.
- Detach the adapter from the tube.
- Open the cap of the nebuliser.
- Make sure that all medication residues are removed from the nebuliser.
- Dismantle the nebuliser into its individual parts.
- Rinse all components of the nebuliser under running tap water. This must be done especially thoroughly if cleaning and disinfection are not carried out straight away.

4.2 Drying the connection tubes

- The PARI SINUS tube system must not be cleaned or disinfected.
- Insert one of the two tubes into the air connection on the compressor.
- · Switch the compressor on.
- Allow air to blow through the tube until the "Air" has removed all moisture from the tube.
- · Switch the compressor off.
- Disconnect the tube from the compressor, and attach the other tube to the air connection.
- · Repeat the process as described for the first tube.

4.3 Maintaining the compressor

Do not attempt to carry out any maintenance on the device unless it is switched off and the power plug has been disconnected from the socket.

Cleaning

Danger of electrocution

Liquids can conduct electricity, thereby posing a risk of electric shock.

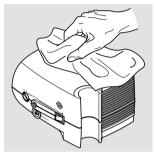
 Always switch the compressor off and disconnect the power plug from the mains socket before starting to clean it.

NOTICE

Danger of device fault due to liquid penetration

If liquids get into the interior of the device, this may cause a fault in the device.

- Never spray any liquids onto the compressor or the power cord.
- Wipe the outer surface of the housing with a clean, damp cloth.

If liquid does get into the compressor, the device must not be used under any circumstances. Before using the compressor again, you must contact the PARI GmbH Technical Service department or Service Center.

Replacing the filter

NOTICE

Danger of device fault due to incorrect air filter

If air filters that are not designed for the compressor are used, the compressor may be damaged.

 Only use air filters provided by PARI for use with your compressor.

In normal operating conditions, the compressor filter must be replaced after every 200 operating hours, but at least once a year.

Also please check the filter at regular intervals (after every 10 applications). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new one if it has become damp.

The filter cannot be cleaned and then reused!

To replace the filter, proceed as follows:

- Unscrew the filter holder from your compressor using a suitable coin.
- Pull the old filter out of the filter holder and fit the new filter in its place.

 Reinsert the filter holder together with the new filter in your compressor and screw it tight with the coin.

Repairing

The compressor must only be serviced by the PARI GmbH Technical Service department or a service centre expressly authorised by PARI. If the compressor is opened or manipulated by anyone else, all claims under the warranty shall be void. In these cases, PARI GmbH will also accept no liability.

Storage

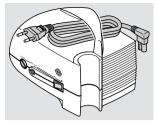
NOTICE

Damage to the power cord by kinking

If the power cord is wound or bent very tightly, the wires inside the cable may break. This will render the power cord unusable.

Do not wind the power cord around the compressor.

- Disconnect the power cord from the compressor and wind it up.
- Clip the power cord below the carrying handle for storage.

Store the compressor and accessories safely out of continuous direct sunlight.

Always disconnect the compressor from the mains for storage because electrical devices that remain plugged in present a potential hazard source.

4.4 Care of the nebuliser and accessories

The nebuliser and the accessories used must be cleaned thoroughly after each application and disinfected at least once a week. For high risk groups (e.g., cystic fibrosis patients), the nebuliser and the accessories used must be disinfected at least once a day.

Cleaning

 Place all components (except the PARI SINUS tube system) in warm tap water and a little dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose residue (the brush must be reserved exclusively for this purpose).

- · Rinse all parts thoroughly in running water.
- You can remove excess water more quickly by shaking all parts.

Disinfecting

Risk of infection due to growth of bacteria

A damp environment may encourage the growth of bacteria.

- Remove all parts from the pot or disinfector as soon as disinfection has finished.
- · Dry all the parts.

After cleaning, disinfect the dismantled nebuliser and accessories used (except the PARI SINUS tube system). Effective disinfection is only possible if the nebuliser has been cleaned.

In boiling water

- Place all the individual parts in boiling water for at least 5 minutes. Use a clean pot and fresh water with reduced calcium content, if possible.
 - **NOTICE!** Risk of damaging plastic parts! Plastic will melt if it comes into contact with the hot base of the pot. Make sure the water level in the pot is high enough to enable the parts to float clear of the base.
- You can remove excess water more quickly by shaking all parts.

Using a standard thermal disinfector for baby bottles (not a microwave oven)

Risk of infection due to inadequate disinfection

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Make sure that the disinfector is clean and working properly before every use.
- Allow disinfection to continue until the disinfector automatically switches itself off, or the minimum disinfection time specified in the instructions for use of the disinfector has elapsed. Do not switch the device off prematurely.

For effective disinfection, use a disinfector with a runtime of at least 6 minutes. Regarding disinfection, the duration of the disinfection procedure and the quantity of water required for this, follow the instructions for use of the disinfector you are using.

With a microwave disinfector

↑ CAUTION

Risk of burns

In a thermal disinfector, all of the components become very hot. If they are removed before they have had time to cool down after disinfection, they may cause burns on contact with skin.

 Allow all parts to cool down for about 10 minutes after they have been disinfected before taking them out of the disinfector.

⚠ CAUTION

Risk of infection due to growth of bacteria

Inadequate or ineffective disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Always comply with the specified minimum disinfection time and the required microwave power.
- · The disinfector must be perfectly clean.
- Check regularly that the microwave and the disinfector are functioning perfectly.
- · Use only water for disinfection, never use chemicals.

NOTICE

Risk of damaging the components with microwave radiation

If the product components are exposed to microwave radiation directly and without protection, they will be damaged.

- When heating the components in the microwave, always use a suitable microwave disinfector.
- · Do not use the grill function.

Microwave disinfectors ensure effective disinfection of the product components in a standard commercial microwave oven.

 Place the product components in the disinfector with enough space between them to allow the steam to reach all surfaces reliably.

Maintain the following times when disinfecting:

Wattage	Disinfection time	Cooling time
850 – 950	4 min.	2 min.
600 - 850	6 min.	2 min.

Information about the amount of water required and about the disinfection process is included in the instructions for use of the devices in question (disinfector and microwave).

Chemical cleaning with disinfection

Cleaning and disinfection can be carried out in a single cycle using a chemical preparation process. For this, use the cleaning disinfecting agent Bomix® plus.

To ensure safety when handling chemicals, follow the instructions for use of the disinfecting agent, particularly the accompanying safety instructions.

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the mixing proportions indicated and the specified treatment time.
- Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.
- Prepare a 2% solution of Bomix® plus by mixing 10 ml of the concentrate with 500 ml tap water.
- Place all the individual parts in the prepared solution and leave them to soak for 5 min.

- If the application period is exceeded significantly, the
 plastic parts may take on the smell of the disinfectant.
- Rinse off all parts thoroughly in running water (residues of the disinfectant can cause allergic reactions or irritations of the mucous membrane).
- You can remove excess water more quickly by shaking all parts.
- Dispose of the used solution (the diluted solution can be disposed of down the drain).

Drying and storing

- Place all parts on a dry, clean and absorbent surface and let them dry completely.
- Wrap the nebuliser in a clean, lint-free cloth (e.g., a teatowel) and keep it in a dry, dust-free environment.

5 TROUBLESHOOTING

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center.

Fault	Possible cause	Procedure
The compressor does not start up.	The power cord is not plugged into the device properly.	Check that the power cord is seated correctly in the compressor power cord socket.
	The power plug is not plugged into the mains socket properly.	Check that the power plug is seated correctly in the mains socket.
No medication is coming out of the nebuliser.	The nebuliser nozzle is blocked.	Clean the nebuliser.
	The compressed air supply tube is not connected to the air connection on the compressor correctly.	Check that both ends of the tube are seated properly.
	The tube attached to the air connection on the compressor is leaking.	Replace the defective tube or the entire PARI SINUS tubing system.

6 TECHNICAL DATA

6.1 General compressor data

Compressor type	Double compressor: Piston for compressing air, diaphragm for bolus generation
Supply voltage [V] Mains frequency [Hz] Power consumption [A]	see back cover page
Housing dimensions (W × H × D)	19.2 × 14.5 × 15.0
Max. compressor flow (FreeFlow)	9.5 l/min.
Compressor flow ³	4.8 l/min
Sound pressure level ⁴	approx. 56 dBA
Operating pressure ⁵	150 kPa (1.5 bar)
Bolus pressure amplitude, absolute	20 kPa (0.2 bar)
Frequency of bolus pressure amplitude	44 Hz
Weight	1.8 kg
Flow	4.5 l/min.

³⁾ Towards PARI nebuliser nozzle (Ø 0.48 mm)

⁴⁾ Maximum A-evaluated sound pressure level of the compressor (in accordance with DIN EN 13544-1, section 26)

⁵⁾ With PARI LC SPRINT SINUS nebuliser

6.2 Nebuliser data

Operating gases	Air, oxygen
Minimum flow	3.0 l/min.
Minimum operating pressure	0.5 bar / 50 kPa
Maximum flow	6.0 l/min.
Maximum operating pressure	2.0 bar / 200 kPa
Minimum fill volume	2 ml
Maximum fill volume	8 ml

Aerosol data according to DIN EN 13544-1 is available on request from PARI GmbH.

6.3 Classification according to DIN EN 60601-1

Type of electric shock protection	Protection class II
Degree of protection from electric shock of the part used (nebuliser)	Type BF
Degree of protection against water ingress in accordance with EN 60529 (IP rating)	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air, with oxygen, or with nitrous oxide	No protection
Operating mode	Continuous operation

6.4 Information about electromagnetic compatibility

Electrical medical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Such equipment must only be installed and operated in accordance with EMC instructions.

Portable and mobile high-frequency communication devices can disrupt electrical medical equipment. Using accessories, converters and power cords other than those specified (with the exception of converters and power cords that the manufacturer of the medical electrical device sells as spare parts for internal components) can result in higher emission levels or lower the resistance to interference of the device.

The device must not be placed directly beside or on top of other devices for operation. If the medical electrical device must be placed beside or on top of other devices to operate it, it should be monitored constantly to ensure that it is operating properly in the arrangement used.

Technical data on electromagnetic compatibility (EMC information) is available in table format upon request from PARI GmbH or on the internet at the following linked page:

https://www.pari.com/fileadmin/Electromagnetic-compatibility-1.pdf

6.5 Ambient conditions

Operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 hPa

Use of the device in professional healthcare facilities is limited to the inpatient wards and the intensive care unit. Use of the

device in areas with elevated magnetic or electrical radiation (e.g., close to an MRI scanner) is not permitted.

The device is intended for operation in healthcare situations at home. It is not permitted to operate it in trains, motor vehicles or aeroplanes.

Transportation and storage

Minimum ambient temperature (without monitoring of relative humidity)	-25 °C
Maximum ambient temperature (with relative humidity of up to 93%, noncondensing)	+70 °C
Humidity	max. 93%
Atmospheric pressure	500 hPa - 1,060 hPa

7 MISCELLANEOUS

7.1 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE)⁶. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers). Materials recycling helps to reduce the consumption of raw materials and protect the environment.

⁶⁾ Directive 2012/19/EU of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of July 4, 2012 on waste electrical and electronic equipment.

7.2 Terms and conditions of warranty

PARI guarantees that your device, if used according to the instructions, will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate, beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided by PARI applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects are not limited by the warranty or any claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

What does the warranty cover?

If, exceptionally, a defect is discovered, PARI will at its discretion repair or replace the device, or refund the purchase price of the product. If it is replaced, the replacement device may either be the same model or a model that is at least comparably equipped. Replacement or repair of the device shall not serve as the basis for a new warranty. All replaced old devices or parts shall become the property of PARI. Further claims are excluded. This applies particularly for any claims for compensatory damages. This disclaimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful wrongdoing and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

The warranty shall be cancelled if

- the device has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, or the like
- the damage was caused by transporting the device incorrectly or a falling impact
- the device has been misused or not cared for correctly

- the serial number on the device has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the device by persons not authorised by PARI

Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the entire device to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

Explanation of symbols

The following symbols can be found on the product and/or the packaging:

Please follow the instructions for use.

Manufacturer

(€ 0123 This product satisfies the requirements of 93/42/EEC (Medical devices) and 2011/65/EU (RoHS).

REF

Order no.

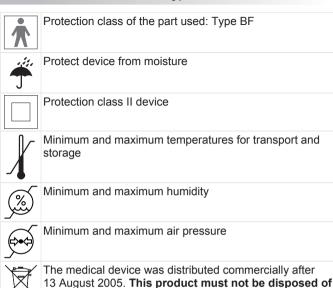
SN

Serial number of the device

Lot no.

●/O Off / On

Alternating current

7.4 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

with normal domestic waste. The symbol of the refuse bin with a cross through it indicates that it must be

Tel.: +49 (0)8151-279 279 (German-speaking) +49 (0)8151-279 220 (international)

disposed of separately.

PARI SINUS

Type 028

CERTIFICATE OF GUARANTEE

We guarantee this device for 3 years, commencing on the date of purchase.

Bar c	ode label	Application no:
Confirmation of purchase: The appliance with the above appliance number was sold in its original packaging.		
Purchase date Stamp and signature of the dealer		

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©2018 PARI GmbH Spezialisten für effektive Inhalation, 028D1020-B 06/18