## de MucoClear® 3% Inhalationslösung

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## en MucoClear® 3% inhalation solution

### Instructions for use

Please read these instructions for use carefully.

### IMPORTANT NOTES

**Intended purpose:** MucoClear 3% inhalation solution is used to mobilise secretions in the respiratory tract in the event of mucous consolidation. MucoClear 3% inhalation solution is suitable for all age groups, using a jet, membrane or ultrasonic nebuliser. No data is available regarding the inhalation of hypertonic saline solutions during pregnancy and when breastfeeding. If you are pregnant or breastfeeding, please consult your doctor or pharmacist before using this inhalation solution. The dose and duration of use is 4 ml up to four times daily as required. Individuals in the following groups must be supervised by a person who is responsible for their safety:

- Babies, infants, and children
- Individuals with limited capabilities (e.g. physical, mental, sensory).

**Indications:** Respiratory diseases (e.g. bronchitis, acute viral bronchiolitis, cystic fibrosis) with increased build up of mucus or mucous consolidation.

**Contraindications:** The manufacturer is not aware of any contraindications.

**Side effects:** Inhaling saline solutions can lead to increased coughing. Patients with a hypersensitive bronchial system may experience shortness of breath due to a temporary constriction of the bronchial tubes. In this case, stop the treatment and contact your doctor. Inform your doctor and the manufacturer if you notice any side effects that are not listed in the instructions for use.

**Interactions:** Always consult your doctor before mixing MucoClear 3% with other medications. When MucoClear 3% is mixed with other medications, there is always a risk of chemical cross-reactions which can impair the effectiveness of the treatment.

**Safety instructions:** The instructions for use contain important information, safety instructions and precautionary measures:

- Read all instructions for use carefully before you start using this PARI product.
- Use this PARI product only as described in the instructions for use.
- Keep the instructions for use in a safe place.
- Always follow the instructions of the inhalation system and nebuliser used.

**⚠️ WARNING** – may result in death or severe injury.

Choking hazard due to small parts that can be swallowed.

- Small parts can block the airways and lead to choking.
- Store all product components out of the reach of children.

### Danger of condition worsening

If your condition does not improve or your symptoms worsen, seek professional medical advice.

### Serious adverse incidents

Report serious adverse incidents to the manufacturer and the responsible competent authority.

**Ingredients:** Sodium chloride, water; sterile, free from preservatives.

**Mode of action:** MucoClear 3% inhalation solution is administered through nebulisation using an inhalation system. Due to an osmotic effect, water is drawn out of the inner layer of the lung tissue into the mucous layer that lines the lungs. This flow of water liquefies the mucus, supporting the clearance mechanism of the lungs. Mucus and other foreign particles are then cleared from the airway and can be coughed up more effectively.

**Shelf life:** Do not re-use opened ampoules as they may no longer be sterile. Dispose of the inhalation solution after the expiry date.

### USE

Use the inhalation solution only at room temperature. Use the ampoule immediately after opening it. Carry out the treatment by following all steps listed below exactly.

- Filling the nebuliser with the inhalation solution:
1. Wash your hands thoroughly.
  2. Check that the ampoule has not been damaged. Never use an ampoule that has been opened already or damaged, or if the solution is discoloured or cloudy.
  3. Open the ampoule by twisting off the upper part.
  4. Add the inhalation solution to the nebuliser as described in the nebuliser's instructions for use. Stop the treatment immediately and inform the manufacturer if you notice an unusual odour during inhalation.

### FURTHER INFORMATION

**Components:** As per packaging information.

**Disposal:** Comply with the applicable country-specific disposal regulations. Always dispose of the product in accordance with the Ordinance on the European Waste Catalogue (EWC).

### DOCUMENT INFORMATION

**Validity of instructions for use:** MucoClear 3% inhalation solution type 077

**Conformity:** This product conforms to Medical Device Regulation (EU) 2017/745.

### Contact:

Email: info@pari.de  
Tel.: +49 (0)8151-279 220 (international)  
+49 (0)8151-279 279 (German)

**Copyright:** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved.

**Trade marks:** Registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries: PARI®, MucoClear®

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**Copyright:** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Technische und optische Änderungen sowie Druckfehler vorbehalten.
**Copyright :** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Allemagne. Sous réserve de modifications techniques et visuelles, ainsi que d'erreurs d'impression.
**Copyright:** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Alemania. Salvo modificaciones técnicas y de forma y salvo errores de imprenta.
**Copyright:** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Duitsland. Technische en optische wijzigingen en drukfouten voorbehouden.
**Copyright:** ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Reservados os direitos a alterações técnicas e visuais, assim como a erros de impressão.