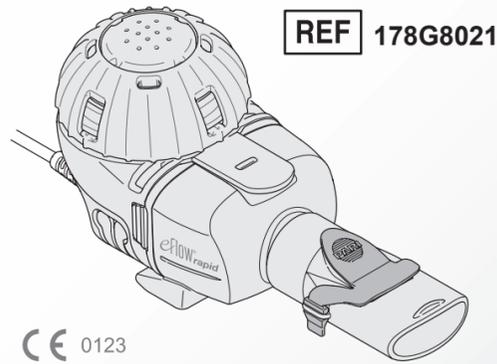
eFlow[®]rapid nebuliser handset for Colfinair[®]

REF 178G8021

CE 0123

en Instructions for Use

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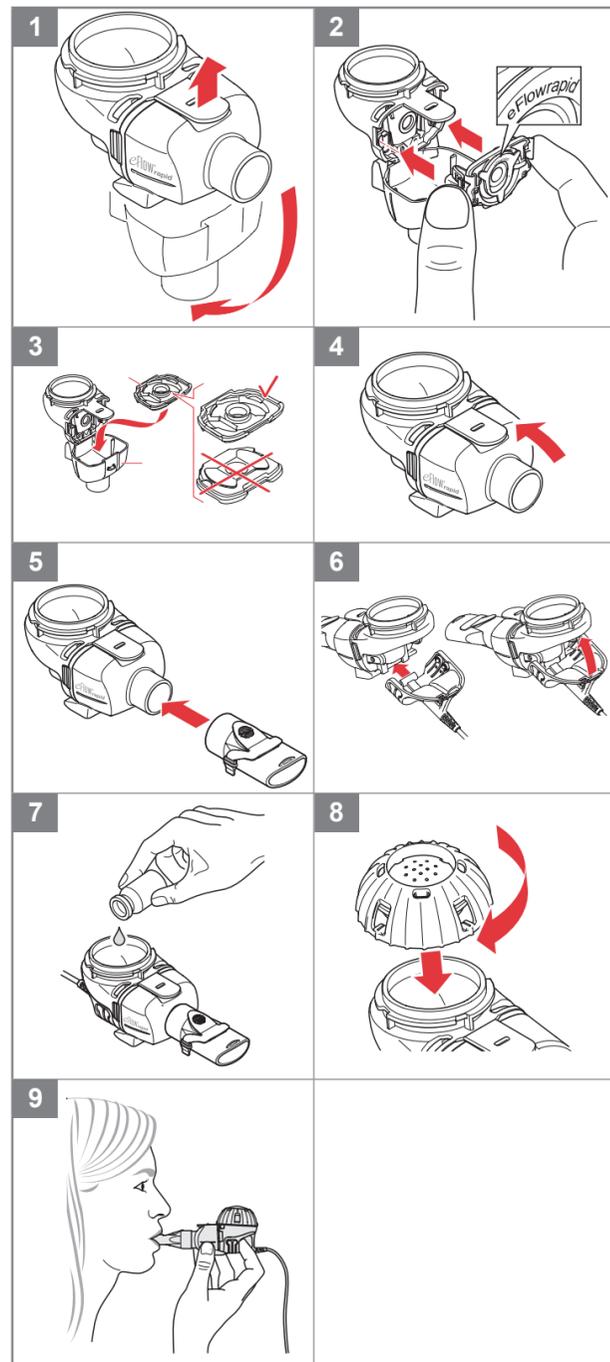
en Instructions for use of eFlow[®]rapid nebuliser handset

- i** – Read these instructions for use carefully and completely.
– Also take note of the instructions for use of the eFlow[®]rapid nebuliser system or the eBase[®] Controller before using for the first time. These contain further important instructions for the safe use of the eFlow[®]rapid nebuliser handset, particularly regarding connection, operation and hygiene.

1 Intended purpose

The eFlow[®]rapid nebuliser system is intended to administer nebuliser solutions or suspensions by inhalation approved for the treatment of diseases of the lower airways.

2 Use of the nebuliser handset

The eFlow[®]rapid nebuliser handset is intended for use with an eBase[®] Controller.

For hygienic reasons and to avoid cross-infections, the eFlow[®]rapid nebuliser handset must only be used by one patient.

The nebulisation time depends on the medication and the fill volume.

- Before using for the first time, disassemble and clean the nebuliser handset including the aerosol head as described in the instructions for use of the eFlow[®]rapid nebuliser system or the eBase[®] Controller.
- Assemble the nebuliser handset as shown in the diagram:

- 1 Open the nebuliser handset.
- 2 Insert the aerosol head. **Do not touch the membrane (metal part, in the middle) of the aerosol head.**
- 3 Insert the inspiratory valve.
- 4 Close the nebuliser handset.
- 5 Attach the mouthpiece.
- 6 Connect the connection cord.
- 7 Fill with the medication.
- 8 Close the medication reservoir.

This makes inhaling easier and improves uptake of the medication in the airways.

3 Cleaning and disinfection

The nebuliser handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow[®]rapid nebuliser system or the eBase[®] Controller).

4 Replacing the nebuliser handset

The nebuliser handset is a part of every pack of Colfinair[®] (1 month pack = 1 treatment cycle). We recommend using a new nebuliser handset for each treatment cycle.

5 Components

One eFlow[®]rapid nebuliser handset with one eFlow[®]rapid aerosol head.

6 Explanation of symbols

	Manufacturer
	Catalogue number
	Serial number
	CE marking: This product is compliant with the requirements set forth in Directive 93/42/EEC (Medical Devices).
	Caution
	Please follow the instructions for use
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of a crossed-through refuse bin indicates that separate collection is required.

7 Particle size distribution

Information on particle size distribution is available upon request from PARI Pharma GmbH.

For further information, contact your local service partner.

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