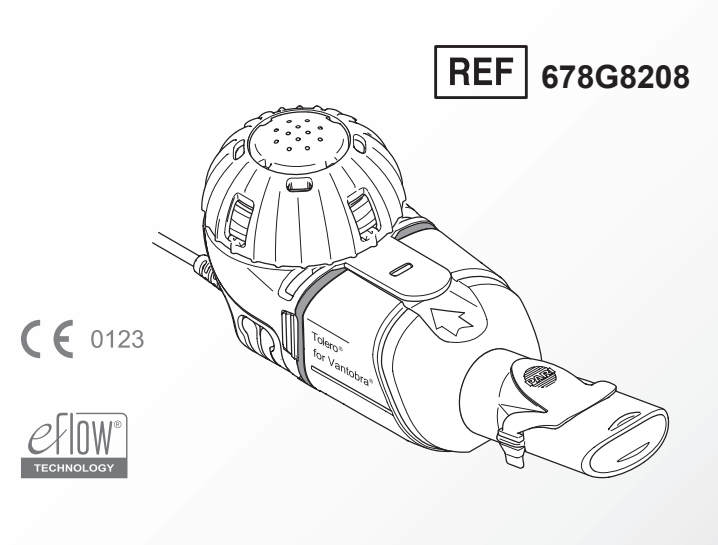
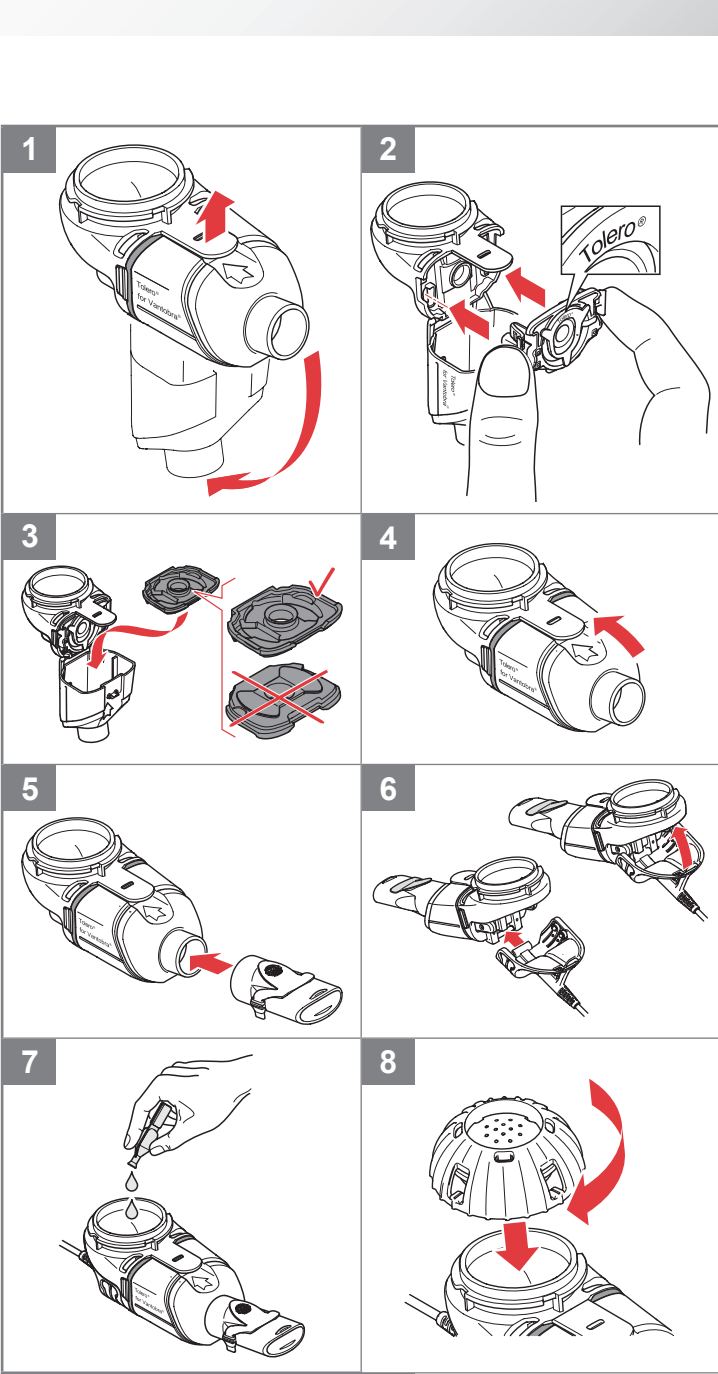
# Tolero® nebuliser handset

<b>de</b>	<b>Gebrauchsanweisung</b>	<b>fr</b>	<b>Notice d'utilisation</b>
<b>en</b>	<b>Instructions for Use</b>	<b>it</b>	<b>Istruzioni per l'uso</b>
<b>da</b>	<b>Brugsanvisning</b>	<b>nl</b>	<b>Gebruiksaanwijzing</b>
<b>el</b>	<b>Οδηγίες χρήσης</b>	<b>pt</b>	<b>Manual de instruções</b>
<b>es</b>	<b>Istrucciones de uso</b>		

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### Tolero® Vernebler

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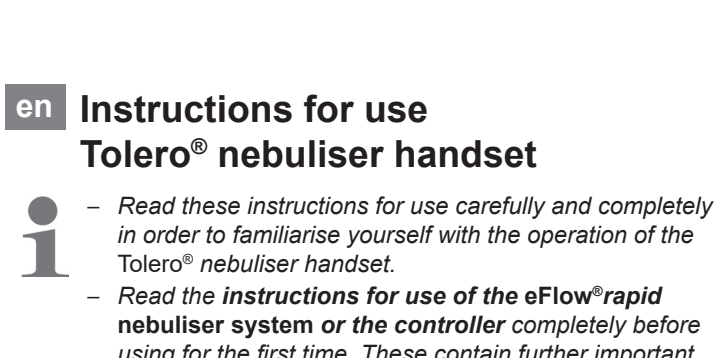
- *Read these instructions for use carefully and completely in order to familiarise yourself with the operation of the Tolero® nebuliser handset.*
- *Read the **instructions for use of the eFlow®rapid nebuliser system or the controller** completely before using for the first time. These contain further important instructions for the use of the Tolero® nebuliser handset, particularly regarding connection, operation, cleaning and disinfection.*

### 1 Intended purpose

The Tolero® nebuliser handset with an eBase® or eTrack® Controller is intended to administer Vantobra® as an aerosol for inhalation.

### 2 Indication

The Tolero® nebuliser handset is indicated to administer Vantobra® zur Behandlung von Lungeninfektionen durch *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

### 3 Contraindication

For the Tolero® nebuliser handset no contraindication is known to PARI Pharma GmbH.

Take note of contraindications of the medicine in the Vantobra® package leaflet.

### 4 Use of the nebuliser handset

The Tolero® nebuliser handset must only be used in accordance with its intended purpose.

For reasons of hygiene and to avoid cross-infections, the Tolero® nebuliser handset must only be used by one patient.

- Before using for the first time, disassemble and clean the nebuliser handset including the aerosol head as described in the instructions for use of the eFlow®*rapid* nebuliser system or the controller.
- Assemble the nebuliser handset as shown in the diagram:
  - Open the nebuliser handset.
  - Insert the aerosol head. **Do not touch the membrane (metal part, in the middle) of the aerosol head!**
  - Insert the inspiratory valve.
  - Close the nebuliser handset.
  - Attach the mouthpiece.
  - Connect the connection cord.
  - Fill with the medication.
  - Close the medication reservoir.
  - Sit in an upright position and relax. This makes inhaling easier and improves deposition of the medication in the airways.

- ⚠

Hold the nebuliser handset horizontally all the time while performing the inhalation. If the handset is tilted, errors in the dosage may occur.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The inhalation session should last between 4 and 5 minutes. Inhalation is ended as soon as no more aerosol is generated.

- Always check the medication reservoir to ensure that the entire dose of Vantobra® has been nebulised.

### 5 Cleaning and disinfection

The nebuliser handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®*rapid* nebuliser system or the controller).

### 6 Replacing the nebuliser handset

Each 28-day pack of Vantobra® contains a Tolero® nebuliser handset. Use the new nebuliser handset for each treatment cycle.

### 7 Package contents

One Tolero® nebuliser handset with one Tolero® aerosol head. There are no accessories for the Tolero® nebuliser handset.

### 8 Symbols

<b>MD</b>	Medical device
	Consult instructions for use
	Single Patient – Multiple use
<b>LOT</b>	Batch code
<b>REF</b>	Item number
<b>SN</b>	Serial number
	Minimum and maximum ambient temperatures
	Minimum and maximum atmospheric humidity
	Minimum and maximum atmospheric pressure

**CE** <sup>0123</sup> CE marking: This product is compliant with the requirements set forth in Directive 93/42/EEC (Medical Devices).

 The medical device was distributed commercially after 13 August 2005. **This product must not be disposed of with normal domestic waste.** The symbol of a crossed-through refuse bin indicates that separate collection is required.

	Caution
	Manufacturer
	Date of manufacture

### 9 Service and contact

In case of technical problems or questions about the device, please contact your service partner (contact addresses at: info@paripharma.com). A digital version of your Tolero® instructions for use and the technical information on Vantobra® with the Tolero® aerosol data can be found at **www.pari.com/vantobra**

We reserve the right to make technical changes. Information as of: 2020-11

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