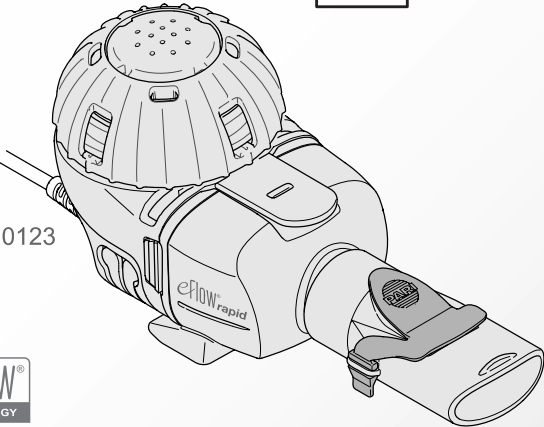
eFlow[®]

nebuliser handset

for Colfinair[®]

REF

178G8021

en

Instructions for Use

it

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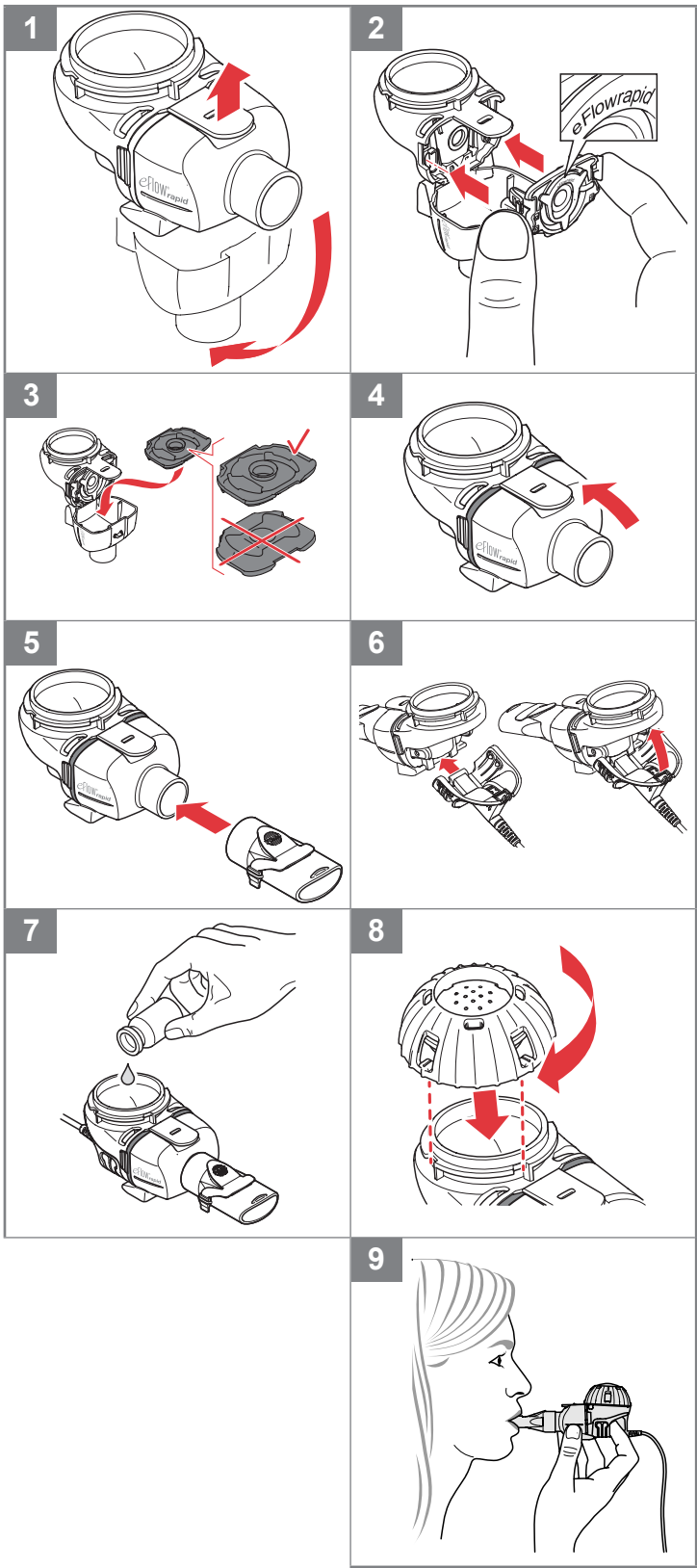
es

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el

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en Instructions for use of eFlow[®]rapid nebuliser handset

- i

- Read these instructions for use carefully and completely.
 - Read the **instructions for use of the eFlow[®]rapid nebuliser system or the controller** completely before using for the first time. These contain further important instructions for the use of the eFlow[®] nebuliser handset, particularly regarding connection, operation, cleaning and disinfection.

Report serious incidents to the manufacturer and the competent authorities.

1 Intended purpose

The eFlow[®]rapid nebuliser system is intended to administer solutions or suspensions for a nebuliser handset as an aerosol for inhalation.

2 Indication

The eFlow[®]rapid nebuliser system is intended to administer solutions or suspensions for a nebuliser handset which are approved for the treatment of diseases of the lower airways.

3 Contraindication

For the eFlow[®]rapid nebuliser system no contraindication is known to PARI Pharma GmbH. Take note of contraindications of the medicine in the accompanying package leaflet.

4 Use of the nebuliser handset

The eFlow[®]rapid nebuliser handset must only be used in accordance with its intended purpose. The eFlow[®]rapid nebuliser handset is intended for use with an eBase[®] or eTrack[®] Controller.

For reasons of hygiene and to avoid cross-infections, the eFlow[®]rapid nebuliser handset must only be used by one patient.

- Before using for the first time, disassemble and clean the nebuliser handset including the aerosol head as described in the instructions for use of the eFlow[®]rapid nebuliser system or the controller.
- Assemble the nebuliser handset as shown in the diagram:
 - 1 Open the nebuliser handset.
 - 2 Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**
 - 3 Insert the inspiratory valve.
 - 4 Close the nebuliser handset.
 - 5 Attach the mouthpiece.
 - 6 Connect the connection cord.
 - 7 Fill with the medication.
 - 8 Close the medication reservoir.
 - 9 Sit in an upright position and relax. This makes inhaling easier and improves uptake of the medication in the airways.

- 

Hold the nebuliser handset horizontally all the time while performing the inhalation. If the handset is tilted, errors in the dosage may occur.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The nebulisation time depends on the medication and the fill volume. Inhalation is ended as soon as no more aerosol is generated.

5 Cleaning and disinfection

The disassembled nebuliser handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow[®]rapid nebuliser system or the controller).

6 Replacing the nebuliser handset

The nebuliser handset is a part of every pack of Colfinair[®] (1 month pack = 1 treatment cycle). We recommend using a new nebuliser handset for each treatment cycle.

7 Package contents

One eFlow[®]rapid nebuliser handset with one eFlow[®]rapid aerosol head.

8 Symbols

	Medical device
	Consult instructions for use
	Single patient – Multiple use
	Batch code
	Item number
	Serial number
	Unique Device Identifier
	Minimum and maximum ambient temperature
	Minimum and maximum atmospheric humidity
	Minimum and maximum atmospheric pressure
	CE marking: This product is compliant with the requirements set forth in Regulation (EU) 2017/745 on medical devices.
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of a crossed-through refuse bin indicates that separate collection is required.
	Caution
	Manufacturer
	Date of manufacture

9 Service and contact

In case of technical problems or questions about the device, please contact your service partner (contact addresses at: info@paripharma.com). A digital version of the instructions for use of your eFlow[®]rapid nebuliser handset and eFlow[®]rapid nebuliser system and aerosol data are available at **www.pari.com/eflowrapid**

We reserve the right to make technical changes. Information as of: 2022-03

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