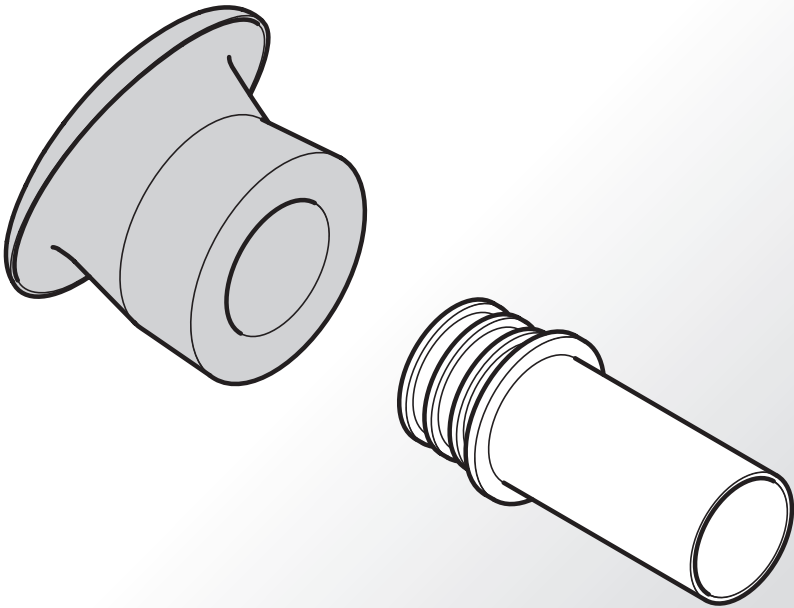
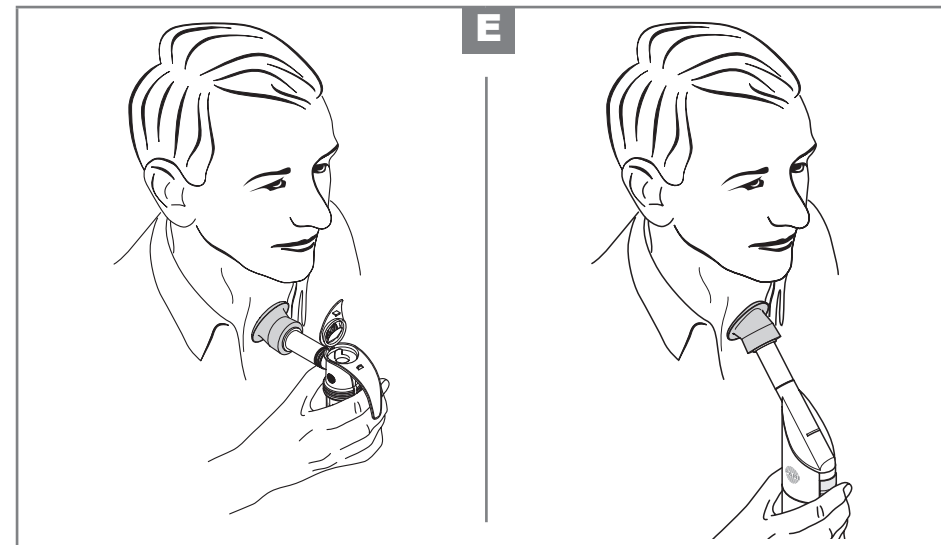
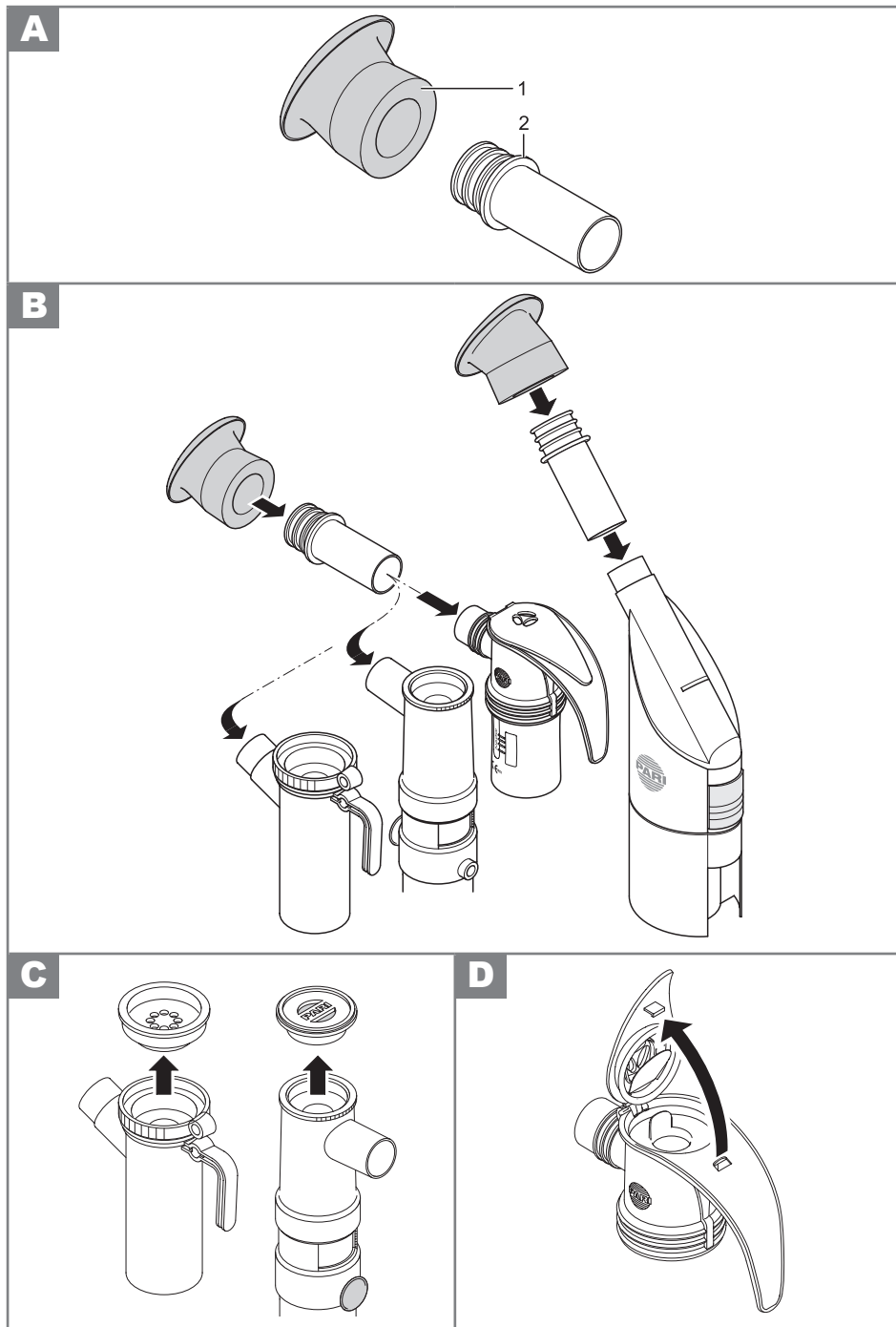
TRACHEO SET

CE 0123

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en Instructions for use

Important information

Please read these instructions for use through in their entirety. Do not discard so you can consult them at a later date.

Please also read and follow the instructions for use of the nebuliser you are using together with the PARI TRACHEO SET. If you fail to comply with the instructions for use, injury or damage to the product cannot be ruled out.

The signal word WARNING is used to indicate hazards which, without precautionary measures, might result in serious injury or even death.

Components

Figure A:

PARI TRACHEO SET		
Pos. no.	Description	Art. no.
1	Mask	025G0007
2	Extension tube	

Please check that all components shown are included in your package.

Materials used

Polypropylene	Extension tube
Silicone	Mask

Intended use

The PARI TRACHEO SET enables patients who have undergone a tracheostomy to receive inhalation therapy using PARI inhalation systems. It includes a comfortable mask made from soft, latex-free silicone that has been developed specifically for tracheostomised patients and an extension tube so that the nebuliser can be held in a comfortable position during inhalation.

The PARI TRACHEO SET has been designed for repeated use. It may be used for multiple patients.

WARNING

The PARI TRACHEO SET is not equipped with an expiratory valve. This can therefore present a choking hazard, so the PARI TRACHEO SET must only be used by patients who are conscious and able to breathe by themselves.

The PARI SINUS nebuliser and the PARI eFlow® nebuliser cannot be used with the PARI TRACHEO SET.

You must ask the manufacturer whether the PARI TRACHEO SET can be used with PARI nebulisers that are not described in these instructions for use.

Preparing the PARI nebuliser

In order for the patient to be able to exhale through the nebuliser, the nebuliser must be prepared as follows.

Figure B:

- Attach the PARI TRACHEO SET to the top of the PARI nebuliser. Ensure that the mask is fitted the right way round for the nebuliser you are using.

Figure C (PARI LC PLUS or PARI LL nebulisers only):

- Remove the inspiratory valve for the duration of the inhalation; otherwise it would prevent the patient from exhaling through the nebuliser.

Figure D (PARI LC SPRINT nebulisers only):

- Leave the nebuliser cap open for the entire inhalation period; otherwise it will prevent the patient from exhaling through the nebuliser.

Figure E:

- Use the PARI TRACHEO SET with the respective nebuliser as shown in the figure.

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PARI GmbH