SMARTMASK®

CE 0123

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Stand der Information: Januar 2016

1 IMPORTANT INFORMATION

Please read these instructions for use and the instructions for use of your PARI nebuliser carefully and completely. The instructions for use of your nebuliser contain all the safety-critical information you need, and which also applies to the PARI SMARTMASK.

Do not discard, so you can consult the instructions for use at a later date.

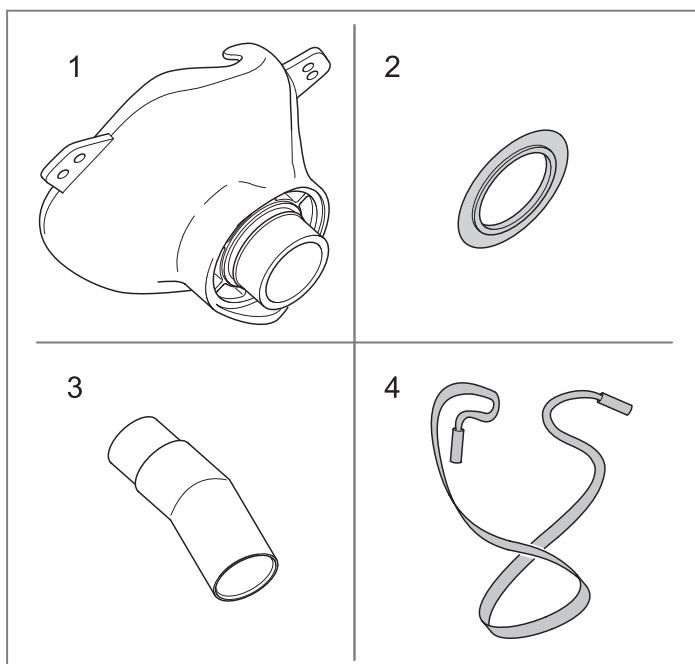
The product contains small parts. Small parts can block the respiratory tract and pose a **choking hazard**.

Therefore, make sure that you always keep the PARI SMARTMASK out of the reach of babies and infants.

2 PRODUCT DESCRIPTION

Item no.: 041G0730 (package without PARI nebuliser)

2.1 Components

- | | |
|-----|-------------------------|
| (1) | PARI SMARTMASK |
| (2) | Expiratory valve (blue) |
| (3) | 45° elbow piece |
| (4) | Rubber band |

Check that all components are contained in your package. Should anything be missing, please notify the dealer from whom you obtained the PARI SMARTMASK at once.

2.2 Intended purpose

The PARI SMARTMASK is intended for inhalation treatment of the lower respiratory tract if it is not possible to use a mouthpiece, or if the patient prefers to inhale using a mask.

2.3 Intended use


The PARI SMARTMASK is intended exclusively for use with a PARI nebuliser (not the PARI TIA) or an eFlow® nebuliser. It is designed for use by adults.

Each mask must only be used at home by a single patient for hygiene reasons. In hospitals, it may be used for multiple patients provided appropriate professional cleaning and disinfection procedures are followed rigorously.

2.4 Materials used

Polypropylene	45° elbow piece
Silicone	PARI SMARTMASK, expiratory valve
Polyester	Rubber band
Synthetic rubber (latex-free)	

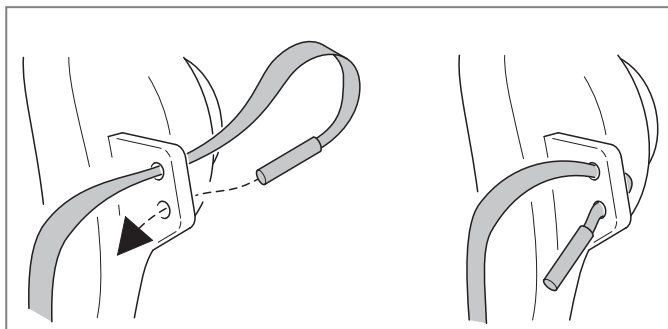
3 APPLICATION

- Attach the rubber band (4) to the mask.

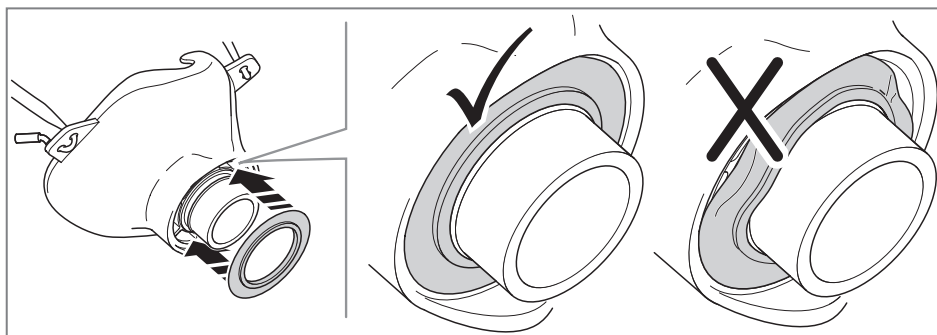
The rubber band ensures that the mask remains securely in place throughout the inhalation session.


- Attach the blue expiratory valve (2):
 - Carefully push the expiratory valve onto the mask.
 - Press the expiratory valve over the lip until it is completely seated in the notch.

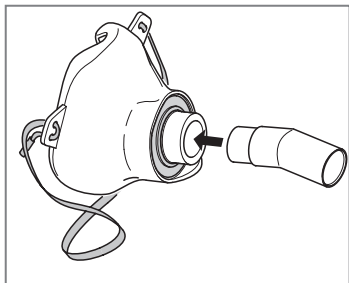Ensure that the valve is lying flat and has not become caught in the slots on the mask.

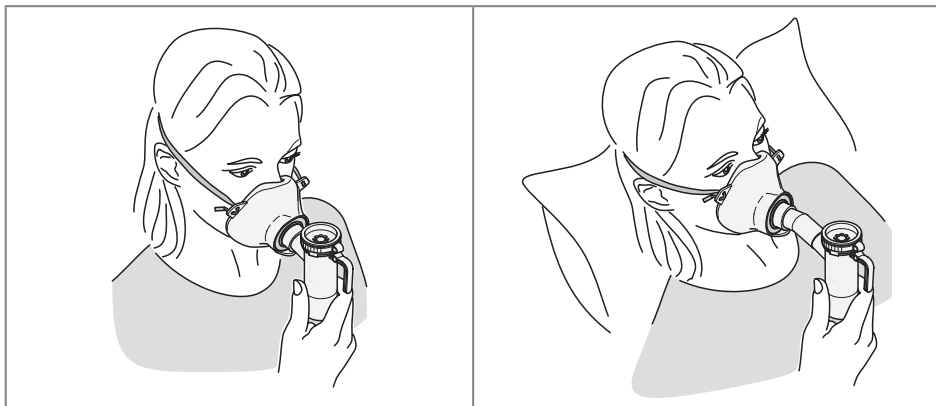- If necessary, attach the elbow piece (3).
- Attach the mask (1) to the nebuliser.

- Inhale as described in the instructions for use of the nebuliser you are using.

- Read and follow the notes on inhaling in the respective nebuliser instructions!
- Always hold the nebuliser upright during inhalation. If it is tilted too far, the medication may leak out of the nebuliser.

4 CLEANING AND DISINFECTION

PARI Nebuliser

Information for professional cleaning: The mask is suitable for cleaning by mechanical disinfection and steam sterilisation. Hygienic preparation for re-use is carried out similarly to the method used for PARI nebulisers.

The rubber band can only be cleaned as it might be damaged by the high temperatures used during disinfection.

5 EXPLANATION OF SYMBOLS

The following symbols can be found on the nebuliser and/or the packaging:

	Manufacturer
	The product satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.
	Please follow the instructions for use.
	Order no.
	Lot no.

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PARI GmbH