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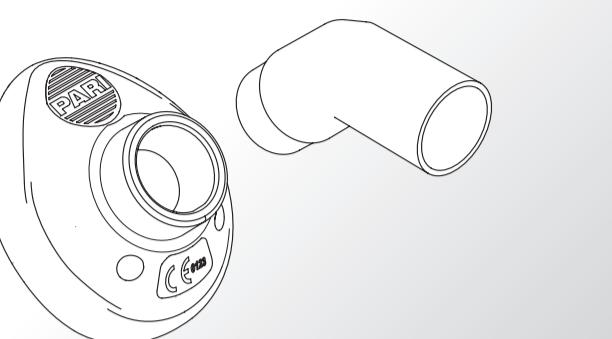
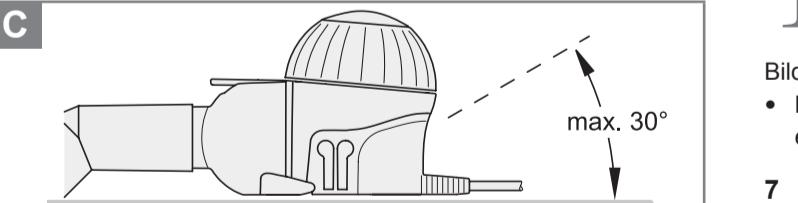
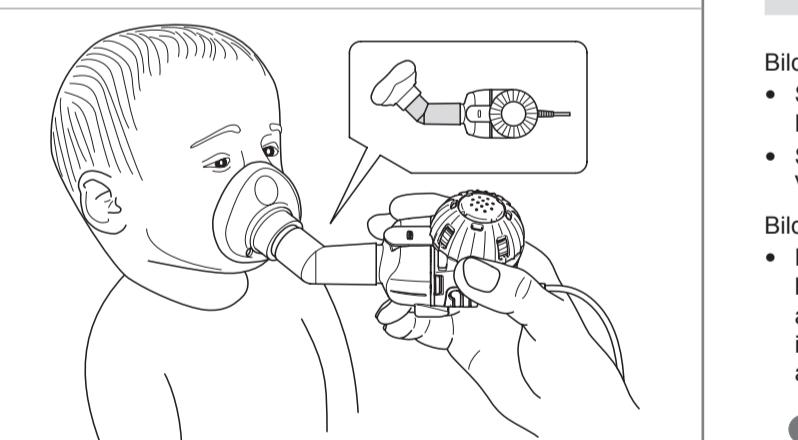
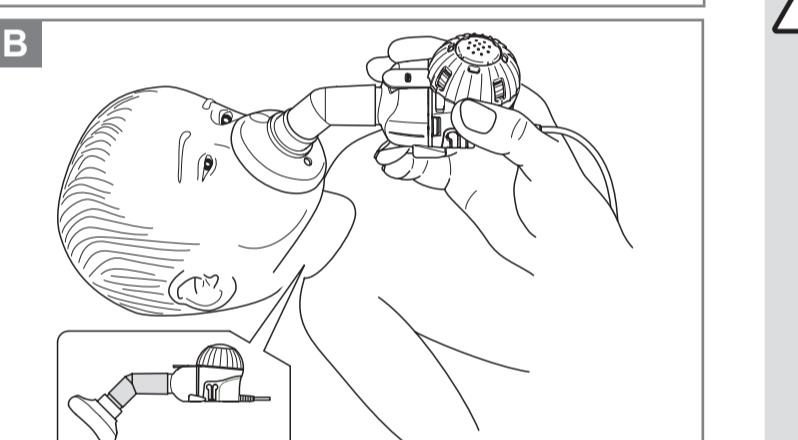
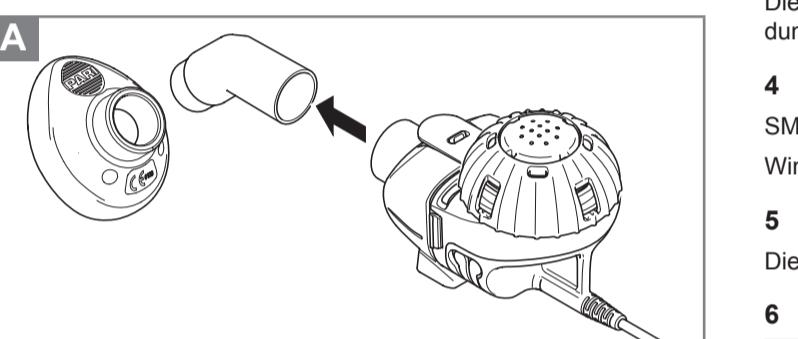
The mask may also be left in place while the baby breathes out.

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en Instructions for use

A Please read the instructions for use before operating for the first time. Keep these instructions in a safe place.

The PARI SMARTMASK® Baby with bend is always to be treated as a component or accessory of the eFlow®/rapid nebuliser handset, of which the instructions for use must also be followed accordingly. In particular, the instructions in the chapter entitled "Cleaning and disinfection" also apply for the PARI SMARTMASK® Baby with bend.

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2 Intended use

The SMARTMASK® Baby with bend is intended for inhalation treatment of the lower airways when it is not possible to use a mouthpiece (or when the patient prefers to use a mask for inhaling).

Caution:

For optimum function, the mask must cover both corners of the mouth and the nose completely. If the SMARTMASK® Baby with bend is too small, the SMARTMASK® Kids or a mouthpiece must be used.

3 Proper use

The SMARTMASK® Baby with bend is intended for use with an eFlow®/rapid nebuliser handset. It is configured so that nursing babies and infants weighing about 2,5 kg and more up to the age of two can inhale efficiently with a mask.

The SMARTMASK® Baby with bend is designed for repeated use.

4 Material information

SMARTMASK® Baby: Silicone

Bend: Polypropylene

5 Life cycle

The expected life cycle of the mask is 2 years.

6 Use of the mask

- When using the mask, make sure that it covers both corners of the mouth and the nose completely. Otherwise, aerosol may escape, reducing the effect of the treatment. For possible side effects of escaping aerosol, please read the instructions for use of the respective medication.
- Make sure that the holes in the side of the mask are not covered, so the patient can breathe in and out.
- Before using the mask for the first time, clean it according to the instructions in the chapter "Cleaning and disinfection" in the instructions for use of the eFlow®/rapid nebuliser handset you are using.
- Hold the nebuliser handset as horizontally as possible. It must not be tilted by more than 30°. This is the only way to ensure that the mask is used safely and effectively, and it prevents liquid that has not been nebulised from running into the throat of the baby or infant.

Figure A:

- Insert the matching end of the bend in the connector on the mask.
- Insert the other end of the bend in the connector on the eFlow®/rapid nebuliser handset aerosol chamber.

Figure B:

- Twist the bend and the mask to match the baby's position when inhaling (lying down or sitting up) as shown in the figure. The eFlow®/rapid nebuliser handset should remain as horizontal as possible during the inhalation session. Make sure that the mask fits correctly.

The mask may also be left in place while the baby breathes out.
Figure C:

- Do not tilt the eFlow®/rapid nebuliser handset by more than about 30°, because an incorrect dosage will be delivered.

7 Disposal

All product components can be disposed of with normal domestic waste unless other regulations exist in the respective country of use.

8 Explanation of symbols

■ Manufacturer

REF Item number

LOT Lot no.

CE 0123 CE marking: This product is compliant with the requirements set forth in 93/42/EEC (Medical Devices) and 2011/65/EU (RoHS).

! Safety instructions

i Please follow the instructions for use (background = blue, symbol = white)

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