

# User Guide:

## Nebulising with the PARI LC SPRINT®



**1** Wash your hands thoroughly before each session.



**2a** Attach the mouthpiece or the mask to the nebuliser.



**2b** If using a baby mask it must only be used in conjunction with the baby bend. Adjust the alignment of the bend and position the mask according to the position of the baby or infant.



**3** Connect the nebuliser and compressor with the connection tubing. Connect the compressor to the mains power supply.



**4** Open the nebuliser cap. Pour in the inhalation solution. Close the nebuliser cap.



**5a** Sit upright in a relaxed position and switch on the device. Take the mouthpiece between your teeth and close your lips around it.

OR



**5b** For infants, ensure that the nebuliser is upright. Gently press the mask against the face so that it fits snugly.



**6** Breathe in slowly and deeply. If your nebuliser is fitted with PIF Control, you will feel a resistance when breathing in too quickly. If you notice this, pause for a moment, then try to inhale more slowly.

When a different noise comes from the nebuliser chamber (it will change from a smooth hissing to a crackling noise), this indicates that nebulisation has finished.



**7** Switch off device. Disconnect the nebuliser chamber from the tubing. Switch the device back on to dry the tubing.



**8** Disconnect the power plug.

# User Guide: Nebulising with the PARI LC SPRINT®



**1** Disassemble the nebuliser after each use and **wash** all individual parts in warm water with dishwashing liquid, before rinsing thoroughly in running water.




**2** **Disinfect** the nebuliser once a day for 5 minutes in boiling water or for at least 6 minutes in a standard commercial disinfectant, such as one designed for baby bottles.



**3** After every cleaning and/or disinfection, place the parts on a dry, clean, absorbent surface and allow to **dry** completely before use.



**4** **Store** all components in a dry, dust-free place.

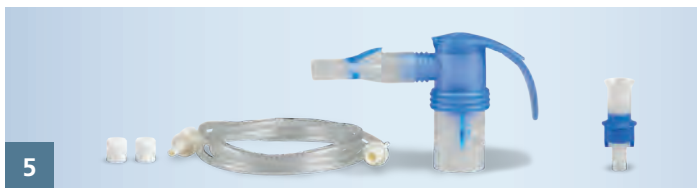

**5** Air filter, nebuliser and connection tubing should be **replaced** every year. These components are included in the **PARI Year Pack**. Components may have to be replaced sooner if they are damaged or if contamination cannot be removed.

Please ensure you read the Instructions For Use that come with the nebuliser and compressor, which give a more detailed description than this short guide. The instruction manual is included in the device package. If lost you can download the manual at [www.pari.com](http://www.pari.com) or request it by calling **01932-341122**.

## PARI Medical Ltd,

Unit 8 Oyster Park, 109 Chertsey Road,  
Byfleet, West Byfleet, Surrey KT14 7AX


Tel: +44 (0) 1932 341 122

Fax: +44 (0) 1932 341 134

Email: [infouk@pari.eu](mailto:infouk@pari.eu)

[www.pari.com](http://www.pari.com)

Scan code with your  
mobile phone camera  
to view video

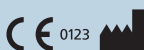

PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, GERMANY  
e-mail: [export@pari.de](mailto:export@pari.de), phone: +49 (0)8151 279 220, fax: +49 (0)8151 279 6220, [www.pari.com](http://www.pari.com)