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Model: PARI BABY Maske (Type 041)

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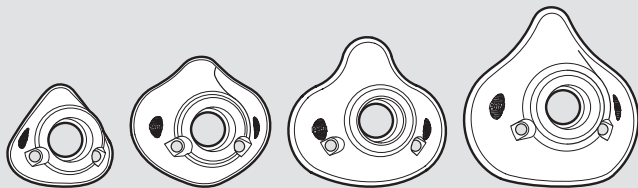
Masks for PARI inhalation systems

## **Read the instructions for use**

Read these instructions carefully before using the product. Follow all instructions and safety directions. Keep the instructions in a safe place.

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Moosstrasse 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved. The same applies to illustrations.

## **Disclaimer**

These instructions for use describe the components of PARI products and optional accessories. For this reason, these instructions for use also describe and illustrate features not present in your PARI product because they are, for instance, country-specific and/or optional. When using the systems, products and functions, the applicable country-specific regulations must be observed.

## **Trade marks**

Registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries:

LC SPRINT®, PARI®

# TABLE OF CONTENTS

<b>IMPORTANT INFORMATION</b> .....	<b>4</b>
Intended purpose.....	4
Indication .....	5
Contraindication.....	5
Labelling .....	6
Safety and warning instructions.....	7
<b>PRODUCT DESCRIPTION</b> .....	<b>9</b>
Components .....	9
Overview and designations .....	9
Product variants.....	9
Product combinations .....	9
Description of function .....	10
Material information .....	10
Service life .....	10
<b>USE</b> .....	<b>10</b>
Preparing for treatment.....	11
Performing treatment.....	12
Ending the treatment .....	12
<b>REPROCESSING</b> .....	<b>13</b>
Reprocessing cycles.....	14
Limits of reprocessing.....	15
Preparation for reprocessing .....	15
Reprocessing of the mask .....	15
Further information about reprocessing.....	18
<b>FURTHER INFORMATION</b> .....	<b>19</b>

# 1 IMPORTANT INFORMATION

## 1.1 Intended purpose

The PARI BABY mask is an accessory for nebuliser treatment. That enables inhalation of aerosol<sup>1</sup> through the mouth and nose.

The different mask sizes are suitable for treating patients in the following age groups:

- PARI BABY mask size 0: Premature babies from 2.5 kg.
- PARI BABY mask size 1: Babies aged from 0 to 1 year.
- PARI BABY mask size 2: Babies and infants aged from 1 to 3 years.
- PARI BABY mask size 3: Children aged 3 years and above.

The specified ages are approximate. The actual size of the mask depends on the size and shape of the person's face.

The mask must be used only with PARI nebulisers.

This PARI product can be used in a home environment, as well as in professional health institutions. When used in a home environment, this PARI product is intended for single-patient use only (no patient change). In a professional environment, the device can be used with different patients as long as the corresponding hygiene reprocessing measures are complied with.

This product must be used only by individuals who understand the contents of the instructions for use and are able to use the product safely.

Individuals in the following groups must be supervised by a person who is responsible for their safety:

- Babies, infants, and children
- Individuals with limited capabilities (e.g. physical, mental, sensory)

If the patient is not able to use this product safely on their own, then the treatment must be carried out by the responsible person.

This PARI product is suitable only for patients who are able to breathe by themselves and are conscious.

For nebuliser treatment in a lying position, a PARI BABY Bend must always be used.

---

1) Aerosol: Small particles of solid, liquid or mixed composition (fine "mist") suspended in gases or air.

## **1.2 Indication**

For patients who cannot inhale using a mouthpiece, or if inhalation via mask is preferred. The mask forms a system with a nebuliser. The indication for this system is the same as the indication for the nebuliser used.

## **1.3 Contraindication**

There are no contraindications known to PARI GmbH.

## 1.4 Labelling

The following symbols can be found on the product and/or the packaging:

	Medical device
	Legal manufacturer
	Date of manufacture
	Item no.
	Production batch number, lot number
	This product conforms to the EU Medical Device Regulation 2017/745.
	Unique Device Identifier (UDI)
	Consult instructions for use
	PARI BABY mask size 0
	PARI BABY mask size 1
	PARI BABY mask size 2
	PARI BABY mask size 3
	PARI BABY bend

## 1.5 Safety and warning instructions

The present instructions for use contain important information, safety instructions, and precautionary measures. The user must follow these in order to guarantee safe operation of this PARI product.

Please also follow the instructions for use of the nebuliser and compressor you are using.

### Labelling and classification of warning instructions

In these instructions for use, safety-critical warnings are categorised according to the following hazard levels:

#### **DANGER**

DANGER indicates a hazardous situation which will lead to very severe injuries or death if it is not avoided.

#### **WARNING**

WARNING indicates a hazardous situation which can lead to very severe injuries or death if it is not avoided.

#### **CAUTION**

CAUTION indicates a hazardous situation which can lead to mild or moderate injuries if it is not avoided.

#### **NOTE**

NOTE indicates a hazardous situation which can lead to material damage if it is not avoided.

## General

Skin care products containing oils or fats can damage the soft plastic components. The patient should refrain from using skin care products of this kind while using the device.

**If your health condition does not improve or it even worsens as a result of the treatment, seek professional medical advice.<sup>2</sup>**

## Hazard due to small parts which can be swallowed

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Keep all components of the product out of the reach of babies and infants at all times.

## Treatment of babies, infants, and anyone who requires assistance

For individuals who are not able to perform the therapy session without assistance or cannot appreciate the hazards, the risk of injury is greater. Such individuals include for example babies, children and people with limited capabilities. For these individuals, a person responsible for safety must supervise or carry out the application.

## Impairment of the therapy by disregarding the reprocessing instructions

### CAUTION

#### Impairment of the therapy

Exceeding the reprocessing instructions can result in damage to the product. This can impair the therapy.

Falling short of the instructions can result in insufficient reprocessing. This can increase the risk of infection.

- Comply with the instructions about the reprocessing limits, temperature, holding time, and concentration of the chemical used.

## Reporting serious incidents

Report serious adverse incidents to the manufacturer and to the competent authority.

---

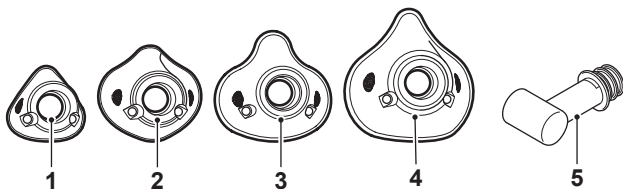
2) Professional medical staff: Doctors, pharmacists, and physiotherapists.

## 2 PRODUCT DESCRIPTION

### 2.1 Components

Please refer to the package for information on the supplied components.

### 2.2 Overview and designations

1	PARI BABY mask size 0
2	PARI BABY mask size 1
3	PARI BABY mask size 2
4	PARI BABY mask size 3
5	PARI BABY bend

### 2.3 Product variants

The PARI BABY mask is available in various sizes for different age groups:

Size	Age group
Size 0	Premature babies from 2.5 kg
Size 1	Babies aged from 0 to 1 year
Size 2	Babies and infants aged from 1 to 3 years
Size 3	Children aged 3 years and older

The specified ages are approximate. The actual size of the mask depends on the size and shape of the person's face.

### 2.4 Product combinations

The PARI BABY masks can be used in combination with all PARI nebulisers (except the PARI LC SPRINT SINUS, PARI LC SPRINT Tracheo, eFlow rapid, Velox, and PARI BOY free).

## 2.5 Description of function

The PARI BABY mask is an accessory for the PARI inhalation systems. The PARI BABY mask allows aerosol to be inhaled through the mouth and the nose. The baby or infant can breathe in and out without obstruction through the expiratory holes in the PARI BABY mask. The PARI BABY bend allows the patient to inhale while lying down. It is used as an intermediate element with which the nebuliser can be held in an upright position. The baby or infant can breathe in and out without obstruction through the ventilation slits in the PARI BABY bend.

## 2.6 Material information

The individual product components are made from the following materials:

Product component	Material
PARI BABY mask	Silicone
PARI BABY bend	Polypropylene

## 2.7 Service life

The individual product components have the following expected lifetimes:

Product component	Service life
PARI BABY mask, PARI BABY bend	Home environment [see: Limits of reprocessing in a home environment, page 15]
PARI BABY mask, PARI BABY bend	Professional environment [see: Limits of reprocessing in a professional healthcare institution, page 15]

When the expected operating life has been reached, replace the affected component.

## 3 USE

All the steps described below must be carried out properly.

Do not use product components unless they have been thoroughly cleaned and dried. Wash your hands thoroughly before every use. You must perform cleaning and disinfection before using the device for the first time.

## 3.1 Preparing for treatment

 *The mask is used instead of a mouthpiece.*

### **WARNING**

#### **Warning of shortness of breath**

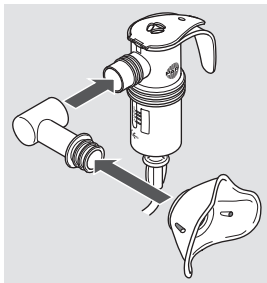
Using a PARI BABY mask without expiratory holes may lead to shortness of breath, as it is not possible to breathe out.

- Check to ensure that your PARI BABY mask has expiratory holes.
- The PARI BABY mask without expiratory holes must be used only in conjunction with the PARI BABY bend.

## With a PARI nebuliser


### **Use with PARI BABY bend**

- If applicable, detach the mouthpiece from the nebuliser.
- Attach the mask to the PARI BABY bend.  
Use your thumb to press the mask from the inside over the first ridge on the PARI BABY bend.
- Fit the PARI BABY bend onto the nebuliser.

### **Use without the PARI BABY bend**

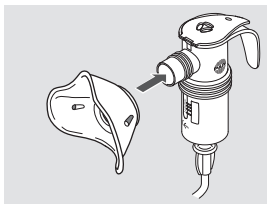- For inhalation treatment without the PARI BABY bend, push the mask to the nebuliser.

## 3.2 Performing treatment

### WARNING

#### Warning of shortness of breath

If the nebuliser is tilted by more than 70°, the inhalation solution can run through the mask and into the child's mouth and nose, resulting in shortness of breath. Nebuliser therapy can be carried out only in a recumbent position if a PARI BABY bend is used. Make sure that the holes under the mask connector are unobstructed and that the nebuliser is in its **specified** position.

### CAUTION

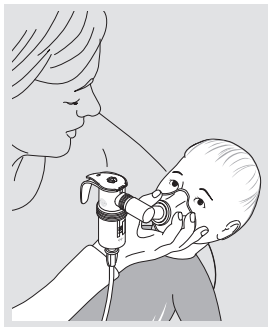
#### Impaired treatment due to escaping aerosol

If the mask does not form a seal on the face, aerosol may escape. This may result in medication underdosage.

- Make sure that the mask completely covers both corners of the mouth and the nose.
- Take note of possible side effects caused by escaping aerosol. These are described in the information for use of the respective medication.

## With a PARI nebuliser

- Adjust the alignment of the bend and the mask according to the position the baby or infant is in. Make sure that the nebuliser is upright.
- Gently press the mask snugly over the mouth and nose.

- The child should breathe in as slowly and deeply as possible through the mask and out again calmly.

## 3.3 Ending the treatment

End the therapy as specified in the appropriate instructions for use.

## 4 REPROCESSING

### CAUTION

#### **Risk of infection due to cross-contamination in the case of a change in patients**

If a product is used for more than one patient, there is a risk that germs may be transmitted from one patient to the next.

- Clean, disinfect and sterilise all separated parts before every patient change.

### CAUTION

#### **Risk of infection due to growth of bacteria**

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- During disinfection, comply with the specified holding time and the concentration of the chemical used.
- When using a disinfectant, ensure that the device is clean and operating properly. Allow the disinfection to continue until the disinfectant switches off automatically or the minimum disinfection time stated in the instructions for use of the disinfectant has elapsed. Do not switch the device off prematurely.

### CAUTION

#### **Risk of infection by residual germs**

If there is dirt on the parts, germs capable of reproduction may remain despite the sterilisation process. As a result, there is a danger of infection.

- Clean, disinfect, and dry all parts thoroughly before sterilising.
- Use only validated procedures for cleaning and disinfection.

### CAUTION

#### **Risk of infection due to moisture**

Moisture encourages the growth of bacteria.

- Ensure that the components are allowed to dry sufficiently after each processing step.

## **NOTE**

### **Risk of damage to plastic parts!**

Plastic melts if it comes into contact with hot surfaces.

- During thermal disinfection, you must ensure that the water level in the container is sufficient, in order that the individual parts do not come into contact with hot surfaces.

*Always use drinking water for reprocessing in a home environment.*

 *In professional health care institutions, drinking water is sufficient for precleaning. For all other reprocessing steps and processes, use deionised water with a low microbiological load (at least drinking water quality).*

Please observe the instructions for use for the chemicals used.

Wash your hands thoroughly before every reprocessing.

## **4.1 Reprocessing cycles**

Perform the reprocessing steps as per the specified reprocessing cycles.

### **Reprocessing cycles in a home environment**

PARI BABY mask and PARI BABY bend	– Clean immediately after every use – Disinfect once a week <sup>3</sup>
--------------------------------------	---

### **Reprocessing cycles in professional healthcare institution**

#### **SINGLE PATIENT USE**

PARI BABY mask and PARI BABY bend	– Clean immediately after every use – Disinfect once a week <sup>3</sup>
--------------------------------------	---

#### **BEFORE A CHANGE OF PATIENT**

PARI BABY mask and PARI BABY bend	– Cleaning – Disinfection – Sterilisation
--------------------------------------	---

---

3) For the therapy of patients at risk, disinfect the individual parts once per day. Take note of the additional information for patients at risk [see: Reprocessing for patients at risk, page 15].

## Reprocessing for patients at risk

### CAUTION

#### Danger of infection for patients at risk

For patients at risk, airway infections represent an increased risk of deterioration of general health, because these patients are particularly vulnerable to residual germs. Patients at risk include cystic fibrosis patients, patients with immunosuppression or immunodeficiency, and vulnerable patient groups.

- If you are a patient at risk, disinfect the individual parts once per day during the therapy.
- If you are unsure whether you are a patient at risk, consult with specialist medical personnel before use.

## 4.2 Limits of reprocessing

### Limits of reprocessing in a home environment

PARI BABY mask and PARI BABY bend, Disinfection	300 processing cycles, max. 1 year
---	------------------------------------

### Limits of reprocessing in a professional healthcare institution

PARI BABY mask and PARI BABY bend, disinfection	300 processing cycles, max. 1 year
PARI BABY mask and PARI BABY bend, sterilisation	100 processing cycles, max. 1 year

## 4.3 Preparation for reprocessing

### Procedure

#### Mask

- Disconnect all mask components from the nebuliser.
- Detach the PARI BABY bend from the mask if applicable.

## 4.4 Reprocessing of the mask

The following products can be cleaned, disinfected and sterilised according to the procedure described below:

- PARI BABY mask
- PARI BABY bend

## Procedure

### Step 1: Precleaning

All individual parts must be precleaned immediately after use.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water.

### Home environment

### Step 2: Cleaning

EQUIPMENT:

- Drinking water temperature of about 40 °C
- Standard commercial washing-up liquid
- Container having sufficient capacity

PROCEDURE:

- Unless otherwise specified by the manufacturer of the washing-up liquid, add about 1 teaspoon of washing-up liquid to 3 l warm drinking water.
- Place all the parts in the washing-up water.  
Application time: 5 minutes
- Occasionally move the parts back and forth.
- In case of visible soiling, use a medium-soft brush (e.g. a toothbrush) which is used exclusively for this purpose.

RINSING:

- Rinse off all parts thoroughly in running drinking water at about 15 °C for 3 minutes.

## Procedure

Home environment	Step 3: Disinfection	<p><b>A - Thermal disinfection using boiling water</b></p> <p>EQUIPMENT:</p> <ul style="list-style-type: none"> <li>– Clean cooking pot</li> <li>– Drinking water</li> </ul> <p>PROCEDURE:</p> <ul style="list-style-type: none"> <li>• Place all the individual parts in water at a rolling boil for at least 5 minutes.</li> </ul> <hr/> <p><b>B - Using a standard thermal disinfectant for baby bottles</b></p> <p>EQUIPMENT:</p> <ul style="list-style-type: none"> <li>– Thermal disinfectant with a runtime of at least 6 minutes.</li> </ul> <p>PROCEDURE:</p> <p>Regarding the performance of the disinfection, the duration of the disinfection procedure and the quantity of water required, follow the instructions for use of the disinfectant you are using.</p>
In professional health care institutions	Step 2: Cleaning and disinfection	<p>EQUIPMENT:</p> <ul style="list-style-type: none"> <li>– Neodisher® MediClean forte (concentration: 0.5%)</li> <li>– Deionised water</li> <li>– Cleaning and disinfection device as per ISO 15883.</li> </ul> <p>PROCEDURE:</p> <ul style="list-style-type: none"> <li>• Programme for cleaning and thermal disinfection (at least A0 = 3000) as per manufacturer's instructions.</li> </ul>
In professional health care institutions	Step 3: Sterilisation	<p>For sterilisation, use a process compliant with the ISO 17665 series of standards.</p> <p>PROCEDURE:</p> <p>Sterilisation temperature and holding time:</p> <ul style="list-style-type: none"> <li>– Temperature: 134 °C</li> <li>– Holding time: minimum 3 minutes to maximum 5 minutes</li> </ul> <ul style="list-style-type: none"> <li>• Pack all the disassembled parts in a sterile barrier system according to ISO 11607-1 (e.g. foil-paper packaging).</li> <li>• Carry out the sterilisation in a steam steriliser in accordance with the manufacturer's instructions.</li> </ul>

## Procedure

<b>Drying</b>	Ensure that there is no residual moisture remaining in the components. If necessary: <ul style="list-style-type: none"><li>• Shake the water out of all of the parts.</li><li>• Place all parts on a dry, clean and absorbent surface and allow them to dry completely.</li><li>• Using a standard thermal disinfecter for baby bottles: Leave all individual parts in the closed thermal disinfecter for max. 24 hours until the next use.</li></ul>
<b>Visual inspection</b>	Inspect all product components after each cleaning, disinfection or, where applicable, sterilisation. Replace defective, deformed or seriously discoloured parts.
<b>Storage</b>	Store this product as described below: <ul style="list-style-type: none"><li>– Dry and dust-free, e.g. in a clean, lint-free cloth (e.g. a tea towel)</li><li>– Protected against contamination, where necessary (e.g. using optional sterile packaging)</li></ul>

### 4.5 Further information about reprocessing

#### Further validated processes for reprocessing

The instructions provided were validated by PARI and were found to be suitable for preparing your medical device for its reuse.

Further validated processes for reprocessing:

[https://www.pari.com/fileadmin/041D0624\\_Professional\\_healthcare\\_institution\\_Validated\\_Reprocessing\\_Methods.pdf](https://www.pari.com/fileadmin/041D0624_Professional_healthcare_institution_Validated_Reprocessing_Methods.pdf)

Ensure that the reprocessing actually performed by your personnel on your equipment with the chemicals used achieves the desired results. For this, validation and routine monitoring of the process are usually required. In particular, if you have to deviate from our validated process, ensure that the reprocessing method selected by you is appropriately effective, and that potential adverse effects are assessed.

## **5 FURTHER INFORMATION**

### **5.1 Disposal**

All product components may be disposed of with normal domestic waste. The country-specific disposal regulations must be observed.

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## **Copyright**