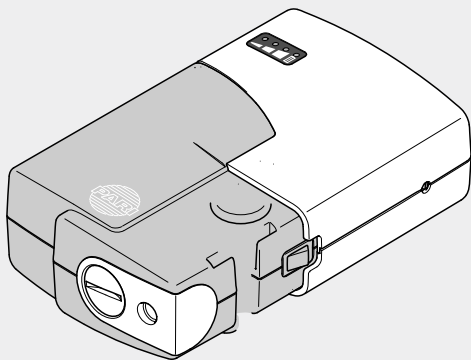
PARI BOY mobile S

Type 047

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en

Instructions for use

Information as of: April 2017. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

IMPORTANT INFORMATION	31
General.....	31
Information about the instructions for use	31
Structure of safety instructions	31
Using the compressor	32
Treatment of babies, children and anyone who requires assistance	33
PRODUCT DESCRIPTION	34
Components	34
Intended purpose	34
Intended use.....	35
Contraindications.....	35
Product combinations	35
Working parts	36
Operating life	37
PREPARATION FOR USE	37
Setting up the compressor	37
Preparing the treatment.....	38
Performing treatment.....	40
Ending the treatment	42
MAINTENANCE	42
Cleaning	42
Replacing the filter.....	43
Repairing	44
Storage.....	44
TROUBLESHOOTING	45

TECHNICAL DATA	46
General compressor data	46
Classification according to DIN EN 60601-1	47
Information about electromagnetic compatibility	47
Recommended safety distances	48
Ambient conditions	49
MISCELLANEOUS	49
Disposal.....	49
Terms and conditions of warranty	50
Explanation of symbols	51
Contact	52

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

You must also follow the instructions for use of the nebuliser, the accessories and the battery you are using.

1.2 Information about the instructions for use

If these instructions for use are lost, you can request another copy from PARI GmbH [see: Contact, page 52]. The instructions for use for some products can be retrieved on the internet in both German and English. Simply visit: www.pari.com (on the respective product page).

1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.4 Using the compressor

The compressor is an electrical device that is operated at extra-low voltage. It has been designed so that no live parts are accessible. However, if ambient conditions are unfavourable or if the compressor or power adapter is damaged, this protection may no longer be assured. Therefore, please follow the instructions below to avoid damage to the device and the associated danger of contact with live parts (e.g., electric shock):

- The compressor, the power adapter and the rechargeable battery are not splashproof. Protect all device components and the power adapter from liquid and moisture. Never take hold of the device, the rechargeable battery, the cable or the power adapter with damp hands. In addition, the device must not be operated in a damp environment.
- Only use the PARI power adapter to operate the compressor.
- Make sure that the local supply voltage complies with the voltage data marked on the compressor identification label.
- Never leave the compressor unattended while it is in use.
- For safety reasons, always disconnect the power adapter from the socket under the following circumstances:
 - if a malfunction occurs during operation
 - before cleaning and maintaining the device
 - immediately after use
- Never pull the power adapter out of the socket by the cable.
- Make sure that the power adapter cable is never kinked, pinched or jammed. Do not pull the cable over sharp edges.
- Keep the compressor, the rechargeable battery, the power adapter cable and the power adapter itself away from hot surfaces (e.g., stove top, electric fire, open fire). Otherwise, the compressor housing, the rechargeable battery, the cable insulation and/or the power adapter itself may be damaged.
- Keep the power adapter away from domestic animals (e.g., rodents). They may damage the cable insulation.

- The compressor must not be operated and/or the power adapter must be unplugged from the socket immediately if the compressor, the power adapter cable, the power adapter itself or the rechargeable battery is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock.

1.5 Treatment of babies, children and anyone who requires assistance

Babies, children and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. This is the only way to ensure safe and effective treatment. Such persons often underestimate the hazards involved (e.g., strangulation with the power adapter cable or the connection tube), thus resulting in a risk of injury.

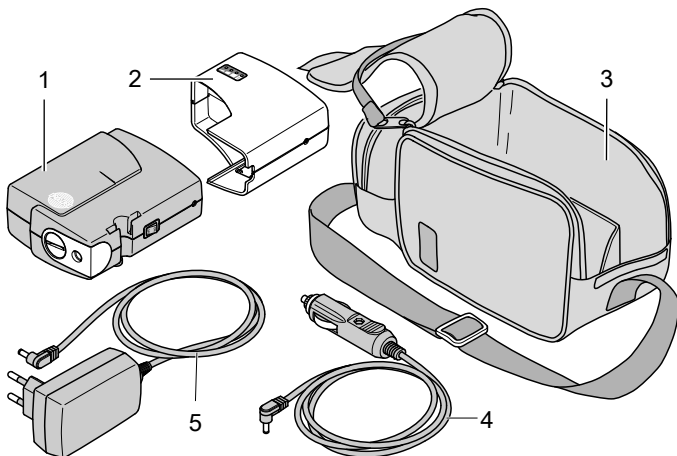
The product contains small parts. Small parts can block the airways and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

The device, the power adapter and the connection tube at the compressor outlet can become very warm during normal operation. Therefore, particularly with individuals who require special assistance, ensure that the user's skin does not come into contact with the compressor. Prolonged contact may cause burns, particularly for patients with impaired temperature perception.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all of the components belonging to your compressor are included in the package. If anything is missing, please notify the dealer from whom you purchased the PARI compressor immediately.

- | | |
|--------------------------|---|
| (1) Compressor | (4) 12 V car adapter |
| (2) Rechargeable battery | (5) Power adapter |
| (3) Storage bag | (-) Instructions for use of re-chargeable batteries (not shown) |

2.2 Intended purpose

The purpose of the compressor is to generate compressed air for operating a PARI nebuliser.

2.3 Intended use

The compressor must only be used for PARI nebulisers. It is intended exclusively for indoor use and can be operated by the patient himself/herself.

Only original PARI nebulisers and PARI accessories must be used.

2.4 Contraindications

There are no contraindications known to PARI GmbH.

2.5 Product combinations

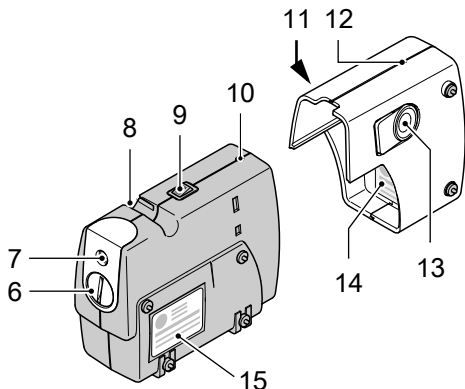
The compressor can be operated using the rechargeable battery, a 12 V car adapter, or a mains power supply.

The compressor is intended for operation with a PARI nebuliser.

Separate instructions for use accompany all nebulisers, whether they are included in the basic equipment or purchased subsequently.

2.6 Working parts

The compressor includes the following working parts:

- (6) Filter
- (7) Air outlet
- (8) Holder for nebuliser
- (9) On/off switch
- (10) Power cord socket on the compressor
- (11) Charging status display
- (12) Power cord socket on the rechargeable battery
- (13) Release button for rechargeable battery
- (14) Rechargeable battery label
- (15) Compressor label

2.7 Operating life

The compressor has an average expected operating life of:

- approx. 600 operating hours
- max. 2 years

3 PREPARATION FOR USE

3.1 Setting up the compressor

WARNING

Risk of fire due to a short circuit

A short circuit in the device can cause a fire. In order to reduce the risk of fire in such an event, follow the instructions below:

- Do not operate the compressor close to readily flammable objects such as curtains, tablecloths, or paper.
- Do not operate the compressor in areas where there is a risk of explosion or in the presence of oxidising gases (e.g., oxygen, nitrous oxide, flammable anaesthetics).

CAUTION

Risk of injury from tripping or caused by the device falling

A compressor placed in a poor position or for which the cables are routed untidily poses a risk of injury.

- Route the power adapter cable in such a way that it is not a tripping hazard and no one can become entangled in it.
- Do not place the compressor above head height and make sure that it cannot be pulled down by the power adapter cable or the connection tube.

NOTICE

Danger of device fault caused by falling

If the compressor falls, it may be damaged so badly inside and outside that it is no longer usable.

- Place the compressor on a firm, flat, dry surface. A soft base such as a sofa, a bed or tablecloth is not suitable.

NOTICE

Danger of device fault caused by dust

If the device is operated in a very dusty atmosphere, more dust may collect inside the housing and cause a fault in the device .

- Do not operate the compressor on the floor, under the bed or in workshops.
- Only operate the compressor in a low-dust environment.

The room in which the compressor is operated must satisfy certain conditions [see: Operation, page 49]. The compressor is not intended for outdoor use.

3.2 Preparing the treatment

WARNING

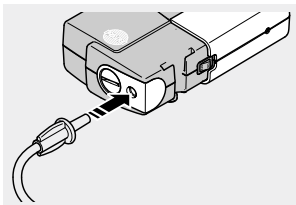
Danger of death by electrocution from a faulty device!

If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example.

- Check the compressor housing, the rechargeable battery, the 12 V car adapter or the power adapter for damage every time before using.
- Only use the compressor if all components are intact.

- Assemble the nebuliser as described in the instructions for use of the nebuliser.


- Insert the connection tube of the PARI nebuliser with a slight twist into the air outlet on the compressor.

- Insert the nebuliser in the holder on the compressor intended for this purpose.
- Fill the nebuliser as described in the instructions for use of the nebuliser.

Connecting the 12 V car adapter

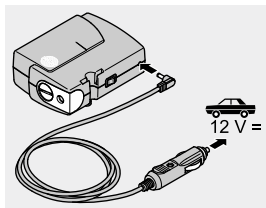- Connect the 12 V car adapter to the power cord socket on the compressor.
Alternatively: Operating with inserted rechargeable battery pack:
Connect the 12 V car adapter to the power cord socket on the rechargeable battery.

- Connect the 12 V car adapter to a suitable 12 V power supply.

Connecting the power adapter

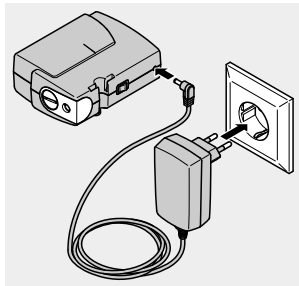
WARNING

Danger of electrocution due to damp hands

Liquids can conduct electricity, thereby posing a risk of electric shock.

- Never take hold of the power adapter with damp or wet hands.

- Connect the power adapter cable to the power cord socket on the compressor.
- Plug the power adapter into a suitable socket.

3.3 Performing treatment

WARNING

Danger of death by mixing up tubes

When several tubing systems are present, there is a possible danger of death if different connection options are confused with each other. This danger applies particularly **in the case of patients in need of constant care** who must receive artificial nutrition or other infusion treatments, for example.

- Check carefully to ensure that the other end of the PARI connection tube that is connected to the compressor is attached to the PARI nebuliser.

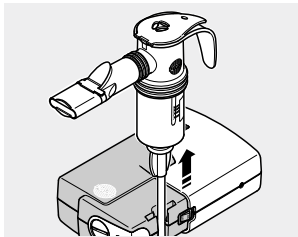
NOTICE

Danger of device fault due to overheating


Air must be able to circulate freely around the compressor to dissipate the heat generated in the housing.

- Never operate the compressor while it is in a bag.
- Never cover the compressor while it is operating.

- Before starting the treatment, pull the nebuliser out of the holder.

- Switch the compressor on.
⚠ WARNING! Danger of death by electrocution from a faulty device! If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example. Switch the compressor off and disconnect the power adapter from the mains socket immediately if a fault is suspected (e.g., following a fall or if there is a smell of burning plastic).

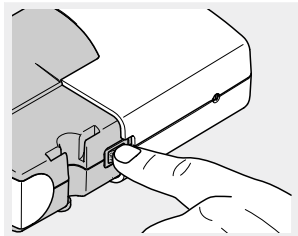- Check that an aerosol is being generated before you start with the inhalation.
- Perform the inhalation as described in the instructions for use of the nebuliser.

3.4 Ending the treatment

- Switch the compressor off.
- Place the nebuliser back in its holder.
- Pull the power adapter or the 12 V car adapter out of the socket.

i Complete disconnection from the mains is only certain when the power adapter or 12 V car adapter has been unplugged from the socket.

4 MAINTENANCE

4.1 Cleaning

WARNING

Danger of electrocution

Liquids can conduct electricity, thereby posing a risk of electric shock.

- Always switch the compressor off and disconnect the power adapter or the 12 V car adapter from the socket before starting to clean it.

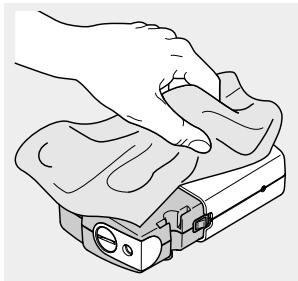
NOTICE

Danger of device fault due to liquid penetration

If liquids get into the interior of the device, this may cause a fault in the device.

- Never spray any liquids onto the compressor or the power adapter.

- Wipe the outer surface of the housing with a clean, damp cloth.

In order to clean the nebuliser, the connection tubing and the accessories, follow the notes in the respective instructions for use.

If liquid does get into the compressor, the device must not be used under any circumstances. Before using the compressor again, you must contact the PARI GmbH Technical Service department or Service Center.

4.2 Replacing the filter

NOTICE

Danger of device fault due to incorrect air filter

If air filters that are not designed for the compressor are used, the compressor may be damaged.

- Only use air filters provided by PARI for use with your compressor.

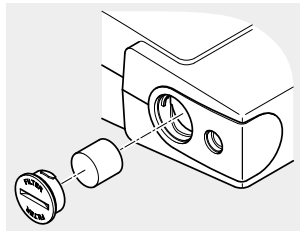
In normal operating conditions, the compressor filter must be replaced after every 200 operating hours, but at least once a year.

Also please check the filter at regular intervals (after every 10 applications). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new one if it has become damp.

The filter cannot be cleaned and then reused!

To replace the filter, proceed as follows:

- Unscrew the filter holder from your compressor using a suitable coin.
- Pull the old filter out of the filter holder and fit the new filter in its place.

- Reinsert the filter holder together with the new filter in your compressor and screw it tight with the coin.

4.3 Repairing

The compressor must only be serviced by the PARI GmbH Technical Service department or a service centre expressly authorised by PARI. If the compressor is opened or manipulated by anyone else, all claims under the warranty shall be void. In these cases, PARI GmbH will also accept no liability.

4.4 Storage

NOTICE

Damage to the power adapter cable by kinking

If the power adapter cable is wound or bent very tightly, the wires inside the cable may break. This will render the cable unusable.

- Do not wind the power adapter cable around the compressor.
- Disconnect the power adapter cord from the compressor and wind it up.
- Disconnect the rechargeable battery from the compressor.

Store the compressor and accessories safely out of continuous direct sunlight.

Always disconnect the compressor from the mains for storage because electrical devices that remain plugged in present a potential hazard source.

5 TROUBLESHOOTING

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center.

Fault	Possible cause	Procedure
The compressor does not start up.	The power adapter cable is not plugged into the device properly.	Check that the power adapter cable is seated correctly in the compressor power cord socket.
	The power adapter is not plugged into the mains socket properly.	Check that the power adapter is seated correctly in the mains socket.
No medication is coming out of the nebuliser.	The nebuliser nozzle is blocked.	Clean the nebuliser.
	The connection tube is not connected properly.	Check that both ends of the tube are seated properly.
	The connection tube is leaking.	Replace the connection tube.
All LEDs are flashing	There is a temporary fault in the rechargeable battery.	Reset the rechargeable battery as described in the instructions for use of the battery.

6 TECHNICAL DATA

6.1 General compressor data

Power adapter	
Model	FUHUA UE24WV
Supply voltage	100 V – 240 V
Mains frequency	50 Hz – 60 Hz
Output extra-low voltage	12 VDC / 1.67 A
Compressor	
Input extra-low voltage	12 VDC / 1.67 A
Power consumption	0.6 A
Housing dimensions (W × H × D)	without rechargeable battery: 8.9 × 4.5 × 11.3 cm with rechargeable battery: 8.9 × 4.5 × 14.8 cm
Compressor flow ¹	approx. 3.8 l/min
Sound pressure level ²	approx. 52 dBA
Pressure	approx. 1 bar
Weight	without rechargeable battery: 360 g with rechargeable battery: 520 g

1) Towards PARI nebuliser nozzle (Ø 0.48 mm)

2) Maximum A-evaluated sound pressure level of the compressor (in accordance with DIN EN 13544-1, section 26)

6.2 Classification according to DIN EN 60601-1

Type of electric shock protection	Protection class II
Degree of protection from electric shock of the part used (nebuliser)	Type BF
Degree of protection against water ingress in accordance with EN 60529 (IP rating)	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air, with oxygen, or with nitrous oxide	No protection
Operating mode	Intermittent operation (30 min. on, 30 min. off)

6.3 Information about electromagnetic compatibility

Electrical medical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Such equipment must only be installed and operated in accordance with EMC instructions.

Portable and mobile high-frequency communication devices can disrupt electrical medical equipment. Using accessories, converters and power cords other than those specified (with the exception of converters and power cords that the manufacturer of the medical electrical device sells as spare parts for internal components) can result in higher emission levels or lower the resistance to interference of the device.

The device must not be placed directly beside or on top of other devices for operation. If the medical electrical device must be placed beside or on top of other devices to operate it, it should be monitored constantly to ensure that it is operating properly in the arrangement used.

Technical data on electromagnetic compatibility (EMC instructions) in table format are included with the device. These can also be ordered from PARI GmbH or downloaded from the internet at www.pari.com under "Technical Data" on the respective product page.

6.4 Recommended safety distances

Recommended safety distances between portable and mobile high-frequency (HF) telecommunication devices and the device:

The product is intended for operation in an electromagnetic environment in which radiated HF transients are monitored. The user can help to prevent electromagnetic interference by observing the minimum safety distances between portable and mobile HF communication equipment (transmitters) and the product, as recommended below in accordance with the maximum output power of the communication device.

Safety distance depending on transmitting frequency:

Rated power of the transmitter	150 kHz to 80 MHz $d = (3.5/3) \times \text{SQRT}(P)$	80 MHz to 800 MHz $d = (3.5/3) \times \text{SQRT}(P)$	800 MHz to 2.5 GHz $d = (7/3) \times \text{SQRT}(P)$
0.01 W	0.2 m	0.2 m	0.3 m
0.1 W	0.4 m	0.4 m	0.8 m
1 W	1.2 m	1.2 m	2.3 m
10 W	3.8 m	3.8 m	7.3 m
100 W	12 m	12 m	23 m

For transmitters whose rated power does not appear in the table above, the safety distance can be calculated using the equation associated with the respective column, where "P" is the rated power of the transmitter in Watt (W) according to the information supplied by the device manufacturer.

6.5 Ambient conditions

Operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 hPa

Transportation and storage

The data for the rechargeable battery appears in the instructions for use that accompany the battery.

Minimum ambient temperature (without monitoring of relative humidity)	-25 °C
Maximum ambient temperature (with relative humidity of up to 93%, non-condensing)	+70 °C
Humidity	max. 93%
Atmospheric pressure	500 hPa – 1,060 hPa

7 MISCELLANEOUS

7.1 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE)³. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers). Materials recycling helps to reduce the consumption of raw materials and protect the environment.

3) Directive 2012/19/EU of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of July 4, 2012 on waste electrical and electronic equipment.

7.2 Terms and conditions of warranty

PARI guarantees that your device, if used according to the instructions, will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate, beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided by PARI applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects are not limited by the warranty or any claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

What does the warranty cover?

If, exceptionally, a defect is discovered, PARI will at its discretion repair or replace the device, or refund the purchase price of the product. If it is replaced, the replacement device may either be the same model or a model that is at least comparably equipped. Replacement or repair of the device shall not serve as the basis for a new warranty. All replaced old devices or parts shall become the property of PARI. Further claims are excluded. This applies particularly for any claims for compensatory damages. This disclaimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful wrongdoing and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

The warranty shall be cancelled if

- the device has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, or the like
- the damage was caused by transporting the device incorrectly or a falling impact
- the device has been misused or not cared for correctly

- the serial number on the device has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the device by persons not authorised by PARI

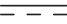
Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the entire device to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

7.3 Explanation of symbols

The following symbols can be found on the compressor and/or the packaging:

	Please follow the instructions for use.
	Serial number of the device
	Direct current
	Alternating current
	Protection class of the part used: Type BF
	Protect device from moisture
	Protection class II device
	Charging process display

	Minimum and maximum temperatures for transport and storage
	Minimum and maximum humidity
	Minimum and maximum air pressure
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of the refuse bin with a cross through it indicates that it must be disposed of separately.
 Li-ion	Recycling (lithium ions)
	Manufacturer
 0123	This product satisfies the requirements of 93/42/EEC (Medical devices) and 2011/65/EU (RoHS).

7.4 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)

+49 (0)8151-279 220 (international)

PARI BOY mobile S

Type 047

CERTIFICATE OF GUARANTEE

We grant a 2 year guarantee on the compressor,
commencing on the date of purchase.

Application
no:

Bar code label

Confirmation of purchase:

The appliance with the above appliance number was
sold in its original packaging.

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©2017 PARI GmbH Spezialisten für effektive Inhalation, 047D1022-B 08/17