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en PARI NaCl inhalation solution (0.9%) for moistening the airways

Contents: Hypertonic sodium chloride inhalation solution (0.9% NaCl) - sterile

Package sizes: 20 × 2.5 ml; 60 × 2.5 ml; 120 × 2.5 ml; 10 × 5 ml; 20 × 5 ml

Indication: The PARI NaCl inhalation solution is a natural, isotonic saline solution (0.9%) in a concentration adapted for compatibility with human body fluids. It can be used:

- to moisten the airways and as secondary treatment for cold infections such as coughing or nasal congestion
- as a carrier solution for diluting medications that may be diluted with isotonic saline solution for inhalation according to the manufacturer's instructions (for dilution information, please read the package insert for the medication concerned)

When used according to the instructions, the PARI NaCl inhalation solution may be administered to infants, people of advanced years and to pregnant and nursing women.

Contraindications: If you have a hypersensitive bronchial system, consult your doctor or pharmacist before use.

Method of administration: Inhalation with jet nebuliser, ultrasonic nebuliser or membrane nebuliser. Follow the instructions for use of the nebuliser you are using.

Application temperature: Room temperature

Quantity: 2.5 ml to 5 ml up to 4 times a day as required.

Side effects: When inhaling saline solutions, patients with a hypersensitive bronchial system (e.g., asthma) may experience shortness of breath due to temporary constriction of the bronchia. In this case, discontinue the treatment and talk to your doctor or pharmacist.

Storage: Do not use ampoules that have been opened or damaged, or if solutions are discoloured or cloudy. Since the solution contains no preservatives, opened ampoules must be disposed of immediately for reasons of hygienic safety. The inhalation solution must be disposed of after the expiry date.

Keep out of the sight and reach of children.

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©2018 PARI GmbH Spezialisten für effektive Inhalation, 077D0006-N 2018-10-05
PARI GmbH