
MONTESOL

Nasenspülung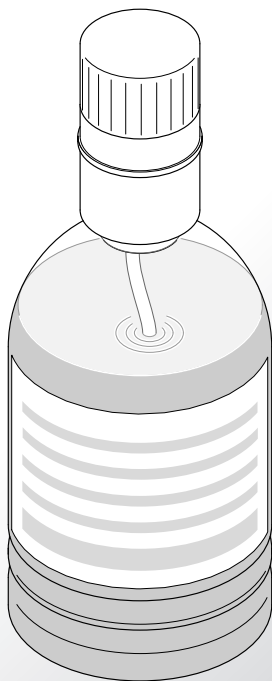

de	Gebrauchsanweisung	3
en	Instructions for use	11
es	Instrucciones de uso	17
fr	Notice d'utilisation	25
it	Istruzioni per l'uso	33
nl	Gebruiksaanwijzing	41
pl	Instrukcja obsługi	49
pt	Manual de instruções	57
ru	Руководство по эксплуатации	65
sk	Návod na použitie	73

de

PARI MONTESOL

Nasenspülung

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nasal rinse

with dexpanthenol

Information as of: 2016-06. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

IMPORTANT INFORMATION.....	12
ACTIVE INGREDIENTS.....	12
CONTENTS.....	13
AREAS OF USE.....	13
RESTRICTIONS.....	13
SIDE EFFECTS.....	14
APPLICATION	14
FREQUENCY OF USE.....	15
STORAGE.....	15
EXPLANATION OF SYMBOLS.....	16

1 IMPORTANT INFORMATION

Read these instructions before using the product for the first time. Keep these instructions in a safe place.

Keep the PARI MONTESOL nasal rinse out of the reach of children.

Do not use the PARI MONTESOL nasal rinse for longer than 2 months after opening. Make a note of the opening date on the bottle label.

Dispose of the PARI MONTESOL nasal rinse after the expiry date.

People with limited mental, physical or sensory capabilities and children do not appreciate the dangers that may be associated with using the PARI MONTESOL nasal rinse. Never leave anyone from these groups unsupervised with the PARI MONTESOL nasal rinse. For example, there is a risk that the cap may be swallowed and block the airways, or that the concentrate may be used without diluting it first.

If the rinsing fluid gets into the eyes, flush with plenty of water.

The PARI MONTESOL nasal rinse with dexpanthenol is a saturated (25% - 27%) saline solution and is not intended for ingestion.

Ensure that the PARI MONTESOL nasal rinse has been diluted before use.

The PARI MONTESOL nasal rinse can be used in conjunction with the PARI MONTESOL nasal douche.

2 ACTIVE INGREDIENTS

Natural saline solution	98.8%
Aqueous dexpanthenol solution (75%)	1.2%

Contains no preservatives.

3 CONTENTS

250 ml concentrated solution, enough for 30 rinses.

4 AREAS OF USE

The PARI MONTESOL nasal rinse, supplemented with the soothing effect of dexpanthenol, is used in diluted, isotonic form for thoroughly rinsing and cleaning the nasal mucous membrane. Rinsing the nose helps to remove germs, dust and dirt from the nasal passages. It is particularly beneficial after exposure to a very dusty atmosphere, including heavy pollen if the individual suffers from pollen allergies. Nasal rinses are also used to moisturise the nasal mucous membrane and soothe the effects of dry air – e.g., in heated or air conditioned rooms. They can be used to loosen encrustations in blocked nasal passages. A nasal rinse also helps with treating acute inflammations of the paranasal sinuses and preventing cold infections.

5 RESTRICTIONS

The PARI MONTESOL nasal rinse must not be used if the following conditions exist:

- injury to the nasal roof, the walls of the paranasal sinuses or the base of the skull
- paralysis of the soft palate
- nosebleeds
- tendency to accidental swallowing
- individual intolerance or hypersensitivity to the ingredients of the saline solution listed above

6 SIDE EFFECTS

When used as directed, mild irritations may occur very occasionally, such as tingling or stinging in the nose, and intolerance in individuals (headache and more abundant mucous flow). If you use cold remedies with local action (e.g., decongestant nasal drops), you should allow a sufficient period of time between use of the two preparations.

Other restrictions or interactions with other agents are not known when the product is used as intended.

7 APPLICATION

- Unscrew the cap on the PARI MONTESOL nasal rinse supply bottle but leave the cap on the bottle.
- Hold the bottle upright.
- Squeeze the bottle gently. The concentrated saline solution will rise up into the dosing chamber.
- Keep squeezing the bottle until the fluid reaches the dosage mark. Excess saline solution will run back into the bottle from the dosing chamber when you stop pressing the sides of the bottle.

- Take the cap off the supply bottle.
- For hygiene reasons, avoid touching the inside of the dosing chamber.
- Pour the contents of the dosing chamber into the cleaned nasal douche.

- Screw the cap onto the supply bottle to close it.
- Fill the nasal douche with water up to the bottom edge of the screw thread. Use drinking water at 37 °C, that is to say at normal body temperature.

CAUTION

Only use drinking water to prepare the nasal rinse solution. People who are at higher risk of infection (for example, people with a weakened immune system, mucoviscidosis/CF patients) must use boiled or sterile water.

Please do not keep any residual quantities of the nasal rinse solution. Please dispose of the residues. Otherwise there is a danger of bacterial contamination. Nasal rinse solutions must always be fresh when used.

To learn about the rest of the procedure, please read the instructions for use of the nasal douche.

8 FREQUENCY OF USE

You can rinse your nose with 250 ml isotonic solution up to 3 times a day as necessary. Application once a day is recommended. To cleanse and clean the nose over a prolonged period, we recommend rinsing 2-3 times per week.

If your condition does not improve after 14 days, consult your doctor or pharmacist.

9 STORAGE

Always keep the bottle closed tightly.
Protect from sunlight.

10 EXPLANATION OF SYMBOLS

	Manufacturer
	Lot no.
	Use by
	Please follow the instructions for use.
	Protect from sunlight.
	Order no.
	The product satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.
	Store out of the reach of children.

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PARI GmbH