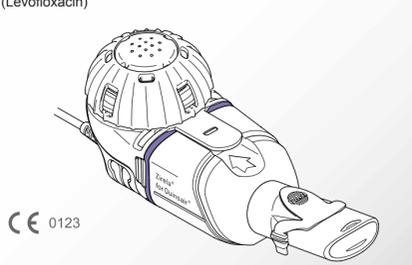
# Zirela®

nebuliser handset

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**www.pari.com/zirela**

## en Instructions for Use Zirela® nebuliser handset

- *Read these instructions carefully to learn how to use your Zirela® nebuliser handset.*
- *Read the **instructions for use enclosed with the eFlow® rapid nebuliser system** before first use. They include important additional information about use of the Zirela® nebuliser handset in particular in relation to connection, operation, cleaning and disinfection.*

### 1 Intended use

The Zirela® nebuliser handset is intended to be used with an eFlow®*rapid* control unit (type 178) or an eBase Controller (type 678).

Use only with Quinsair® (Levofloxacin) for the treatment of lung infections caused by *Pseudomonas aeruginosa* in adults with cystic fibrosis. Also follow the package leaflet of this medicine.

- !** – Do not fill Quinsair® in any other nebuliser handset than Zirela®. Quinsair® is the only product to demonstrate safety and efficacy in the Zirela® nebuliser handset.
- The Zirela® nebuliser handset is intended for use by one person only and must not be used by more than one patient.

Patient group: 18 years and older

Nebulisation time: about 5 minutes

### 2 Use of nebuliser handset

- Disassemble, clean and disinfect the nebuliser handset including the aerosol head according to the instructions for use enclosed with the eFlow®*rapid* nebuliser system before first use.
- Assemble the nebuliser handset as shown in the pictures:
  - 1** Open the nebuliser handset
  - 2** Insert the aerosol head – **Do not touch the membrane (metal part, CENTER) of the aerosol head!**
  - 3** Place the inspiratory valve
  - 4** Close the nebuliser handset
  - 5** Attach the mouthpiece
  - 6** Mount the connection cord
  - 7** Fill in the medication, empty one ampoule completely
  - 8** Close the medication reservoir
  - 9** Sit in an upright position and relax

This makes inhalation easier and improves deposition of the medication in the respiratory tract.

### 3 Cleaning and disinfection

The disassembled nebuliser handset (incl. aerosol head) must be cleaned immediately after each use and disinfected at least once a day (see instructions for use enclosed with the eFlow®*rapid* nebuliser system).

### 4 Replace your handset

Each 28-day pack of Quinsair® contains a Zirela® nebuliser handset. Use a new handset for each treatment cycle.

### 5 Content of the carton

One Zirela® nebuliser handset including a Zirela® aerosol head.

### 6 Explanation of symbols

	Manufacturer
<b>REF</b>	Article number
<b>SN</b>	Serial number
CE 0123	CE marking: This product is compliant with the requirements set forth in 93/42/EEC (Medical Devices) and 2011/65/EU (RoHS).
<b>!</b>	Safety warning
	Follow the instructions for use (background = blue, symbol = white)
	The medical device was distributed commercially after 13 August 2005. <b>This product must not be disposed of with normal domestic waste.</b> The symbol of a crossed-through refuse bin indicates that separate collection is required.

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Information about particle size distribution compliant with DIN EN 13544-1 is available on request from PARI Pharma.

Further information is available from your local service partner.

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