
# Altera® nebuliser handset

Nur für Cayston®  
(Aztreonam Lysin).For use only with Cayston®  
(aztreonam lysine).


REF 678G8210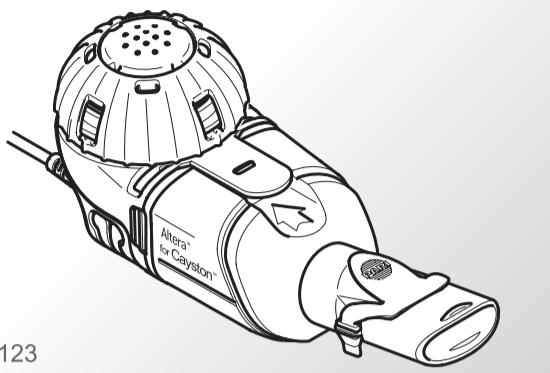

CE 0123

eFLOW®  
TECHNOLOGY

PARI

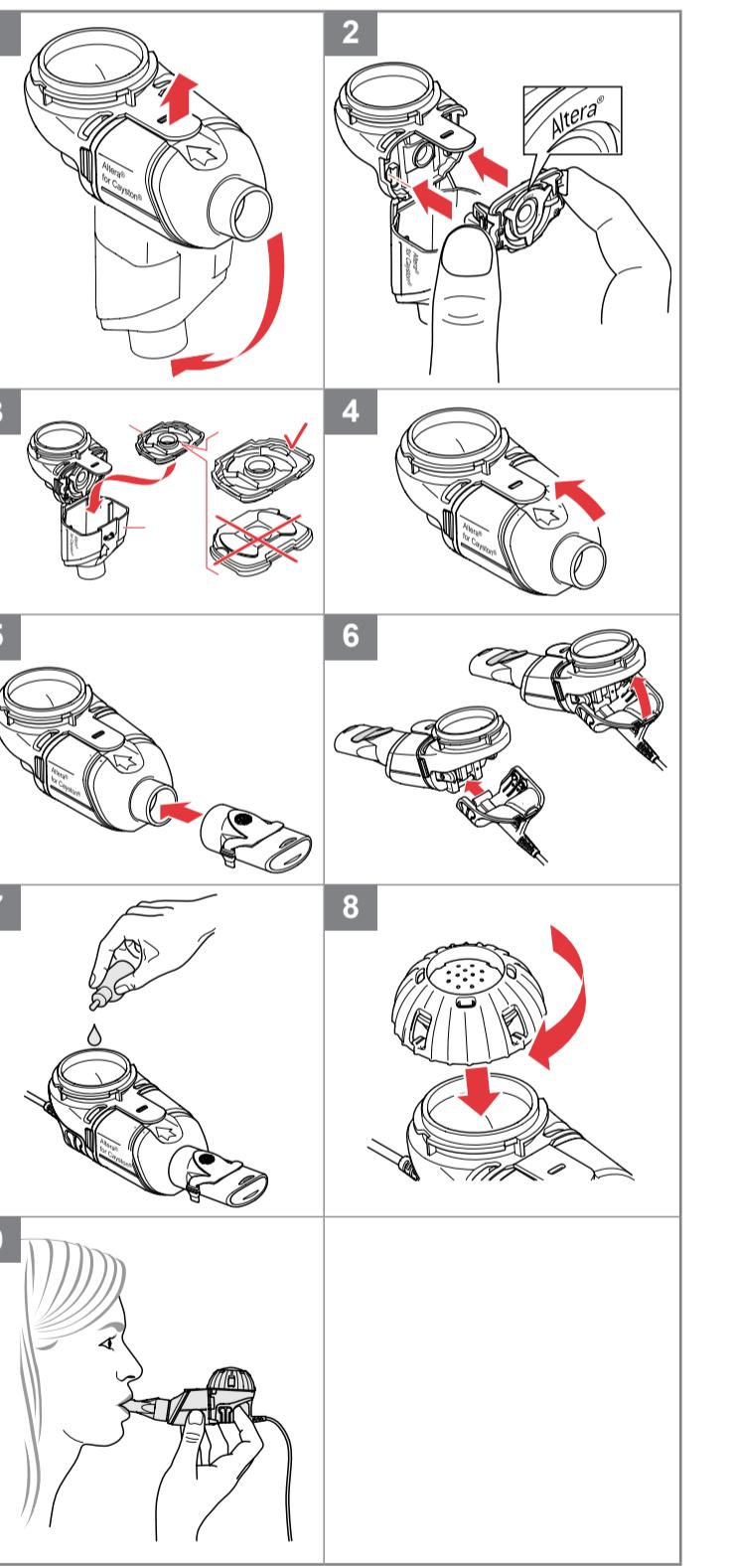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[www.alteranebuliser.info](http://www.alteranebuliser.info)

## en Instructions for use Altera® nebuliser handset

- Read these instructions carefully to learn how to use your Altera® nebuliser handset.
- Make sure you also read the **instructions for use enclosed to the eFlow®/rapid nebuliser system** before first use. They include important additional information about the use of the Altera® nebuliser handset in particular in relation to connection, operation, cleaning and disinfection.

### 1 Intended use

The Altera® nebuliser handset is intended to be used in combination with an eFlow®/rapid control system (type 178) or an eBase Controller (type 678).

It is only for use in combination with Cayston® (aztreonam lysine) for the treatment of respiratory diseases. Follow also the package leaflet of this medicine.

- Do not fill Cayston® in any other nebuliser handset than Altera®, as this will cause delivery of an incorrect dosage.
- The Altera® nebuliser is intended for use by one person only and must not be used by more than one patient.

Patient group: 6 years and older

Nebulisation time: 2 - 3 minutes

### 2 Use of nebuliser handset

- Clean and disinfect the nebuliser handset including the aerosol head according to the instructions for use enclosed to the eFlow®/rapid nebuliser system before first use.

- Assemble the nebuliser handset as shown in the pictures:

- 1 Open the nebuliser handset
- 2 Insert the aerosol head – **Do not touch the membrane (metal part, CENTER) of the aerosol head!**
- 3 Place the inspiratory valve
- 4 Close the nebuliser handset
- 5 Attach the mouthpiece
- 6 Mount the connection cord
- 7 Fill in the medication
- 8 Close the medication reservoir
- 9 Sit in an upright position and relax  
This makes inhalation easier and improves deposition of the medication in the respiratory tract.

Data for use of the nebuliser handset with mask are not available.

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### 4 Hygienic re-processing

The nebuliser handset (incl. aerosol head) must be cleaned immediately after each use and disinfected at least once a day (see instructions for use enclosed to the eFlow®/rapid nebuliser system).

### 5 Content of the carton

One Altera® nebuliser handset including an Altera® aerosol head.

### 6 Explanation of symbols

	Manufacturer
<b>REF</b>	Article number
<b>SN</b>	Serial number
<b>CE</b> 0123	CE marking: This product is compliant with the requirements set forth in 93/42/EEC (Medical Devices) and 2011/65/EU (RoHS).
	Safety warning
	Follow the instructions for use (background = blue, symbol = white)
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of a crossed-through refuse bin indicates that separate collection is required.

### 7 Particle size distribution

Information about particle size distribution compliant with DIN EN 13544-1 is available on request from PARI Pharma.

Further information is available from your local service partner.

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