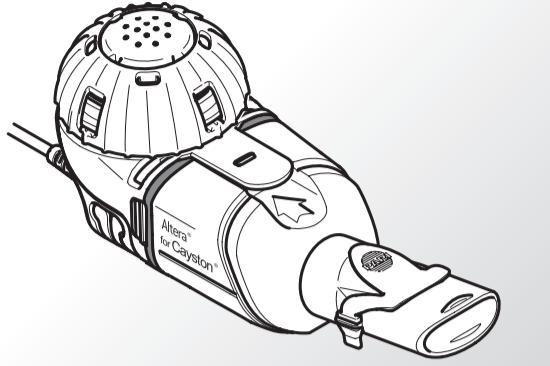
# Altera®

nebuliser handset



REF 678G8210

CE 0123

eFlow®  
TECHNOLOGY

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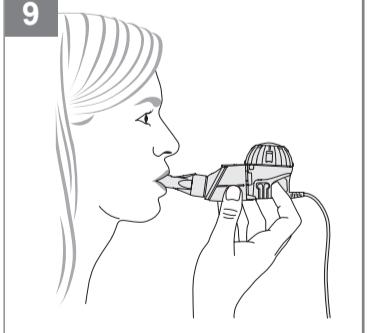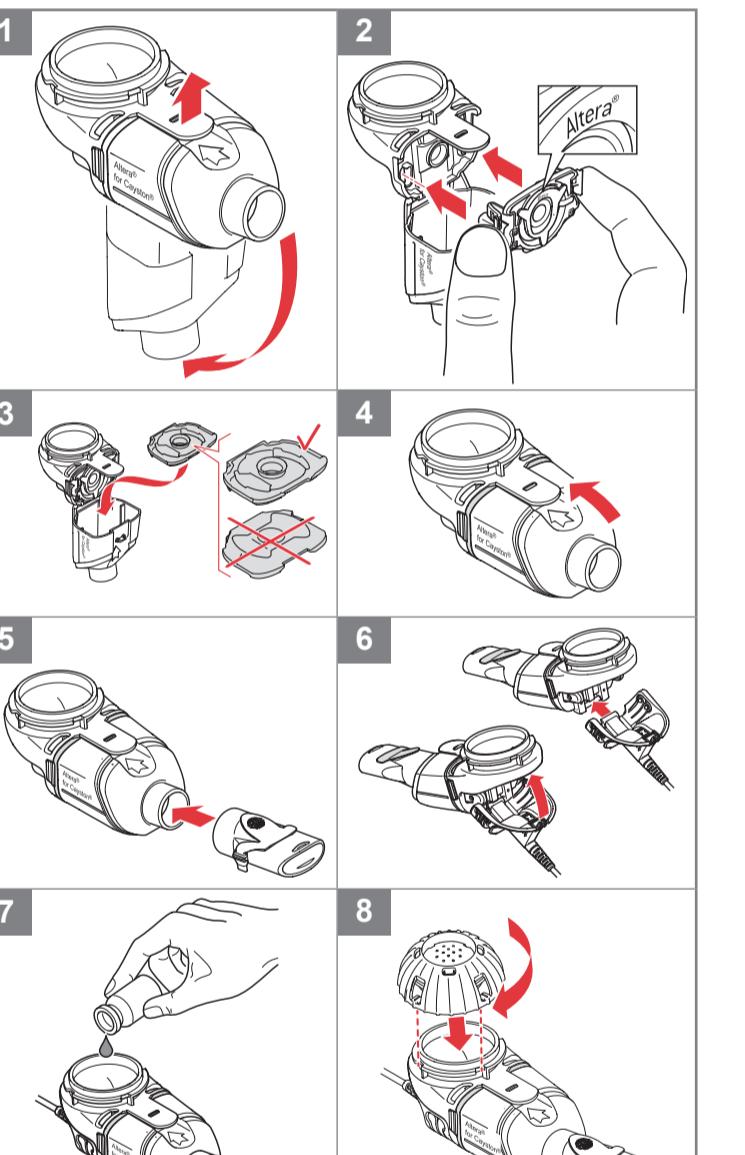## en Instructions for use Altera® nebuliser handset

- Read these instructions for use carefully and completely.
- Read the **instructions for use of the eFlow®/rapid nebuliser system** completely before using for the first time. These contain further important instructions for the use of the Altera® nebuliser handset, particularly regarding connection, operation, cleaning and disinfection.

Report serious incidents to the manufacturer and the competent authority.

### 1 Intended purpose

The Altera® nebuliser handset with an eBase® Controller is intended to administer Cayston® as an aerosol for inhalation.

To ensure correct dosing, the Altera® nebuliser handset must only be used with Cayston®.

### 2 Indication

The Altera® nebuliser handset is indicated to administer Cayston® to treat lung infections caused by *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

### 3 Contraindication

For the Altera® nebuliser handset no contraindication is known to PARI Pharma GmbH.

Take note of contraindications of the medicine in the Cayston® package leaflet.

### 4 Use of the nebuliser handset

The Altera® nebuliser handset must only be used in accordance with its intended purpose. For reasons of hygiene and to avoid cross-infections, the Altera® nebuliser handset must be used by one patient only.

- Before using for the first time, disassemble and clean the nebuliser handset including the aerosol head as described in the instructions for use of the eFlow®/rapid nebuliser system.
- Assemble the nebuliser handset as shown in the diagram:

1 Open the nebuliser handset.

2 Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**

3 Insert the inspiratory valve.

4 Close the nebuliser handset.

5 Attach the mouthpiece.

6 Connect the connection cord.

7 Fill with the medication.

8 Close the medication reservoir.

9 Sit in an upright position and relax.

This makes inhalation easier and improves uptake of the medication in the airways.

**! Hold the nebuliser handset horizontally all the time while performing inhalation. If the handset is tilted, this may result in the incorrect dosage being delivered.**

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The inhalation session should last between 2 and 3 minutes. Inhalation is ended as soon as no more aerosol is generated.

- Always check the medication reservoir to ensure that the entire dose of Cayston® has been nebulised.

### 5 Cleaning and disinfection

The dismantled nebuliser handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®/rapid nebuliser system).

### 6 Replacing the nebuliser handset

Each 28-day pack of Cayston® contains an Altera® nebuliser handset. Use the new nebuliser handset for each treatment cycle.

### 7 Package contents

One Altera® nebuliser handset with one Altera® aerosol head. There are no accessories for the Altera® nebuliser handset.

### 8 Symbols

	Medical device
	Follow the instructions for use
	Single patient – multiple use
	Lot number
	Unique Device Identifier
	Item number
	Serial number
	Minimum and maximum ambient temperature
	Minimum and maximum humidity
	Minimum and maximum air pressure

	CE marking: This product complies with the European requirements for medical devices (EU 2017/745). Do not dispose in domestic waste. This product must be disposed in accordance with the applicable (national, county or municipal) regulations.
	Caution
	Manufacturer
	Date of manufacture

### 9 Service and contact

In case of technical problems or questions about the device, please contact your service partner. The contact addresses of your responsible service partner as well as a digital version of the instructions for use of your Altera® nebuliser handset and performance characteristics determined for Cayston® can be found at:

[www.pari.com/altera](http://www.pari.com/altera)

We reserve the right to make technical changes. Information as of: 2023-05

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