Altera®

Nebulizer Handset

REF

678G8210

de

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fr

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en

Instructions for Use

da

Brugsanvisning

es

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it

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
nl

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
sv

Bruksanvisning


1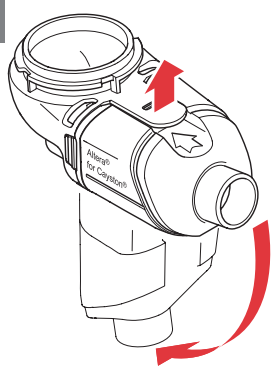


2

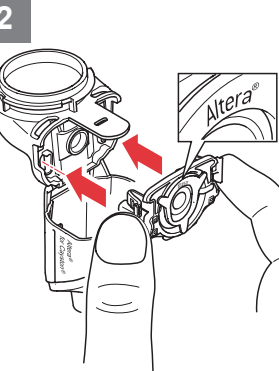
3

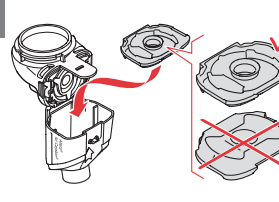
4

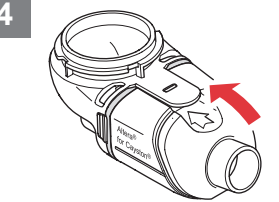
5

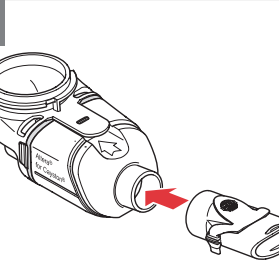
6

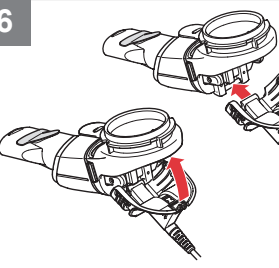
7

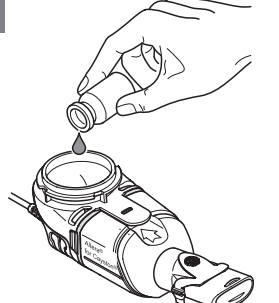8

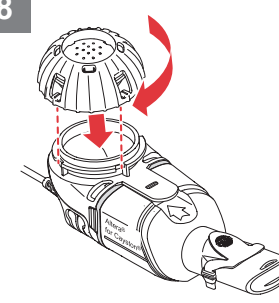9

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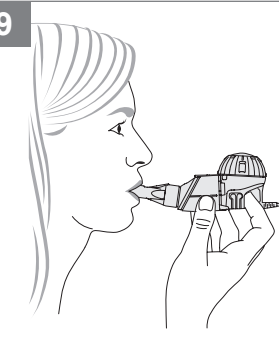en Instructions for use Altera® Nebulizer Handset

i

- *Read these instructions for use carefully and completely.*
- *Read the **instructions for use of the eFlow®rapid nebuliser system** completely before using for the first time. These contain further important instructions for the use of the Altera® Nebulizer Handset, particularly regarding connection, operation, cleaning and disinfection.*

Report serious incidents to the manufacturer and the competent authority.

1 Intended purpose

The Altera® Nebulizer Handset with an eBase® or eTrack® Controller is intended to administer Cayston® as an aerosol for inhalation.

i To ensure correct dosing, the Altera® Nebulizer Handset must only be used with Cayston®.

2 Indication

The Altera® Nebulizer Handset is indicated to administer Cayston® to treat lung infections caused by *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

3 Contraindication

For the Altera® Nebulizer Handset no contraindication is known to PARI Pharma GmbH. Take note of contraindications of the medicine in the Cayston® package leaflet.

4 Use of the nebulizer handset

The Altera® Nebulizer Handset must only be used in accordance with its intended purpose. For reasons of hygiene and to avoid cross-infections, the Altera® Nebulizer Handset must be used by one patient only.

- Before using for the first time, disassemble and clean the nebulizer handset including the aerosol head as described in the instructions for use of the eFlow®rapid nebuliser system.
- Assemble the nebulizer handset as shown in the diagram:
 - Open the nebulizer handset.
 - Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**
 - Insert the inspiratory valve.
 - Close the nebulizer handset.
 - Attach the mouthpiece.
 - Connect the connection cord.
 - Fill with the medication.
 - Close the medication reservoir.
 - Sit in an upright position and relax. This makes inhalation easier and improves uptake of the medication in the airways.

i Hold the nebulizer handset horizontally all the time while performing inhalation. If the nebulizer handset is tilted, this may result in the incorrect dosage being delivered.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The inhalation session should last between 2 and 3 minutes. Inhalation is ended as soon as no more aerosol is generated.

- Always check the medication reservoir to ensure that the entire dose of Cayston® has been nebulized.

5 Cleaning and disinfection

The dismantled nebulizer handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®rapid nebuliser system).

6 Replacing the nebulizer handset

Each 28-day pack of Cayston® contains an Altera® Nebulizer Handset. Use the new nebulizer handset for each treatment cycle.

7 Package contents

One Altera® Nebulizer Handset with one Altera® Aerosol Head. There are no accessories for the Altera® Nebulizer Handset.

8 Symbols

MD	Medical device
	Follow the instructions for use
	Single patient – multiple use
LOT	Lot number
UDI	Unique Device Identifier
REF	Item number
SN	Serial number
	Minimum and maximum ambient temperature
	Minimum and maximum humidity
	Minimum and maximum air pressure
CE ⁰¹²³	CE marking: This product complies with the European requirements for medical devices (EU 2017/745).
	Do not dispose in domestic waste. This product must be disposed in accordance with the applicable (national, county or municipal) regulations.
	Caution
	Manufacturer
	Date of manufacture

9 Service and contact

In case of technical problems or questions about the device, please contact your service partner. The contact addresses of your responsible service partner as well as a digital version of the instructions for use of your Altera® Nebulizer Handset and performance characteristics determined for Cayston® can be found at:

www.pari.com/altera

We reserve the right to make technical changes.
Information as of: 2024-09

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