Altera®

Nebulizer Handset

REF

678G8210

en

Instructions for Use

no

Bruksanvisning

ar

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
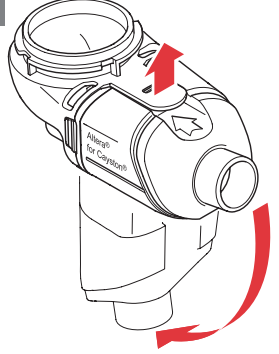
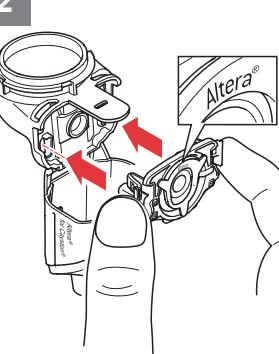
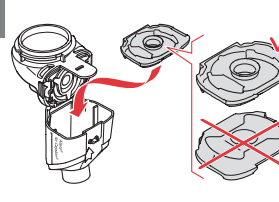
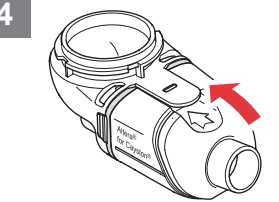
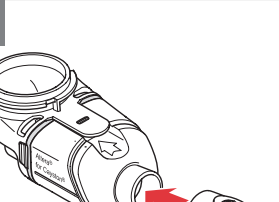
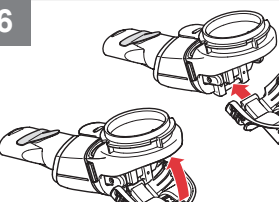
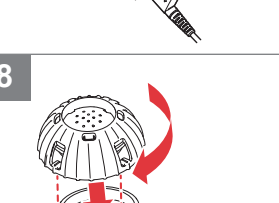
9

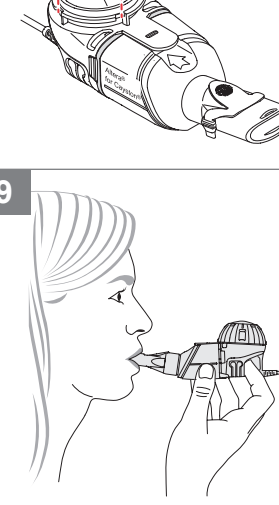Instructions for use
Altera® Nebulizer Handset

- Read these instructions for use carefully and completely.
- Read the **instructions for use of the eFlow®rapid nebuliser system** completely before using for the first time. These contain further important instructions for the use of the Altera® Nebulizer Handset, particularly regarding connection, operation, cleaning and disinfection.

Report serious incidents to the manufacturer and the competent authority.

1 Intended purpose

The Altera® Nebulizer Handset with an eBase® or eTrack® Controller is intended to administer Cayston® as an aerosol for inhalation.

To ensure correct dosing, the Altera® Nebulizer Handset must only be used with Cayston®.

2 Indication

The Altera® Nebulizer Handset is indicated to administer Cayston® to treat lung infections caused by *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

3 Contraindication

For the Altera® Nebulizer Handset no contraindication is known to PARI Pharma GmbH. Take note of contraindications of the medicine in the Cayston® package leaflet.

4 Use of the nebulizer handset

The Altera® Nebulizer Handset must only be used in accordance with its intended purpose. For reasons of hygiene and to avoid cross-infections, the Altera® Nebulizer Handset must be used by one patient only.

- Before using for the first time, disassemble and clean the nebulizer handset including the aerosol head as described in the instructions for use of the eFlow®rapid nebuliser system.
- Assemble the nebulizer handset as shown in the diagram:
 - 1 Open the nebulizer handset.
 - 2 Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**
 - 3 Insert the inspiratory valve.
 - 4 Close the nebulizer handset.
 - 5 Attach the mouthpiece.
 - 6 Connect the connection cord.
 - 7 Fill with the medication.
 - 8 Close the medication reservoir.
 - 9 Sit in an upright position and relax. This makes inhalation easier and improves uptake of the medication in the airways.

Hold the nebulizer handset horizontally all the time while performing inhalation. If the nebulizer handset is tilted, this may result in the incorrect dosage being delivered.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.
- Always check the medication reservoir to ensure that the entire dose of Cayston® has been nebulized.

5 Cleaning and disinfection

The dismantled nebulizer handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®rapid nebuliser system).

6 Replacing the nebulizer handset

Each 28-day pack of Cayston® contains an Altera® Nebulizer Handset. Use the new nebulizer handset for each treatment cycle.

7 Package contents

One Altera® Nebulizer Handset with one Altera® Aerosol Head. There are no accessories for the Altera® Nebulizer Handset.

8 Symbols

	Medical device
	Follow the instructions for use
	Single patient – multiple use
	Lot number
	Unique Device Identifier
	Item number
	Serial number
	Minimum and maximum ambient temperature
	Minimum and maximum humidity
	Minimum and maximum air pressure

	CE marking: This product complies with the European requirements for medical devices (EU 2017/745).
	Do not dispose in domestic waste. This product must be disposed in accordance with the applicable (national, county or municipal) regulations.
	Caution
	Manufacturer
	Date of manufacture

9 Service and contact

In case of technical problems or questions about the device, please contact your service partner. The contact addresses of your responsible service partner as well as a digital version of the instructions for use of your Altera® Nebulizer Handset and performance characteristics determined for Cayston® can be found at:

www.pari.com/altera

We reserve the right to make technical changes. Information as of: 2024-09

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