



Altera® Nebulizer Handset

Specifically designed for use only with Cayston®

Nebulized medium	Cayston® (75 mg aztreonam lysine reconstituted in 1 mL diluent)
Fill volume	1 mL
MMAD ^{a)} (Mass Median Aerodynamic Diameter)	4.1 µm
GSD ^{a)} (Geometric Standard Deviation)	1.6
Respirable fraction ^{a)}	
< 5 µm	68.3 %
< 2 µm	4.5 %
2 - 5 µm	63.8 %
Aerosol output ^{b)}	0.62 mL
Aerosol output rate ^{b)}	0.27 mL/min

Aerosol data according to ISO 27427 related to Annex C and Annex D

a) Measurement with Next Generation Pharmaceutical Impactor (NGI) at 23 °C and 50% relative humidity. Inspiratory flow: 15 l/min

b) Measurement with breathing simulator at 23 °C and 50% relative humidity. 500 ml breathing volume, breathing frequency 15 cycles/minute, sinusoidal breathing pattern, inhalation/exhalation ratio 1:1 (with adults, may vary in children).



PARI Pharma GmbH
Moosstrasse 3 • 82319 Starnberg • Germany
E-Mail: info@paripharma.com • www.paripharma.com