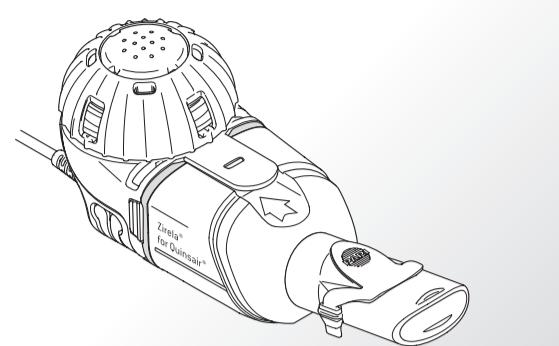
Zirela®

Nebulizer Handset


REF 678G8015

CE 0123

eFlow[®]
TECHNOLOGY

de Gebrauchsanweisung	fr Notice d'utilisation
en Instructions for use	it Istruzioni per l'uso
da Brugsanvisning	nl Gebruiksaanwijzing
es Istrucciones de uso	pt Manual de instruções
fi Käyttöohje	sv Bruksanvisning

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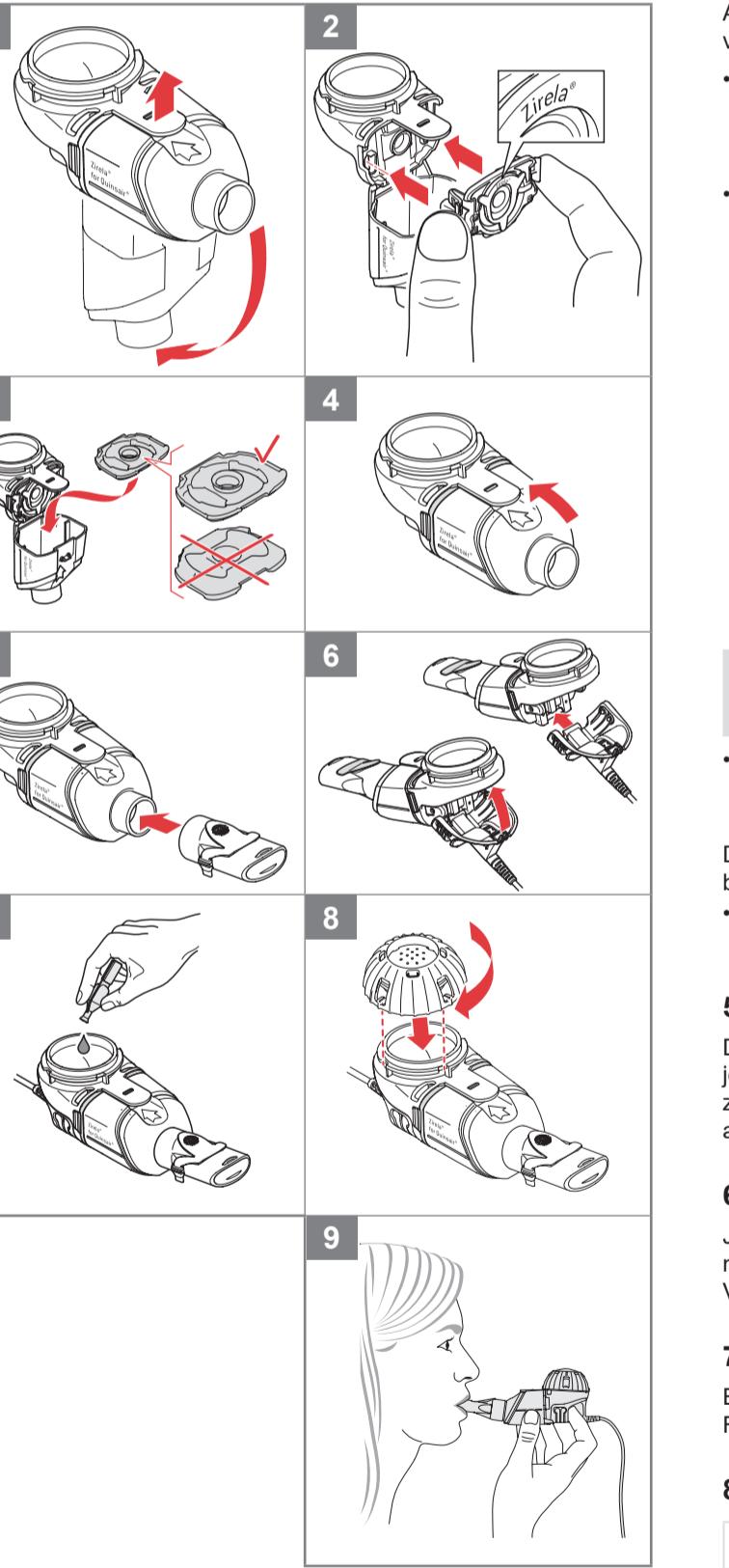en Instructions for use Zirela® Nebulizer Handset

- Read these instructions for use carefully and completely.
- Read the **instructions for use of the eFlow®/rapid nebuliser system** completely before using for the first time. These contain further important instructions for the use of the Zirela® Nebulizer Handset, particularly regarding connection, operation, cleaning and disinfection.

Report serious incidents to the manufacturer and the competent authority.

1 Intended purpose

The Zirela® Nebulizer Handset with an eBase® or eTrack® Controller is intended to administer Quinsair® as an aerosol for inhalation.

Attention! To ensure correct dosing, the Zirela® Nebulizer Handset must only be used with Quinsair®.

2 Indication

The Zirela® Nebulizer Handset is indicated to administer Quinsair® to treat lung infections caused by *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

3 Contraindication

For the Zirela® Nebulizer Handset no contraindication is known to PARI Pharma GmbH.

Take note of contraindications of the medicine in the Quinsair® package leaflet.

The inhalation should last about 5 minutes. Inhalation is beended, soobald kein Aerosol mehr erzeugt wird.

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The Zirela® Nebulizer Handset must only be used in accordance with its intended purpose. For reasons of hygiene and to avoid cross-infections, the Zirela® Nebulizer Handset must be used by one patient only.

- Before using for the first time, disassemble and clean the nebulizer handset including the aerosol head as described in the instructions for use of the eFlow®/rapid nebuliser system.

• Assemble the nebulizer handset as shown in the diagram:

- 1 Open the nebulizer handset.

- 2 Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**

- 3 Insert the inspiratory valve.

- 4 Close the nebulizer handset.

- 5 Attach the mouthpiece.

- 6 Connect the connection cord.

- 7 Fill with the medication; empty an ampoule completely.

- 8 Close the medication reservoir.

- 9 Sit in an upright position and relax.

This makes inhalation easier and improves uptake of the medication in the airways.

Attention! Hold the nebulizer handset horizontally all the time while performing inhalation. If the nebulizer handset is tilted, this may result in the incorrect dosage being delivered.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The inhalation session should last about 5 minutes. Inhalation is ended as soon as no more aerosol is generated.

- Always check the medication reservoir to ensure that the entire dose of Quinsair® has been nebulized.

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5 Cleaning and disinfection

The dismantled nebulizer handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®/rapid nebuliser system).

6 Replacing the nebulizer handset

Each 28-day pack of Quinsair® contains a Zirela® Nebulizer Handset. Use the new nebulizer handset for each treatment cycle.

7 Package contents

One Zirela® Nebulizer Handset with one Zirela® Aerosol Head. There are no accessories for the Zirela® Nebulizer Handset.

8 Symbols

	Medical device
	Follow the instructions for use
	Single patient – multiple use
	Lot number
	Unique Device Identifier
	Item number
	Serial number
	Minimum and maximum ambient temperature
	Minimum and maximum humidity
	Minimum and maximum air pressure
	CE marking: This product complies with the European requirements for medical devices (EU 2017/745).
	Do not dispose in domestic waste. This product must be disposed in accordance with the applicable (national, county or municipal) regulations.
	Caution
	Manufacturer
	Date of manufacture

9 Service and contact

In case of technical problems or questions about the device, please contact your service partner. The contact addresses of your responsible service partner as well as a digital version of the instructions for use of your Zirela® Nebulizer Handset and performance characteristics determined for Quinsair® can be found at:

www.pari.com/zirela

We reserve the right to make technical changes.
Information as of: 2024-09

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