FRANCAIS

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
© 2016 PARI Respiratory Equipment, Inc. DS-44-461 REV C 4-16

PARI PEP MONITOR

PART# 044B4612

FOR USE WITH PARI PEP SYSTEM

FOR SINGLE PATIENT USE (ONE PERSON) ONLY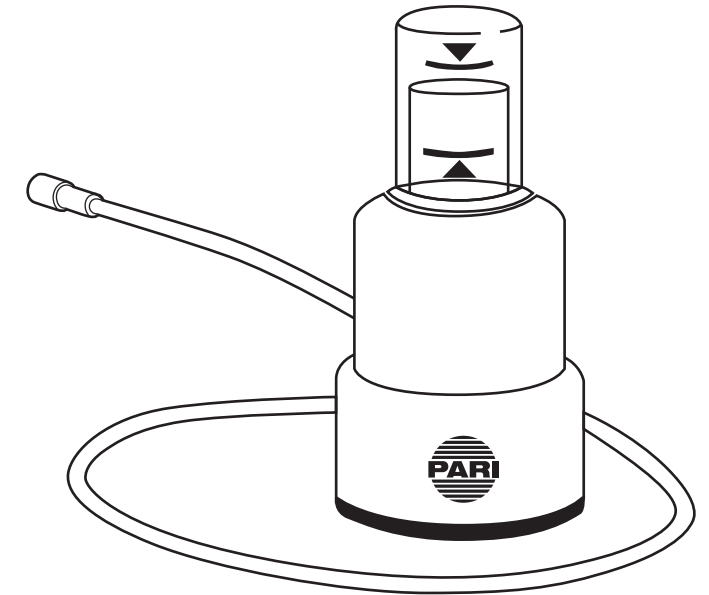

WARNING / ADVERTENCIA / AVERTISSEMENT:

US Federal Law restricts this device to sale by or on the order of a physician.
La Ley Federal de los Estados Unidos de Nortamérica (USA) permite la venta de este producto solo bajo prescripción médica.
La loi fédérale américaine exige que ce dispositif soit vendu sur recommandation ou prescription médicale.

FIG 1

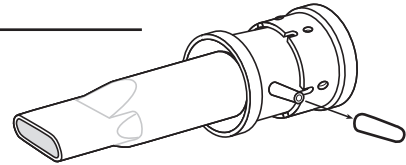
FIG 2


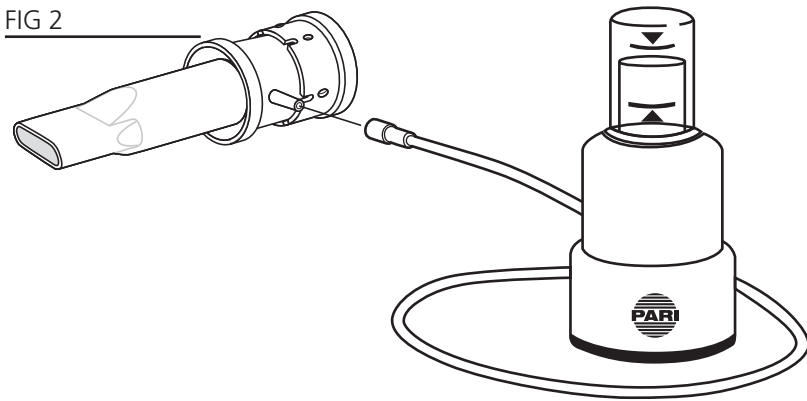
FIG 3

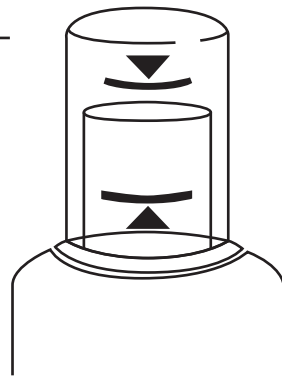ENGLISH

INSTRUCTIONS FOR USE

The enclosed monitor was designed to assist you in receiving optimal benefit from your PARI PEP device. It is important that you read and understand the "Instructions For Use" included with your PARI PEP device before attaching monitor.

I. ATTACHING MONITOR TO PARI PEP

1. Remove the manometer port cap from the PARI PEP (Fig. 1). Store cap in a safe location for future use.
2. Attach monitor tubing to the manometer port (Fig. 2). Push tubing on firmly to ensure a tight seal.
3. When exhaling through your PARI PEP and monitor, always keep the blue indicator top in the area between the black lines (Fig. 3). These lines indicate a pressure between 10-20 cm of water.

II(A). CLEANING INSTRUCTIONS FOR MONITOR

1. The interior of the base of the Pressure Indicator should be inspected for cleanliness periodically.
2. To inspect or rinse, remove the blue protective cap on the bottom.
3. Ensure interior is dry before reassembling.

⚠ WARNING

Do not submerge the entire monitor under water or it will affect accuracy. Protect the clear indicator section of the unit while cleaning. Inspect device on a routine basis to ensure cleanliness and function. If damaged, DO NOT USE. Discard and replace.

II(B). CLEANING INSTRUCTIONS FOR TUBING

1. Clean tubing inside and out, once a day in warm soapy water.
2. Rinse and drain all water out of tube.
3. Allow to air dry in a clean surface. Do not reuse tubing until completely dry.

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