# VORTEX®

## Non Electrostatic Holding Chamber

NOT MADE FROM BPA OR LATEX  
DISHWASHER SAFE

SAVE THESE INSTRUCTIONS

Vortex is indicated for use as a spacer/valved holding chamber for use in delivery of aerosol medication with metered dose inhalers.

### SINGLE PATIENT USE

#### ⚠ CAUTION

U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Use the product only for its intended use as described in these Instructions For Use. Improper use can result in serious or fatal illness/injury.

#### ⚠ WARNING

1. Close supervision is necessary when this product is used by children or near the physically or mentally impaired. This device contains small parts that may present a choking hazard to small children. Always keep out of the reach of children three years old and under. All parts should be seated firmly in place. Use of an improperly assembled PARI VORTEX® Non Electrostatic Valved Holding Chamber could prevent adequate delivery of medication. If not sure of the suitability and/or functioning of your MDI, consult MDI instructions and/or your physician.
2. To reduce the risk of infection, illness or injury from contamination and to reduce the risk of improper performance and treatment, you must thoroughly clean and dry all parts of the PARI VORTEX® as instructed on reverse side.
3. To reduce the risk of improper treatment, use only PARI accessories (e.g. masks).
4. To reduce the risk of improper treatment, product should be replaced if valves or parts are damaged or missing.
5. To reduce the risk of improper treatment, make sure duckbill valve is properly fitted.
6. If unsure of how to use this product, talk to your physician or pharmacist.
7. Administer medication per MDI instructions.

Note: MDI Receptacle is not designed to be removed. Attempting to remove may cause damage.

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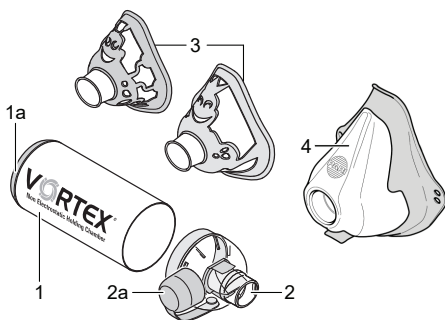
1. Non-Electrostatic Aluminum Chamber
- 1a. MDI Receptacle for metered dose inhaler
2. Mouthpiece with inspiratory and expiratory valve
- 2a. Protective cap
3. Ladybug Toddler or Frog Child Mask\*
4. Adult Mask\*

\*Available in certain configurations only

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**PARI VORTEX HOLDING CHAMBER**  
has a 6 month warranty for home use.

**PARI** Respiratory Equipment, Inc.

Tel: 1-800-FAST-NEB (327-8632) | 1-804-253-7274

www.pari.com | product.info@pari.com

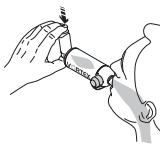
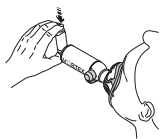
2412 PARI Way, Midlothian, VA 23112

© 2022 PARI Respiratory Equipment, Inc.  
051D7001 Rev C 12-22

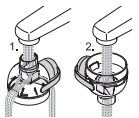
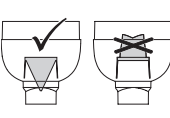
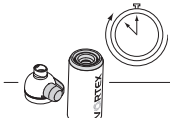
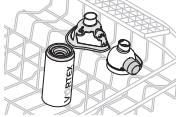
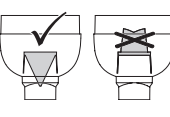
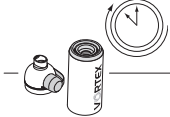
# INSTRUCTIONS FOR USE—PARI VORTEX® Non Electrostatic Valved Holding Chamber

**IMPORTANT:** Follow instructions included with your metered dose inhaler (MDI).

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**DISINFECTION:** To disinfect after cleaning, place all parts in boiling water for 5 minutes. Dry as instructed above.

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