# ONE-HANDED OPERATION AID

## INSTRUCTIONS FOR USE

Product is non-sterile, not made with BPA or latex.  
For use only with PARI Vortex® Non Electrostatic Holding Chamber.  
READ ALL INSTRUCTIONS BEFORE USE.


Vortex is indicated for use as a spacer/valved holding chamber for use in delivery of aerosol medication with metered dose inhalers.

### ASSEMBLING THE PARTS


#### Step 1:

- Push the one-handed operation aid onto the inhalation chamber from the back.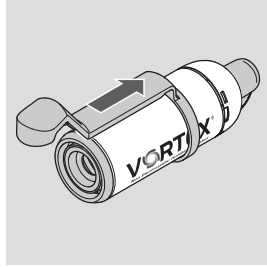


#### Step 4:

- Position the one-handed operation aid so that its round end encloses the metered dose inhaler. To do this, you may have to push the one-handed operation aid farther forwards or backwards.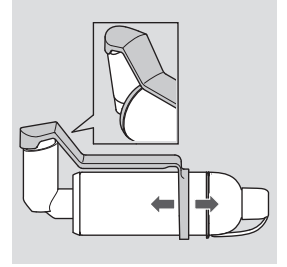

#### Step 2:


- Remove the cap on the metered dose inhaler.
- Shake the metered dose inhaler vigorously before every use.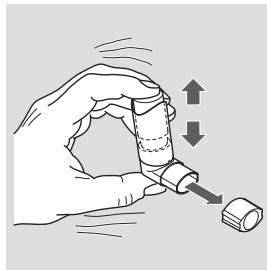

#### Step 5 (only if using a mask):

*Attach the mask (pediatric mask)*


- Take the protective cap off the mouthpiece of the VORTEX.
- Attach the mask to the VORTEX mouthpiece.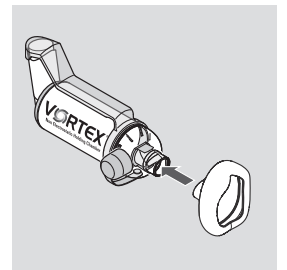

OR


#### Step 3:

- Insert the metered dose inhaler in the connecting ring on the inhalation chamber.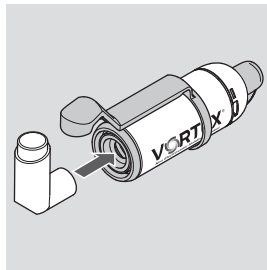

*Attach the mask (adult mask)*

- Take the protective cap off the mouthpiece of the VORTEX.
- Fit the mask connector into the opening of the PARI adult mask soft.
- Attach the mask to the mouthpiece of the VORTEX with the mask connector.

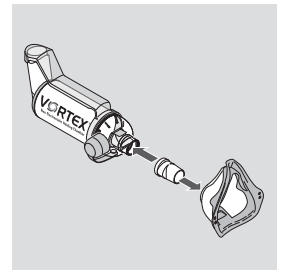### INSTRUCTIONS FOR CLEANING

| INSTRUCTIONS FOR CLEANING   |   |  |   |   |
|---|---|--|---|---|
| Washing by Hand   |   |  |   |   |
| Clean as instructed before first use. After first use, clean and disinfect once a week. | Remove the one handed operation aid from the chamber. | A. Soak part for 15 minutes in luke warm water with liquid detergent. Agitate gently then rinse in clear water.<br>OR<br>B. Place part in top shelf of dishwasher. | A. Rinse in clean water. Shake out in sink.<br>OR<br>B. Remove from dishwasher. Shake out excess water. | Thoroughly air dry on a clean, lint-free cloth or on a drying rack. When dry, store for future use. |

**DISINFECTION:** To disinfect after cleaning, place all parts in boiling water for 5 minutes. Dry as instructed above.

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