INSTRUCTIONS FOR USE

Nebulizer for single patient use only

INCLUDES INSTRUCTIONS FOR THE PARI LC® SPRINT NEBULIZER

IMPORTANT: READ INSTRUCTIONS AND SAFETY PRECAUTIONS BEFORE USE


PORTABLE AEROSOL SYSTEM

Instructional Video Available Online

Scan this code with your smartphone or tablet to view the Trek® S Instructional Video, or simply visit www.PARI.com/instructional-videos online.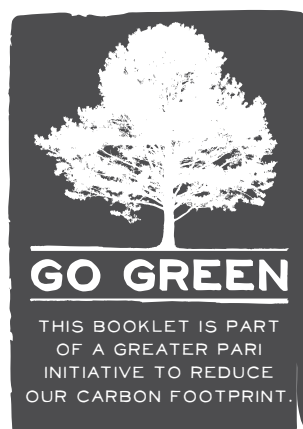

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE. USE THE PRODUCT ONLY FOR ITS INTENDED USE AS DESCRIBED IN THESE INSTRUCTIONS FOR USE. IMPROPER USE CAN RESULT IN SERIOUS OR FATAL ILLNESS/ INJURY, IMPROPER TREATMENT OR PROPERTY DAMAGE.

! REMEMBER
■ REPLACE YOUR REUSABLE NEBULIZER CUP EVERY SIX MONTHS

PORTABLE AEROSOL SYSTEM

Congratulations on your new PARI Trek® S Portable Aerosol System. Your doctor has prescribed the highest quality of delivery systems for your nebulizer medication(s) to treat your respiratory condition. Selected for use in more than 25 clinical trials for new respiratory medications, PARI LC® reusable nebulizers deliver consistent particle size and fast aerosol treatments for targeted delivery to the lungs.

Your compressor and nebulizer play an important role in treating your respiratory condition. Follow your physician's advice and prescribed treatment regimen closely to achieve the most effective therapeutic outcomes.

Read and understand the entire PARI Trek® S Instructions for Use before using. Take special note of all safety precautions marked CAUTION, WARNING, and DANGER.

*SAVE THESE INSTRUCTIONS
FOR FUTURE USE*

INDICATIONS FOR USE: LC® SPRINT

The LC® Sprint is a handheld nebulizer, designed to aerosolize medication approved for nebulization and prescribed by a physician. The LC® Sprint is intended for adult and pediatric patients consistent with the indications for the aerosol medication.

A. SAFETY PRECAUTIONS - LC® SPRINT

⚠ CAUTION

US Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication.

The LC® Sprint is a handheld nebulizer, designed to aerosolize medication approved for nebulization and prescribed by a physician. The LC® Sprint is intended for adult and pediatric patients consistent with the indications for the aerosol medication. Improper use can result in serious or fatal illness or injury. Nebulizer is not sterile.

⚠ WARNING

Use only when prescribed by a physician and with the prescribed medication. In order to achieve the specified droplet characteristics the compressor or central air supply must provide 3.5 - 8 lpm at a minimum of 20psi.

⚠ WARNING

1. To reduce the risk of increased bacterial growth, infection, illness, or injury from contamination: thoroughly clean and dry all parts of the nebulizer and thoroughly dry any moisture or condensation in the tubing at the end of every treatment following the enclosed instructions.
2. Cleaning the nebulizer after each treatment is not enough---you must regularly disinfect or sterilize the nebulizer between treatments to prevent serious or fatal illness caused by contamination of the nebulizer.
3. Never use a nebulizer with a clogged nozzle. If the nozzle is clogged, the aerosol mist will be reduced, altered or stopped, which will diminish or eliminate the effectiveness of the treatment. If clogging occurs, replace with the same model PARI Reusable Nebulizer.
4. Close supervision is necessary when this product is used by or near children and physically or mentally impaired. Never use when sleeping or drowsy.
5. This product contains small parts that may present a choking hazard to small children. The tubing also presents a strangulation hazard. ALWAYS USE CLOSE ADULT SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.
6. Patients under the age of 4 or any patients who have special needs should use a facemask instead of the valved mouthpiece.
7. Before use, check PARI Reusable Nebulizer for proper assembly. All parts should be seated firmly in place. Use of an improperly assembled nebulizer could diminish or prevent adequate delivery of medication and the effectiveness of the treatment.

INDICATIONS FOR USE: PARI TREK® S PORTABLE AEROSOL SYSTEM

The PARI Trek® S nebulizer compressor is a portable, DC powered air compressor intended to provide a source of compressed air for use with jet nebulizers. The Trek® S is intended for adult and pediatric patients. The device is nonsterile, prescription-use only, intended for use in hospital, clinic, or home environments.

B. SAFETY PRECAUTIONS - TREK® S PORTABLE AEROSOL SYSTEM

▲WARNING

Use only with nebulizers and tubing authorized by PARI for this device. Other nebulizers and tubing may result in improper treatment.

When using electrical and medical products, especially when children are present, basic safety precautions must be followed.

▲DANGER

To reduce the risk of serious or fatal injury from electrocution:

1. Do not place or store the product where it can fall or be knocked into bathtub, sink, water, or other liquid. Do not place or drop into water or other liquid. Do not use while bathing.
2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.

▲WARNING

To reduce the risk of serious or fatal injury from electrocution, fire or burns and to reduce the risk of damage and malfunction to the product:

1. Before use, check the serial number label on the bottom of the compressor, battery and on the Multi-Voltage AC Adapter to ensure that the voltage and current indicated on the unit correspond with the voltage and current available.
2. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been exposed to any liquids inside the case. Return the product to PARI for examination and repair.
3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or use extension cords.
4. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.
5. Never drop or insert any object into any openings.
6. Be sure the PARI TREK® S compressor and the Multi-Voltage AC Adapter are unplugged prior to cleaning.
7. Do not open or disassemble the PARI TREK® S Compressor. Refer all servicing to an authorized PARI Service Center.
8. Use only adapters that are authorized for the PARI TREK® S Compressor. Use of unapproved accessories can lead to improper treatment or damage to the compressor, adapters, or the battery.
9. Never spray liquids onto the compressor housing or battery. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluids enter the unit, return the product to PARI for examination and repair.
10. Do not cover up or use the PARI Trek® S compressor inside the carry case. Covering the device may cause excessive heat buildup in the compressor housing.

▲WARNING

1. The compressor has auto-resettable thermal protection. If the compressor shuts down due to temperature, it will restart automatically when it cools off if the switch is not set to the off position. Ensure the switch is set to the off position if the compressor shuts off due to temperature.
2. Always unplug immediately after using.
3. Do not use the PARI TREK® S Compressor or the PARI Reusable Nebulizer while operating a vehicle.
4. Portable and mobile RF communication devices can disrupt electrical medical equipment. The electrical field generated when the compressor is running can interfere with screens and monitors (flickering). For this reason, the compressor must be placed far enough away from screens/monitors to prevent such interference. Technical electromagnetic compatibility data is available in table form upon request from PARI Respiratory Equipment Inc./Patient Connection 1.800.FAST.NEB (327.8632), press 2.

BATTERY SAFETY PRECAUTIONS

▲WARNING

1. Use only the PARI TREK® S Rechargeable Battery described in these Instructions For Use for operating your PARI TREK® S Compressor.
2. Do not immerse the PARI TREK® S Rechargeable Battery in water.
3. Do not touch the battery with wet or moist hands.
4. Do not expose battery to direct sunlight, humidity, extreme temperatures or chemicals.
5. Make sure battery poles do not come into contact with keys or other metal objects, causing a "short circuit".
6. Do not attempt to open the plastic case or to replace any components. Do not disassemble Rechargeable Battery. The Rechargeable Battery is a sealed unit and can not be repaired.
7. Do not incinerate (burn).


▲CAUTION

1. To charge the PARI TREK® S Rechargeable Battery use only the PARI TREK® S Multi-Voltage AC Adapter or optional 12V DC Adapter. Using other adapters may damage the battery.
2. In case of failure in charging or operating, unplug the Rechargeable Battery from the Multi-Voltage AC Adapter and turn the unit off. Should the Rechargeable Battery fail to charge or operate properly, even if the Instructions For Use are followed, notify the Authorized Service Center by calling PARI Patient Connection toll free (USA) at 1.800.FAST.NEB (327.8632), then press 2.
3. Protect the environment: when replacing the Rechargeable Battery, recycle old battery or dispose of properly in accordance with local laws.

C. GETTING STARTED

Check your Trek® S package to ensure you have the following items.

It is important to have a second PARI Reusable Nebulizer on hand at all times.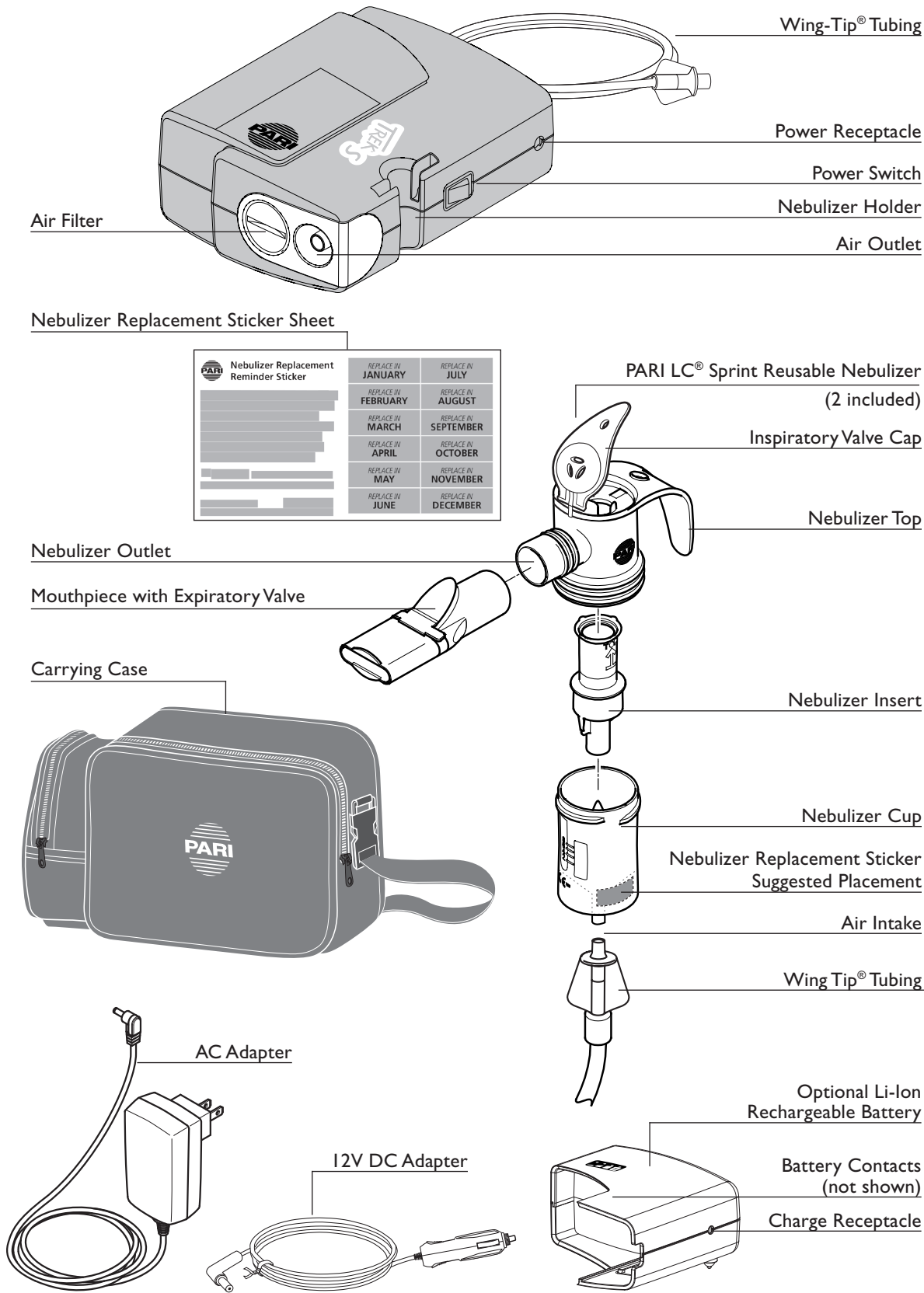

D. TREK® S SETUP

COMPRESSOR SETUP

Make sure that power switch is turned “OFF” ○.

With AC Adapter

1. Plug the AC adapter into a polarized wall outlet. (Fig. A)
2. Plug the other end of the power cord into the power receptacle on the side of the compressor or optional rechargeable battery. (Fig. C)

With 12V DC Adapter

NOTE: DO NOT USE WHILE OPERATING A VEHICLE

1. Plug the 12V DC adapter into the vehicle outlet. (Fig. B)
2. Plug the other end of the adapter into the power receptacle on the side of the compressor or optional rechargeable battery. (Fig. C)

With Optional Rechargeable Battery

The PARI Trek® S Li-Ion rechargeable battery is available as an option to help maximize your freedom with portable treatments. Speak with your physician and homecare company about your daily schedule's needs and if a rechargeable battery is a good fit for you.




1. Completely charge the battery for the first time before use. Refer to the “Li-Ion Rechargeable Battery” section on page 5.

2. The rechargeable battery cannot be connected directly to the PARI Trek® S if either the AC adapter or 12V DC adapter are plugged into the power receptacle. (Fig. D)
3. Attach the rechargeable battery to the PARI Trek® S by placing the battery onto the rear section of the compressor and sliding it into place. Ensure battery is securely locked onto the compressor prior to operation. (Fig. D)
4. Use the indicator lights to determine status of the battery. Refer to “Li-Ion Rechargeable Battery” section on page 5.

When the battery is completely discharged all indicator lights will extinguish and the compressor will shut off.

NOTE: If battery fails to operate, connect the AC adapter into the wall outlet. Plug the other end of the AC adapter into the charge receptacle on the side of the rechargeable battery. The charge indicator should turn “ON” indicating that the battery has re-set and is now charging. If the charge indicator does not appear, or if all four indicators are blinking, the battery has failed and should not be used. Please contact PARI Patient Connection at 1.800.FAST.NEB (327.8632)

5. To remove the battery, push and hold the release button located on the underside of the battery. Slide the battery away from the compressor. (Fig. E)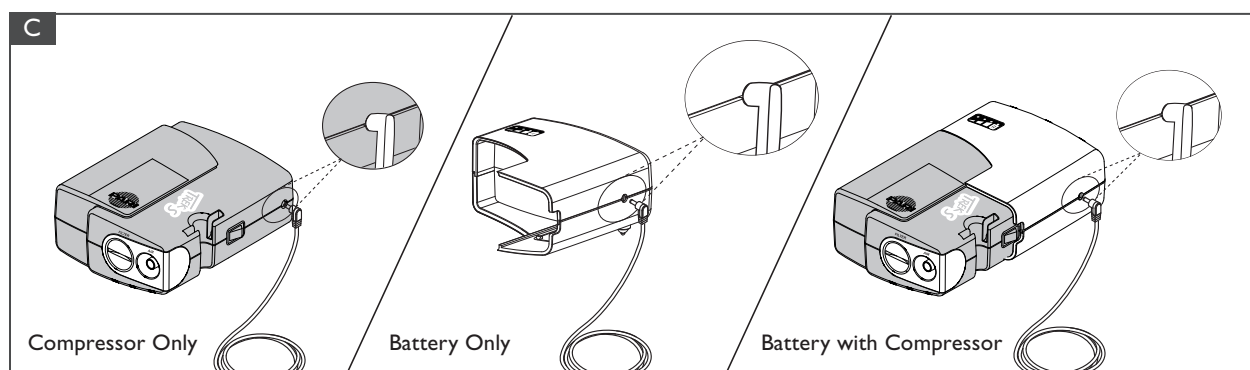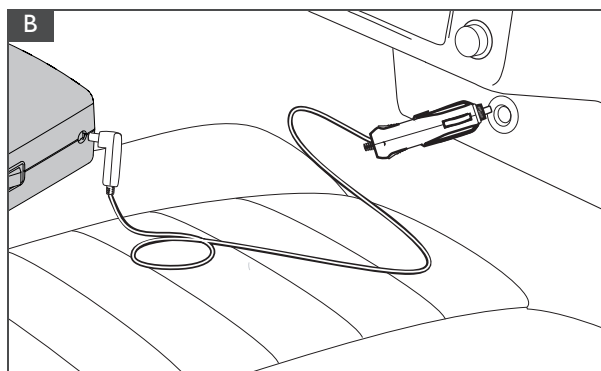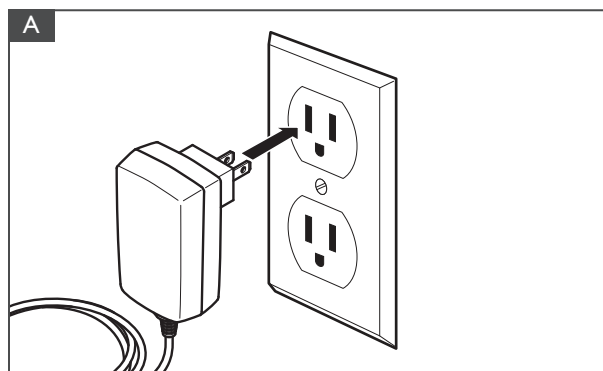

D. TREK® S SETUP (CONTINUED)

LI-ION RECHARGEABLE BATTERY CHARGING

Refer to the below diagram.

- 1. Completely charge the battery before using for the first time for more than two hours or until all three battery level indicators are lit, and the charge indicator turns off.
- 2. To charge the battery, connect the AC adapter into a polarized wall outlet (Fig. A). Plug the other end of the AC adapter into the rechargeable battery charge receptacle (Fig. C). The charge indicator will turn "ON".
- 3. When the rechargeable battery is fully charged, the charge indicator will turn "OFF". Unplug the multi-voltage AC adapter from the rechargeable battery charge receptacle. The amount of time it takes to recharge the battery will depend on the age and charge level of the battery.




NOTE: Do not leave the multi-voltage AC Adapter plugged into the rechargeable battery for more than 7 days. Leaving the battery connected to the power adapter for long periods of time may decrease the life of the battery.

When not regularly using the battery, the battery should be fully charged every six months to optimize and preserve the life of the rechargeable battery.

NOTE: If the battery goes for a long period of time without charging, battery life will be decreased.

Storing

The battery should be stored fully charged in a cool dry place.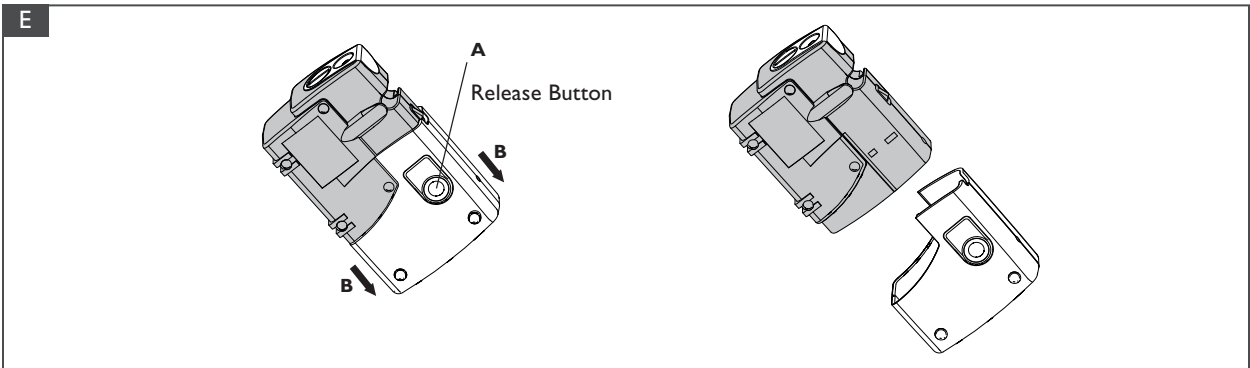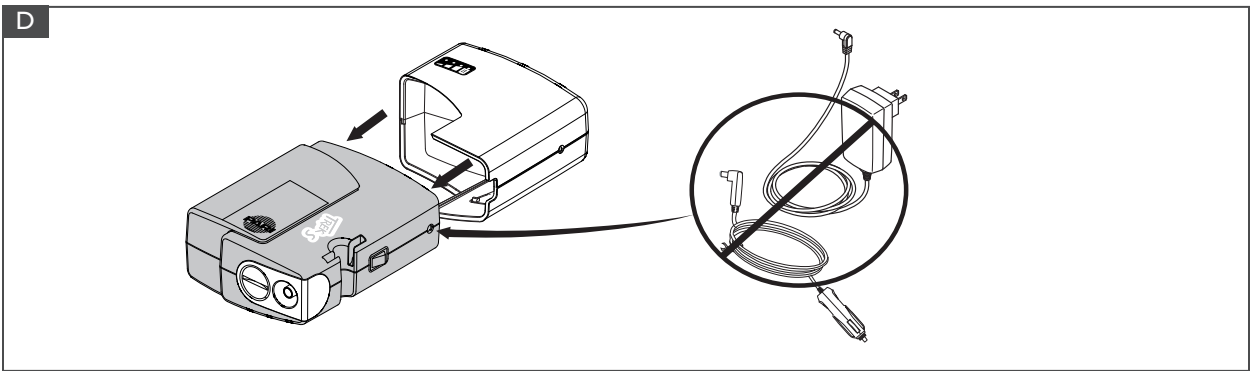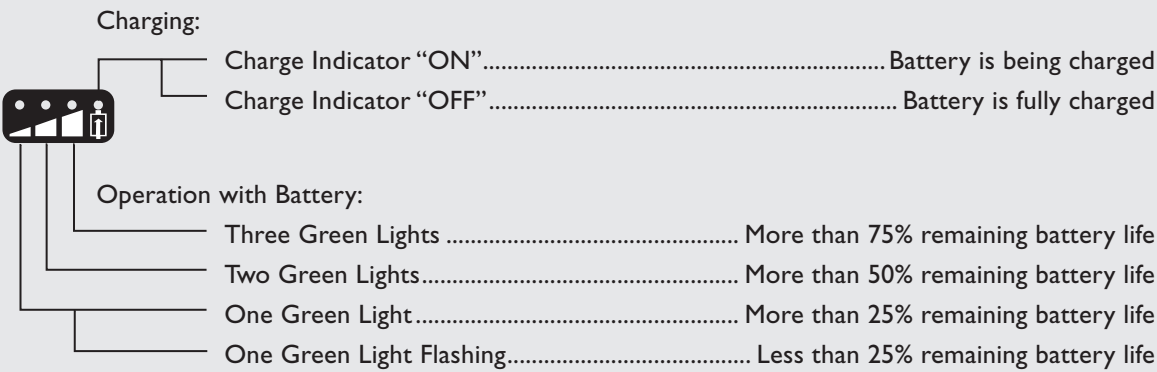

D. TREK® S SETUP (CONTINUED)




You are 10 Steps away from starting your treatment with Trek® S. Please follow step by step.

1. **Clean/Disinfect Nebulizer Cup:** Refer to Sections E-F to properly clean and/or disinfect your nebulizer prior to first use. Re-assemble nebulizer.
2. **Set-up Compressor:** Place the compressor on a flat, stable surface free of dust. Do not operate the compressor while it is in the carry case or covered, as this can cause excessive heat buildup in the compressor housing. Avoid cushioned, upholstered or carpeted surfaces. Make sure power switch is turned "OFF" ○.
3. **Connect Tubing To Compressor:** Insert one end of Wing Tip® tubing to the air outlet on the compressor. It should fit snugly (Fig. F).
4. **Load Medication:** Open the inspiratory valve cap of nebulizer by lifting the cap (Fig. G). Pour the prescribed dose of medication into nebulizer cup. Never fill past the max. fill line of 8 ml.
5. **Attach Mouthpiece or Mask:** If using the mouthpiece, attach the mouthpiece to the nebulizer outlet with the expiratory valve facing up. Then close the inspiratory valve cap by pressing firmly on the nebulizer top. If using a mask, refer to the mask instructions for use.
6. **Attach Tubing To Nebulizer Cup:** Firmly press the other end of the Wing Tip® tubing to bottom of PARI reusable nebulizer (Fig. H).
7. **Starting Treatment:** Sit in a relaxed, upright position and turn the compressor "ON" —.

8. **Nebulizer Position:** Place the mouthpiece on top of tongue and close lips around it. Take slow, deep breaths. As you inhale, the inhalation valve opens.
9. **Breathing:** For mouthpiece, inhale and exhale slowly through your mouth. As you exhale, the valve on the mouthpiece opens to allow the exhaled mist to escape. For mask, place mask over nose and mouth and inhale and exhale slowly.
10. **Complete Treatment:** Continue until the medication is gone or you hear a slight "sputtering" sound. Turn compressor "OFF" ○.

NOTE: The PARI TREK® S Compressor is protected by an automatic resettable thermal switch. If the PARI TREK® S Compressor has reached its temperature limit, it will automatically turn off to prevent damage to the compressor. In the unlikely event this occurs, switch off the PARI Trek® S Compressor, ensure it is on a flat, stable surface and is not covered, and allow the compressor to cool down. If the PARI Trek® S Compressor is not manually switched off, it will automatically restart when the compressor cools.

The PARI TREK® S Compressor may need approximately 30 minutes to cool down prior to resuming the treatment. If the PARI TREK® S Compressor does not start after it is switched on after cooling for 30 minutes, please switch it off, allow another few minutes for the compressor to cool, and try again. If you need to use the PARI Trek® S Compressor for longer than 30 minutes, please contact PARI at 1.800.FAST.NEB (327.8632).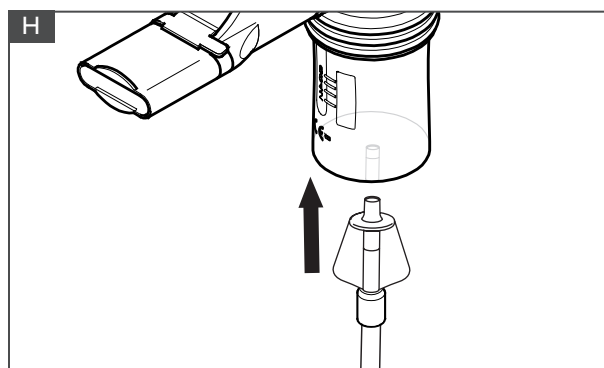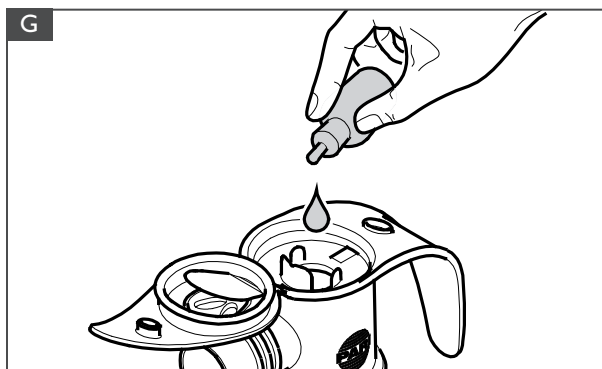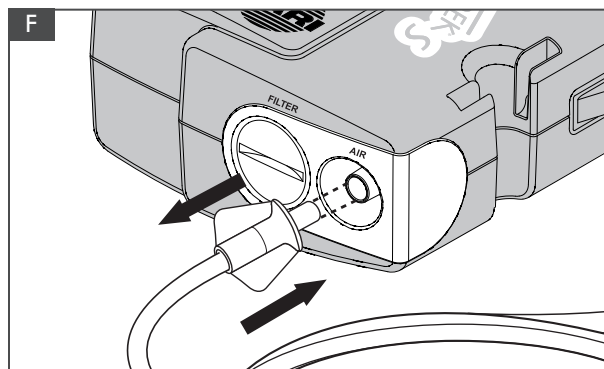

E. CLEANING YOUR PARI REUSABLE NEBULIZER

1. Disconnect the tubing from the air intake on the bottom of the nebulizer cup.
2. If there is any moisture (condensation) in the tubing, let the compressor run with only the tubing attached for 2-3 minutes. The warm air from the compressor will dry out the tubing. Alternatively, you can dry the tubing by removing it from the compressor and hanging it up with the "ends down" to allow any moisture (condensation) to drain out and evaporate. Use a clean damp cloth to wipe the exterior of the tubing.
3. Disassemble reusable nebulizer parts as illustrated on page 3.
4. Wash all parts with warm water and liquid dish soap. **DO NOT WASH THE TUBING.**
5. Rinse the reusable nebulizer parts thoroughly with warm water and shake out water.
6. Air dry or hand dry nebulizer parts on a clean, lint-free cloth. Reassemble nebulizer when parts are dry and store.
7. You can also wash all parts of the reusable nebulizer on the top shelf of a dishwasher. **DO NOT WASH TUBING.** Place the reusable nebulizer parts in a dishwasher basket, then place the basket on the top rack of the dishwasher. Remove and dry the parts when the dishwasher cycle is complete.

⚠ WARNING

TO REDUCE RISK OF INFECTION:

1. Nebulizer for single patient use (one person) only. Do not share your nebulizer with other patients.
2. You must regularly disinfect or sterilize the nebulizer between treatments. Failure to do so could lead to serious or fatal illness.

! REMEMBER

REPLACE YOUR REUSABLE NEBULIZER CUP EVERY SIX MONTHS

It is very important to replace your PARI reusable nebulizer every 6 months for the most effective treatments.

Advantages of Replacing Your Nebulizer Cup:


- Consistent medication delivery to the lungs
- Fast and efficient treatments
- Bacterial contamination can be reduced*

Included in this configuration is a reminder sticker. Place the sticker that corresponds to six months from date of first use to the smooth surface on the bottom of your nebulizer cup as indicated in this picture. Sticker is dishwasher safe.


Allow two hours after label is applied before cleaning.

WARNING: The sticker may present a choking hazard to small children-adult supervision is required.

* Lester, Mary K. Nebulizer Use and Maintenance by Cystic Fibrosis Patients: A Survey Study. Respiratory Care 2004; 49(12): 1504-1508.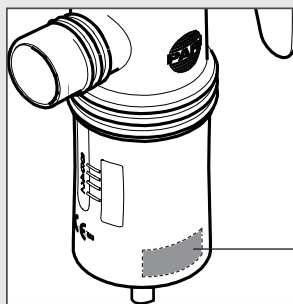

*Nebulizer Replacement Sticker
Suggested Placement*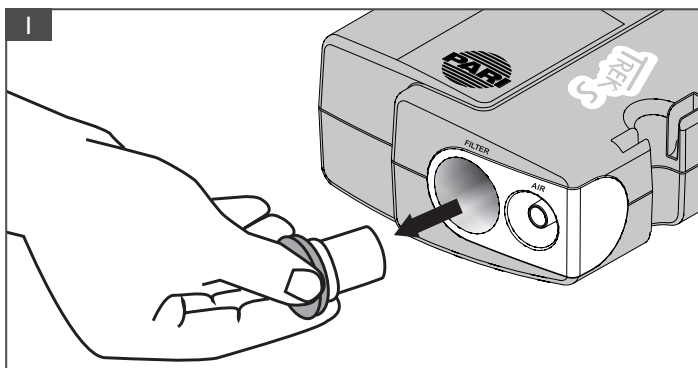

F. DISINFECTING YOUR PARI REUSABLE NEBULIZER

1. Clean the nebulizer as described in Section E. For low-level disinfection every other treatment day, soak all parts of the PARI Reusable Nebulizer (except tubing and mask) in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. You can substitute respiratory equipment disinfectant (like Control III®) for distilled white vinegar (follow manufacturer's instructions for mixing).
2. Rinse all parts of the PARI Reusable Nebulizer thoroughly with warm tap water and dry with a clean, lint-free cloth. Discard the vinegar solution when disinfection is complete.
3. The PARI Reusable Nebulizer parts (except tubing and mask) may also be disinfected by boiling them in a clean pot of water for a full 5 minutes.
4. Hospitals and clinics can sterilize the PARI Reusable Nebulizer parts (except mask and tubing) by autoclaving (a maximum of 277° F or 136° C), follow autoclave manufacturer's instructions.
5. Dry parts on a clean, lint-free cloth.

CONTROL III® is a registered trademark of Maril Products, Inc.

NOTE: FOR PATIENTS WITH CYSTIC FIBROSIS, THE CF FOUNDATION RECOMMENDS BOILING FOR A FULL 5 MINUTES TO DISINFECT.

G. MAINTENANCE OF YOUR TREK® S NEBULIZER SYSTEM

FILTER

Your Trek® S air filter is bright white when new. With use, the filter will slowly turn grey, indicating the need for replacement. The filter must be replaced immediately if it gets wet or becomes clogged. A good rule to follow is to replace your filter every 6 months – at the same time as your PARI reusable nebulizer.

Removing: First make sure the Trek® S power switch is turned off. Using a coin, remove the air filter cover by turning it counter-clockwise. Pull the cap and filter out, then remove the filter from the cap (Fig. I-J). Take a new filter and press into cap with the open end visible (Fig. J-K).

NOTE: There are two types of filters that fit the Trek® S compressor. If your filter is open only on one end as shown in Figure J, follow instructions as shown in Figure J and K. If your filter is open on both ends, press either end of the filter into the cap.

Place the cap and filter back into the front of your Trek® S. Use the coin to tighten by turning the cover clockwise.

NOTE: Do not attempt to wash and reuse Air Filters. Do not use substitute filter material; compressor damage may result.



COMPRESSOR & RECHARGEABLE BATTERY

Use a clean, damp cloth to wipe the exterior of the compressor housing. Be careful not to get liquids onto the Power Receptacle or the battery contact since damage could occur.

MAINTENANCE & REPAIR

Never operate your PARI Trek® S if it is improperly or incompletely assembled or damaged. Use only original PARI parts and attachments authorized by PARI for the PARI Trek® S Portable Aerosol System. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time.

NOTE: There are no serviceable parts inside the PARI Trek® S Portable Aerosol System. Service by anyone voids warranty.

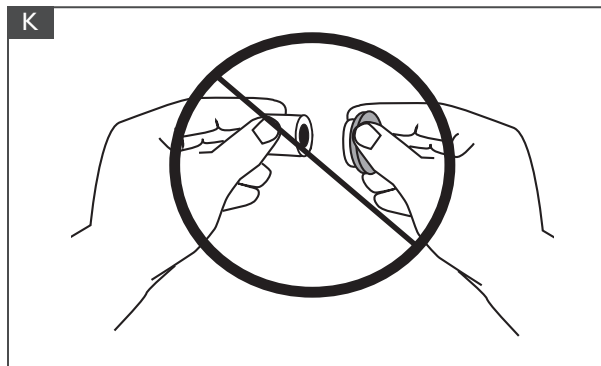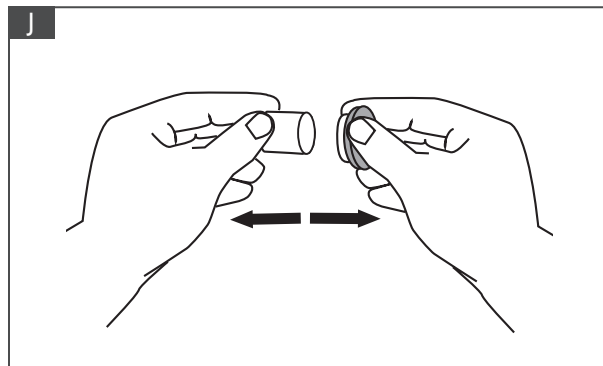H. SPECIFICATIONS

PARI TREK® S PORTABLE AEROSOL SYSTEM SPECIFICATIONS

Weight: (Compressor only)	0.8 lb
With Rechargeable Battery:	1.2 lbs
Dimensions: (Compressor only)	4.5" x 3.5" x 1.75"
Certification:	IEC/EN 60601-I and IEC/EN 60601-1-2

COMPRESSOR PERFORMANCE:

Maximum Pressure of Compressor:	35 PSI (2.4 Bar)
Maximum Liter Flow of Compressor:	8.5 LPM
Operating Pressure of Compressor ¹ :	14.5 PSI (1.0 Bar)
Operating Liter Flow of Compressor ¹ :	4.0 LPM

¹Measured with PARI LC® Sprint or LC® Plus Reusable Nebulizer attached.

AMBIENT OPERATION:

Temperature Range:	50°F to 104°F
--------------------	---------------

TRANSPORT/STORAGE:

Ambient Temperature:	13°F to 158°F
Humidity:	10% to 95%
Atmospheric Pressure:	500 hPa - 1060 hPa
Compressor Electrical Requirements:	Input 12VDC 1.67 A 18 watts

OPTIONAL RECHARGEABLE BATTERY:

Battery Type:	Lithium Ion (Li-Ion)
Operating time with battery:	Approximately 40 minutes
Number of treatments per charge ² :	Approximately 8 treatments
Normal time to charge:	Approximately 2 hours

²Using a PARI Reusable Nebulizer to nebulize 2.5 ml of medication.

ELECTRICAL REQUIREMENTS:

Input:	12VDC 1.67 A
Output:	12VDC 1.67 A

MULTI-VOLTAGE AC ADAPTER:

FUHUA Model # UE24WU-120167SPA or equivalent

Input Voltage:	100-240 VAC 50/60Hz 600mA
Output Voltage:	12V DC 1.67A

EXPLANATION OF SYMBOLS

Attention : Read all 'Instructions for Use' before operation:	
Insulation enclosed Class II equipment:	
Type "B" Protection from electrical shock:	
Do not dispose of in unsorted municipal trash. Please consult local laws and regulations for proper disposal:	
Power Switch:	
'ON' Position (compressor running):	—
'OFF' Position (compressor is not running):	○

NOTE: When switch is in the 'off' position and the compressor is not running, there will still be electrical potential inside the compressor housing. To remove power from the compressor, the power plug or battery must be removed.

I. WARRANTY & RETURN INFORMATION

PARI LC® SPRINT REUSABLE NEBULIZER LIMITED WARRANTY

PARI reusable nebulizers include a 6 month limited warranty for home care use. If you have any problems and need to return your PARI reusable nebulizer, contact PARI Patient Connection at 1.800.FAST.NEB (327.8632).

Please be sure to clean and disinfect your PARI reusable nebulizer and place it in a plastic bag before returning.

PARI TREK® S PORTABLE AEROSOL SYSTEM LIMITED WARRANTY

PARI offers a standard three (3) year warranty on the PARI TREK® S Compressor and six (6) months on accessories. The warranty period will be printed on the serial number label located on the bottom of the compressor. PARI warrants that the PARI TREK® S Compressor and its parts will be free from defects in workmanship and materials for a period of thirty-six (36) months from the date of first retail purchase of the compressor and six (6) months on the PARI Reusable Nebulizer and Li-Ion Rechargeable Battery. This warranty does not cover failures of the tubing or accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service; parts or cleaning solutions, or failure to comply with Instructions For Use. If the warranted equipment should fail during the warranty period, PARI will replace the equipment or parts at issue, provided the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of replacement of the warranted items only. To make a claim under this warranty, contact PARI Patient Connection at: 1.800.FAST.NEB (327.8632), Press 2.

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PARI TREK® S COMPRESSOR AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THIRTY-SIX (36) MONTHS FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. The sole and exclusive remedy under this warranty is to replace the PARI TREK® S Compressor and/or its parts by PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other rights which vary from State to State.

RETURN PROCEDURE

1. Contact PARI Patient Connection at 1.800.FAST.NEB (327.8632), Press 2 for an authorized service center.
2. In writing, describe the malfunction or symptoms of the PARI Trek® S. Include your full name, address and daytime telephone number.
3. Send written description and PARI Trek® S (please clean as described in Section 4) in a protective box to the address provided by Patient Connection.

Online at pari.com/register

Call 1.800.FAST.NEB (327.8632)

Fill out the pre-addressed, postage paid registration card and drop it in any USPS Mailbox.

Failure to register your PARI Trek® S does not affect your standard product limited warranty.

If you have any problems and need to return your PARI Trek® S, contact PARI Patient Connection at 1.800.FAST.NEB (327.8632).

Manufactured for:

PARI Respiratory Equipment, Inc.

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 800.727.4112 | Product.Info@PARI.com

Online at: www.PARI.com

© 2018 PARI Respiratory Equipment, Inc.
047D0206 Rev B 04/18

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