

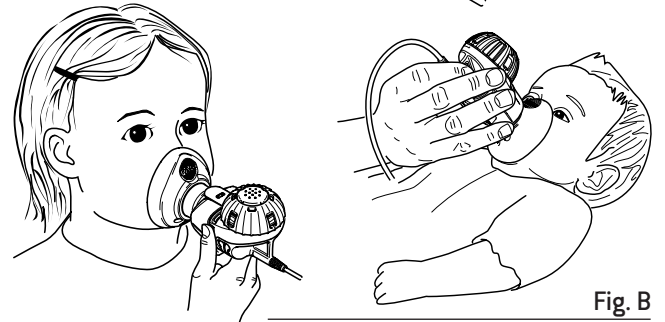


PARI SMARTMASK® Baby


For use only with
eFlow® Technology Devices

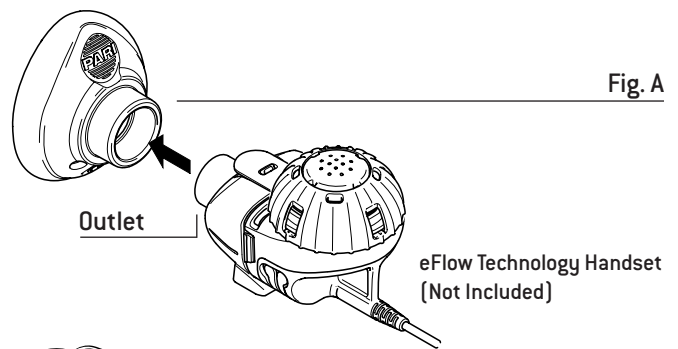

Part No. 078F5024

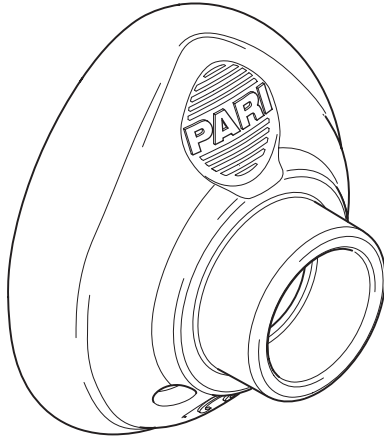

PATENT NO:
7,255,106B2

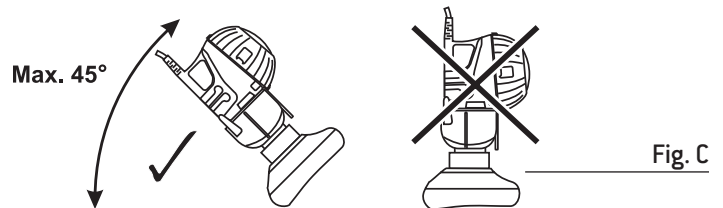

⚠ WARNING

1. Read these Instructions For Use before using this mask with an eFlow Technology Handset. Keep these Instructions For Use for future reference.
2. The PARI SMARTMASK® (SMARTMASK) Baby must always be used as a component of, or an accessory to the eFlow Technology Nebulizer. Accordingly, the Instructions For Use for this product must also be followed. In particular, all cleaning and disinfecting instructions for the eFlow Technology Handset also apply for the SMARTMASK Baby.

Intended use

The SMARTMASK Baby is a mask inhalation accessory for children weighing about 6 lbs and more, up to the age of two.

⚠ CAUTION

For optimum function, the mask must cover both corners of the mouth and the nose completely.

Using the mask

Figure A: Place the mask on the connector to the aerosol chamber of the eFlow Nebulizer.

Figure B: Put the mask on so that it forms an airtight seal over the mouth and nose.

Figure C: Do not tilt the eFlow nebulizer more than about 45°, because this will prevent the medication from being completely nebulized.

⚠ CAUTION

1. The SMARTMASK Baby must not be used with infants who are lying down or asleep.
2. To avoid aerosol losses, make sure that the mask seals tightly against the face.

This product can be used only with the following devices:

1. TRIO® Electronic Nebulizer
2. ALTERA® Nebulizer System
3. eRapid® Nebulizer System
4. Zirela® Nebulizer
5. Investigational eFlow® Nebulizer System

Manufactured In Germany for:

PARI Respiratory Equipment, Inc.
2412 PARI Way
Midlothian, Virginia 23112 USA

Customer Support:

1-866-335-6943
www.pari.com