

PARI LC[®] D

DISPOSABLE NEBULIZER

FOR SINGLE PATIENT USE (ONE PERSON) ONLY

NEBULIZER IS NOT STERILE

NOT INTENDED FOR REPROCESSING



CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all the warnings and instructions before use. Improper use can result in serious or fatal illness or injury.

WARNING: This product is DISPOSABLE and is not intended for prolonged or extended use.


- This product should be replaced at least every 15 treatments, or 15 days, whichever is first.
- This product is not heat resistant. It will deform if cleaned in the dishwasher, boiled, autoclaved or cold sterilized.
- Deformation and/or prolonged use can cause inadequate output, resulting in improper treatments.
- The tubing presents a strangulation hazard. ALWAYS USE CLOSE SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.

INDICATIONS: The PARI LC[®] D Disposable Nebulizer is used to deliver aerosolized medication to the lungs, with patients for whom a physician has prescribed medication for aerosol treatments in a home, doctor's office, hospital, or clinic environment.


I. GETTING STARTED

Before use, clean all parts (except tubing). See section II for cleaning instructions. To achieve the specified droplet characteristics, a liter flow of 3.5-8 liters per minute must be selected using wall air, wall oxygen or compressor (i.e. PARI Vios[®] or PARI Trek[®] S compressor systems).

1. Connect one end of tubing to air source.
2. Open top of nebulizer by twisting counter-clockwise, then lift up (Fig.1).
3. Pour prescribed medication into bottom of nebulizer. Maximum fill volume is 6ml.
4. Replace top of nebulizer. Semi-circle should face forward.
5. Attach mouthpiece or mask to nebulizer outlet (Fig.2).
6. Connect free end of tubing to bottom of nebulizer.

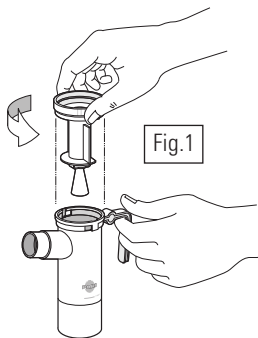

7. Turn on air source when ready to begin treatment.
8. Close lips around mouthpiece and inhale and exhale slowly.
For mask, place the mask over nose and mouth and inhale and exhale slowly.
9. Repeat step 8 until medication is gone.

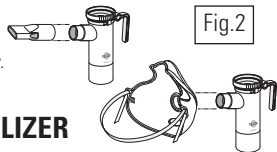

II. CLEANING THE PARI LC[®] D DISPOSABLE NEBULIZER

WARNING:

1. To reduce the risk of infection, illness, or injury from contamination, you must thoroughly clean all parts of the nebulizer as instructed after each treatment.
 2. Never use a nebulizer with a clogged nozzle. A clogged nozzle can cause inadequate or no output or aerosol mist, resulting in improper treatment.
-
1. Remove tubing from nebulizer and disassemble nebulizer parts.
 2. Wash all parts (except tubing) with warm water and liquid dish soap.
 3. Rinse thoroughly with warm water and shake out water.
 4. Air dry or hand dry nebulizer parts with a clean, lint-free cloth. Reassemble PARI LC[®] D when completely dry.

III. DISINFECTING THE PARI LC[®] D DISPOSABLE NEBULIZER

WARNING:

 To reduce the risk of infection and illness:

1. Nebulizer for single patient use (one person) only. Do not share your nebulizer with other patients.
 2. You must regularly disinfect the nebulizer between treatments. Failure to do so could lead to serious or fatal illness.
-
1. Clean the nebulizer as described in section II. For low-level disinfection every other treatment day, soak all parts of the PARI LC[®] D nebulizer (except tubing) in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. You can substitute respiratory equipment disinfectants (like Control[®] III) for distilled white vinegar (follow manufacturer's instructions for mixing). Rinse all parts of the nebulizer thoroughly with warm tap water and dry with a lint-free cloth. Discard the vinegar solution when disinfection is complete.

Control[®] III is a registered trademark of Maril Products, Inc.



Manufactured for:

PARI Respiratory Equipment, Inc.

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 804.253.0246 | Product.Info@PARI.com

www.PARI.com