# PARI SMARTTOUCH™ PEDIATRIC MASKS

044F5110 TODDLER LADYBUG MASK

044F5210 CHILD FROG MASK

## INSTRUCTIONS FOR USE

Single patient use (one person) only.

Product is non-sterile, BPA and latex-free.

For use only with PARI Vortex® Non Electrostatic Holding Chamber.

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE.

## ILLUSTRATIONS:

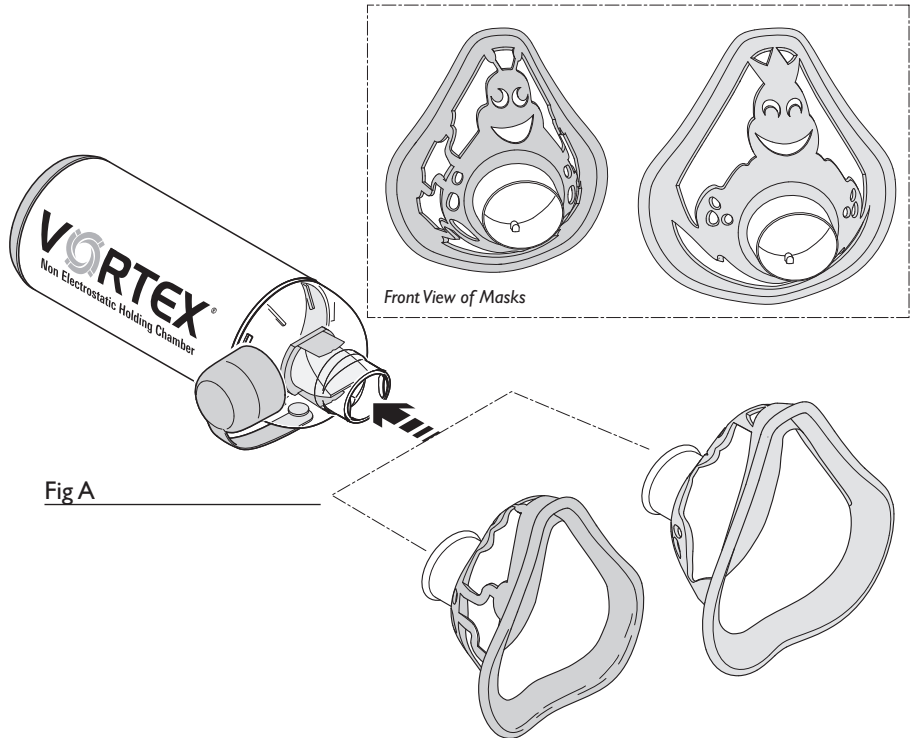
Fig A

Manufactured For:

**PARI Respiratory Equipment, Inc.**

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 804.253.0246 | Product.Info@PARI.com

www.**PARI**.com

# PARI SMARTTOUCH™ PEDIATRIC MASKS

044F5110 TODDLER LADYBUG MASK

044F5210 CHILD FROG MASK

## INSTRUCTIONS FOR USE

Single patient use (one person) only.

Product is non-sterile, BPA and latex-free.

For use only with PARI Vortex® Non Electrostatic Holding Chamber.

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE.

### **⚠ WARNING**

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE. IMPROPER USE CAN RESULT IN SERIOUS OR FATAL ILLNESS OR INJURY. THESE FACE MASKS SHOULD ONLY BE USED WITH THE VORTEX® NON ELECTROSTATIC VALVED HOLDING CHAMBER. THE VORTEX® FACE MASKS SHOULD NOT BE USED WITH ANY OTHER PARI PRODUCTS SINCE THIS COULD CAUSE IMPROPER TREATMENT OR A SUFFOCATION HAZARD.

PLEASE FOLLOW STEP-BY-STEP AND REFER TO ILLUSTRATIONS ON PAGE 1.

1. **Setup:** Firmly attach your mask to the mouthpiece of your Vortex® Non Electrostatic Holding Chamber (Fig A).
2. **Taking a Treatment:** Refer to your Vortex® Non Electrostatic Holding Chamber Instructions for Use for proper Vortex® setup and operation.
3. **Cleaning:** Clean mask one of these easy ways:
  - a. Wash with warm water and liquid dish soap. Allow to dry, then store.
  - b. Place in a dishwasher basket and place on the top rack of a dishwasher. Remove and store when cycle is complete.
4. **Disinfecting:** Clean as described in number 3 above.

For low-level disinfection every other treatment day, soak mask in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. You can substitute respiratory equipment disinfectant (like Control III®) for distilled white vinegar (follow manufacturer's instructions for mixing). Rinse mask thoroughly with warm tap water and allow to dry before storing.

The mask can be disinfected by boiling in a clean pot of water for a full 5 minutes. Hospitals and clinics can sterilize by autoclaving (a maximum of 277° F or 136° C), follow autoclave manufacturer's instructions.

Dry on a clean, lint-free cloth, then store.

Control III® is a registered trademark of Maril Products, Inc.

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Control III® es una marca comercial registrada de Maril Products, Inc.