BUBBLES THE FISH™ II PEDIATRIC AEROSOL MASK

INSTRUCTIONS FOR USE

For single patient use (one person) only. For use with PARI Nebulizers.

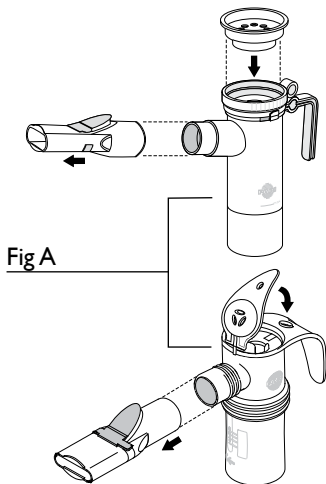
Fig A

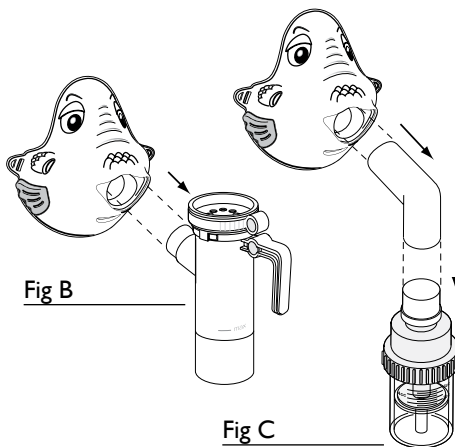
Fig B

Fig C

⚠️ WARNING

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE. IMPROPER USE CAN RESULT IN SERIOUS OR FATAL ILLNESS OR INJURY.

The enclosed elbow adapter should be used only with a "T" piece nebulizer and the Bubbles the Fish™ II pediatric aerosol mask. Do not use elbow adapter with a PARI nebulizer or any other masks

PARI's Bubbles the Fish™ II pediatric aerosol mask is designed with Pro-Vent™ technology. This optimizes inhalation of respirable particles to the lungs while dramatically reducing facial & ocular deposition, resulting in safer treatments.

PLEASE FOLLOW STEP-BY-STEP AND REFER TO ILLUSTRATIONS ABOVE.

1. **With PARI Nebulizer:** Remove the mouthpiece from the nebulizer and ensure inspiratory valve cap is attached (Fig A). Then press mask firmly onto the nebulizer outlet (Fig B).
2. **With Traditional "T" nebulizer:** Attach included elbow adapter to nebulizer; then attach mask firmly to the elbow piece (Fig C).
3. **Elastic Strap:** Your mask includes an elastic strap which can be used for increased stability during treatments. If this is not needed, simply remove it from the mask and store in a safe place.
4. **Cleaning:** Wash with warm water and liquid dish soap. Rinse thoroughly then allow to dry on a clean, lint-free cloth and store. Do not place mask in dishwasher or autoclave.

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© 2014 PARI Respiratory Equipment, Inc.
PARI Respiratory Equipment, Inc.