
MONTESOLTM

PARI

Nasal Wash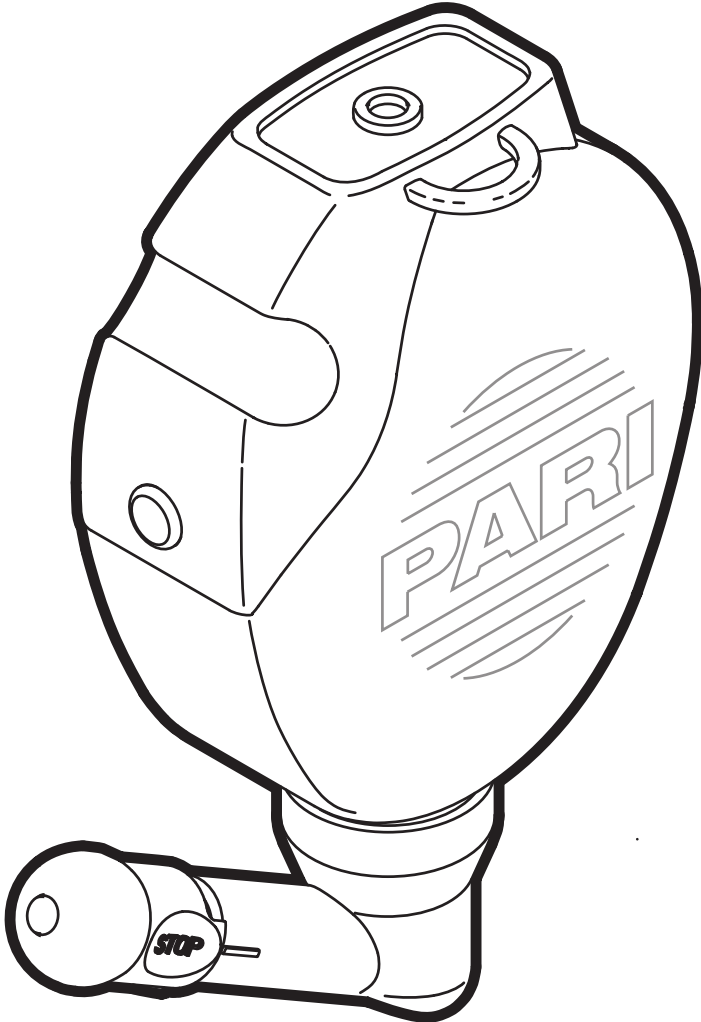

INSTRUCTIONS FOR USE

Read instructions and safety precautions before use.

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
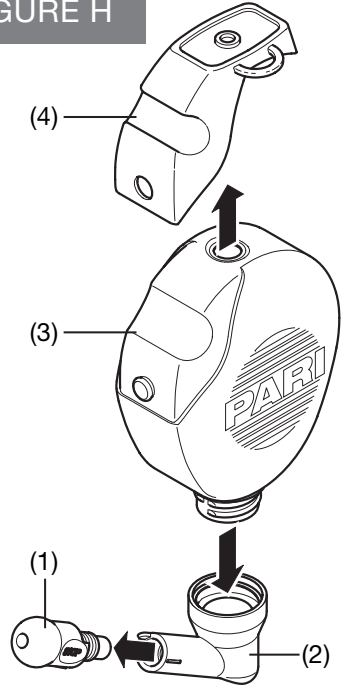
FIGURE G

FIGURE I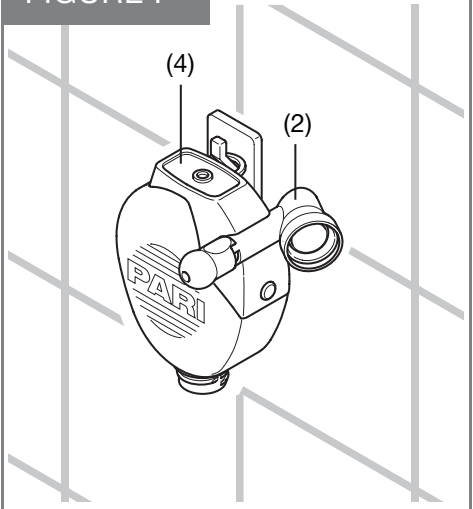

**Read these instructions for use before using for the first time.
Keep these instructions for use in a safe place.**

I. INTRODUCTION

The **PARI** Montesol™ Nasal Wash is a simple medical device used to wash or flush the nasal cavity. Nasal irrigation helps cleanse the nasal cavity of:

- Dirt, dust, and pollen
- Airborne allergens
- Excess mucous
- Crust associated with post-operative nasal and sinus surgery

WARNING

Nasal wash is for single patient use (one person) only. Do not share your nasal wash with other patients.

To reduce the risk of infection, illness, or injury from contamination, always follow proper procedures for cleaning this nasal wash.

It is recommended that you clean the **PARI** Montesol™ Nasal Wash before using for the first time.

DANGER

This product contains small parts that may present a choking hazard to small children. ALWAYS USE CLOSE ADULT SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.

II. SET-UP

The **PARI** Montesol™ Nasal Wash consists of 4 parts: (*Figure A*)

- (1) Nasal Insert
- (2) Bend
- (3) Reservoir
- (4) Valve

1. Firmly insert the nasal insert (1) into the bend (2) ensuring that the small pin (a) has engaged into slot (b). *Figure B*
2. Turn the nasal insert to the STOP position by aligning the STOP label on the nasal insert with the line marking on the bend. *Figure B*
3. Turn the nasal wash upside down so the valve (4) is at the bottom and unscrew the bend (2) from the reservoir (3). *Figure C*
4. Fill the nasal wash with normal saline solution or other irrigation solution as directed by physician up to the screw thread (8oz. or 250mL). *Figure D*
5. Screw the bend (2) firmly back onto the reservoir (3) and ensure that the nasal insert (1) is in the STOP position. *Figure E*

III. TREATMENT

1. Hold the nasal wash in your right hand (4). Turn the nasal insert (1) to the rinse position with your left hand. *Figure F*
2. Bend your head slightly forward over a sink and open your mouth wide. Breathe gently through your mouth. *Figure G*
3. Place the nasal insert over your right nostril so that the nostril is completely sealed. Turn your head slightly to the left.
4. Gently press the sides of the reservoir (3).

The nasal rinse solution should flow in one nostril, through the nasal cavity, and back out through the other nostril.

5. Repeat for left nostril.
6. Discard unused solution.

IV. CLEANING THE **PARI** MONTESOL™ NASAL WASH

1. Disassemble reusable nasal wash parts as illustrated in *Figure H*.
2. Wash all parts with warm water and liquid dish soap.
3. Rinse the nasal wash parts thoroughly with warm water and shake out water.
4. Air dry or hand dry nasal wash parts on a clean, lint-free cloth. Reassemble nasal wash when parts are dry and store in a clean place. You can also hang the nasal wash on a hook to dry as shown in *Figure I*.
5. You can also wash all parts of the nasal wash on the top shelf of a dishwasher. Place the parts in a dishwasher basket, then place the basket on the top rack of the dishwasher. Remove and dry the parts when the dishwasher cycle is complete.

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