


# MASK SET, SILICONE SIZE 3 WITH Y-PIECE

PART# 044F3301

## FOR USE WITH

**PARI** LC® Reusable Nebulizers (Part# 022F81 or 022F51)

Follow the Instructions for Use included with your PARI LC® Reusable Nebulizer. Instead of using the mouthpiece, use the Y-piece and mask set-up as described below. Attach Y-piece by pressing firmly onto nebulizer outlet (step 1). The expiratory valve on the Y-piece should face away from the patient. Attach mask by pressing firmly onto Y-piece as far as it will go (step 2).

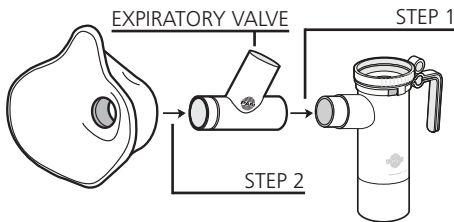

**⚠ WARNING** To reduce the risk of choking and asphyxiation, mask must be connected to the proper PARI Y-Piece with Expiratory Valve. Make sure the expiratory valve is not obstructed. Do not attach mask directly to nebulizer outlet. If the patient begins to choke, stop treatment immediately and do not continue until condition has been rectified. To reduce the risk of infection, follow cleaning, disinfecting and/or sterilizing instructions on the reverse side (page 2). This device contains small parts that might be a choking hazard if swallowed. Do not leave child unattended during treatment.



**PARI Respiratory Equipment, Inc.**

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 804.253.0246 | Product.Info@PARI.com

www.PARI.com

DS-44-331 Rev F 5-16

## CLEANING THE **PARI** Size 3 Mask with Y-Piece

### **⚠ WARNING**

To reduce the risk of infection, illness, or injury from contamination, you must thoroughly clean all parts of the nebulizer as instructed in the nebulizer IFU.

1. Remove the PARI Size 3 Mask with Y-Piece from the nebulizer and disassemble all parts.
2. Wash all parts of the PARI Size 3 Mask with Y-Piece with warm water and liquid dish soap.
3. Rinse thoroughly with warm water and shake out water.
4. Air dry or hand dry PARI Size 3 Mask with Y-Piece parts on a clean, lint free cloth.
5. You can also wash all parts of the PARI Size 3 Mask with Y-Piece in a dishwasher. Place the PARI Size 3 Mask with Y-Piece parts in a dishwasher basket, then place on the top rack of the dishwasher. Remove and dry the parts when the cycle is complete.

## DISINFECTING AND STERILIZING THE **PARI** SIZE 3 MASK WITH Y-PIECE

### **⚠ WARNING: TO REDUCE RISK OF INFECTION:**

1. PARI Size 3 Mask with Y-Piece for single patient use (one person) only. Do not share with other patients.
2. You must regularly disinfect or sterilize the PARI Size 3 Mask with Y-Piece between treatments. Failure to do so could lead to serious or fatal illness.

For patients with Cystic Fibrosis, the CF Foundation recommends boiling for a full 5 minutes to disinfect.

1. Clean the PARI Size 3 Mask with Y-Piece as described in section II. Every other treatment day, soak all parts of the PARI Size 3 Mask with Y-Piece in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. You can substitute respiratory equipment disinfectants (like Control III®) for distilled white vinegar (follow manufacturer's instructions for mixing). Rinse all parts of the PARI Size 3 Mask with Y-Piece thoroughly with warm tap water and dry with a clean, lint-free cloth. Discard the vinegar solution when disinfection is complete.
2. The PARI Size 3 Mask with Y-Piece parts may also be disinfected by boiling them in water for a full 5 minutes. Hospitals and clinics can sterilize the PARI Size 3 Mask with Y-Piece parts by autoclaving (a maximum of 277°F), follow autoclave manufacturer's instructions. Dry parts on a clean, lint-free cloth.