



eFlow<sup>®</sup>rapid  
nebuliser system

The eFlow<sup>®</sup>rapid  
with eBase Controller

With Display.

[www.eFlowrapid.info](http://www.eFlowrapid.info)

PARI Pharma  
Advancing Aerosol Therapies





**eFlow<sup>®</sup>rapid**  
nebuliser system  
Order No. 178G1005

The eFlow<sup>®</sup>rapid nebuliser handset and the eFlow<sup>®</sup>rapid aerosol head are compatible with both the dark blue Control Unit (eBase Controller) and the turquoise Control Unit (previous version).



**eFlow<sup>®</sup>rapid**  
nebuliser handset  
(incl. aerosol head)  
Order No. 678G8222



**eFlow<sup>®</sup>rapid** aerosol head  
Order No. 678B2620

**PARI**  
Export Service  
Centre

**+49 (0) 8151 279-220**

**Please give us a ring ...**

You can find PARI on the Internet:  
E-Mail: [info@pari.de](mailto:info@pari.de)  
[www.eflowrapid.info](http://www.eflowrapid.info)



**eFlow<sup>®</sup>rapid**  
nebuliser system

The eFlow<sup>®</sup>rapid  
with eBase Controller

**With Display.**

# The eFlow<sup>®</sup>rapid with eBase Controller



Compatible:



eFlow<sup>®</sup>rapid nebuliser handset e.g. with:

- Colistin
- Tobramycin
- DNase
- Hypertonic saline solution (MucoClear<sup>®</sup> 3% and MucoClear<sup>®</sup> 6%)

Compatible:



Drug-specific nebuliser handset:


- e.g. Altera<sup>®</sup> for Cayston<sup>®</sup> (aztreonam lysine)
- Other drug-specific nebuliser handsets used in multiple clinical trials

## Display Information


The eFlow<sup>®</sup>rapid with eBase controller can do much more:




**Battery charge level 50 %**  
-> Have new batteries on hand or operate with the power adapter connected

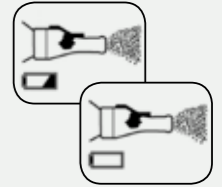

**Cable connection to the control unit or on the nebuliser handset interrupted**  
-> Check cable connections

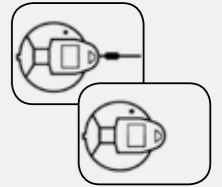

**No medication added**  
-> Add medication

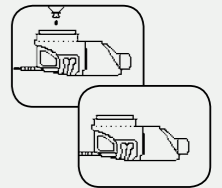

## Standards of Hygiene for the Nebuliser Handset\*

- Can be disinfected in steam disinfectant
- Can be disinfected by boiling in distilled water
- Autoclavable (up to 121 °C)

\*Please refer to the Instructions for Use.

## Worldwide Mobility – Small, Light, Quiet

- Can be run on rechargeable or disposable batteries
- Mains adapter for Europe, US & UK
- Weight approx. 300 g, incl. batteries
- Quiet and discrete to use
- 2 complete nebuliser handsets included to change whilst travelling